



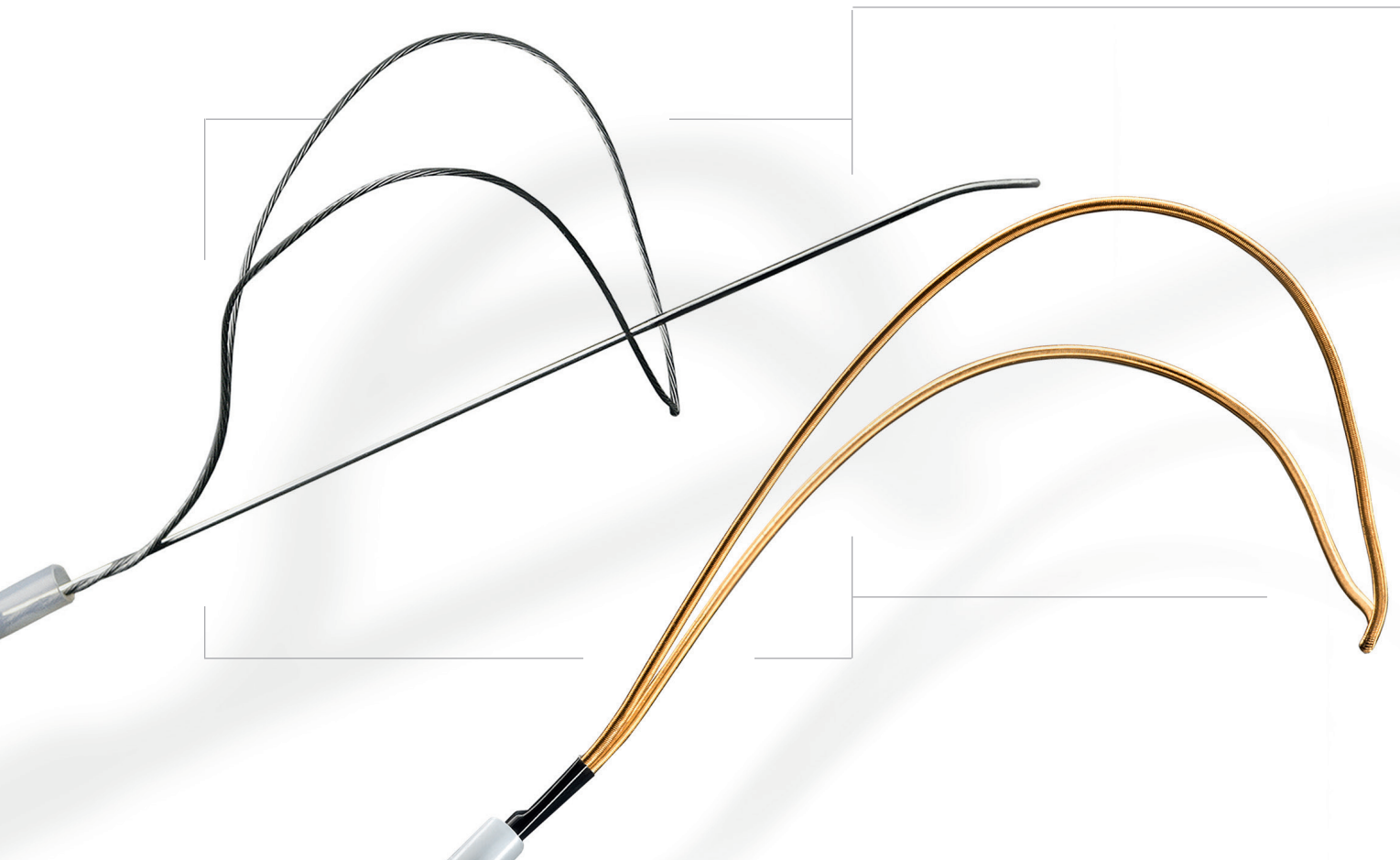

## AndraSnare®/LoopMaster Sochman Snare®

Gold plated atraumatic 3D nitinol snare /  
Atraumatic 3D nitinol snare for foreign body retrieval

# AndraSnare® & AndraSnare Micro®

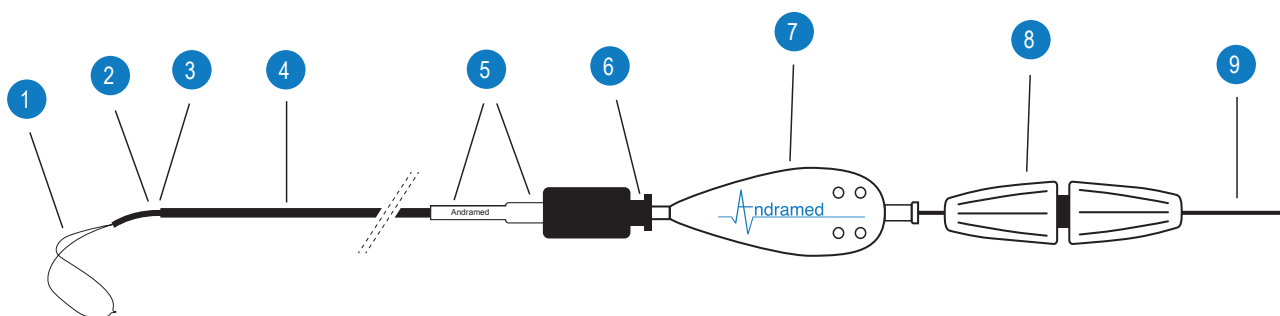

- |                                                                                    |                                                                     |
|------------------------------------------------------------------------------------|---------------------------------------------------------------------|
| 1 Atraumatic nitinol loop with gold coating for better visualization in the X-rays | 6 Luer connection                                                   |
| 2 Pre-formed tip for better navigation                                             | 7 Insertion part for simple insertion of the loop into the catheter |
| 3 X-ray marker at the shaft output                                                 | 8 Large torquer for good operation                                  |
| 4 Shaft visible in X-rays                                                          | 9 Continuous nitinol wire for 1:1 control of the loop               |
| 5 Bend protection for shaft reinforcement at the Luer                              |                                                                     |

## Order Codes AndraSnare®


| Order Code | Loop Ø in mm | Vessel Ø in mm | Loop/Guidewire Length in cm | Introducer Sheath Length in cm | Introducer Sheath Ø in Fr |
|------------|--------------|----------------|-----------------------------|--------------------------------|---------------------------|
| AS-5       | 5            | 3 – 5          | 125                         | 110                            | 4                         |
| AS-10      | 10           | 6 – 10         | 125                         | 110                            | 4                         |
| AS-15      | 15           | 11 – 15        | 125                         | 110                            | 4                         |
| AS-20      | 20           | 16 – 20        | 125                         | 110                            | 4                         |
| AS-25      | 25           | 21 – 25        | 125                         | 110                            | 5                         |
| AS-25/65   | 25           | 21 – 25        | 65                          | 50                             | 5                         |
| AS-25/150  | 25           | 21 – 25        | 150                         | 135                            | 5                         |
| AS-30      | 30           | 26 – 30        | 125                         | 110                            | 5                         |
| AS-35      | 35           | 31 – 35        | 125                         | 110                            | 5                         |

## Order Codes AndraSnare Micro®

| Order Code | Loop Ø in mm | Vessel Ø in mm | Loop/Guidewire Length in cm | Introducer Sheath Length in cm | Introducer Sheath Ø in Fr |
|------------|--------------|----------------|-----------------------------|--------------------------------|---------------------------|
| ASM-2      | 2            | 1 – 2          | 175                         | 150                            | 2,3 – 3                   |
| ASM-4      | 4            | 3 – 4          | 175                         | 150                            | 2,3 – 3                   |
| ASM-7      | 7            | 5 – 7          | 175                         | 150                            | 2,3 – 3                   |

Packaging Unit: 1


# LoopMaster® Sochman Snare

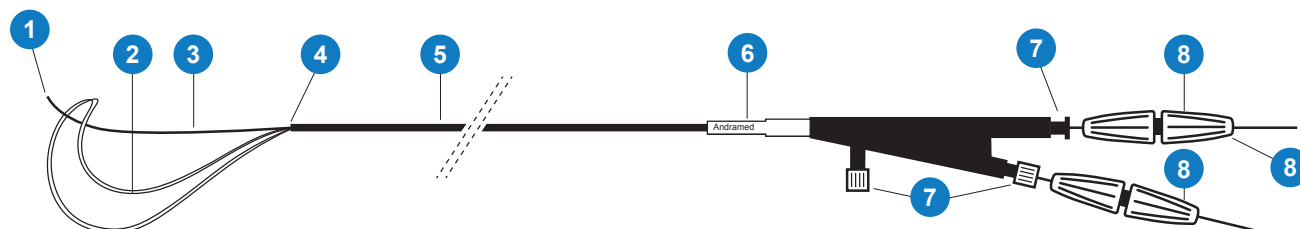


- 1 Atraumatic distal end at the catching wire
- 2 Nitinol loop
- 3 Catching wire
- 4 X-ray marker
- 5 Flexible shaft

- 6 Bend protection at the Luer
- 7 Luer connector
- 8 Large torquers for simple control of the catching snare and the catching wire


## Retrieval technique for foreign bodies (see graphics above)

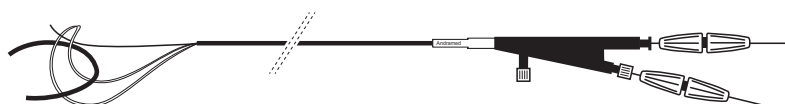

- 1 Open the snare and place it around the foreign body to be removed.

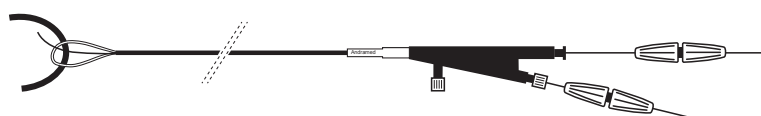

- 2 By closing the snare, the foreign body is enclosed between the catching wire and the catching snare.

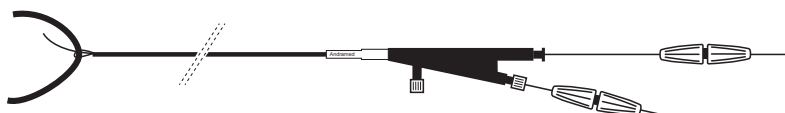

- 3 This makes it possible to remove the foreign body securely from the body.

## Order Codes LoopMaster® Sochman Snare

| Order Code | Loop<br>Ø in mm | Loop/Guidewire<br>Length in cm | Introducer Sheath<br>Length in cm | Introducer Sheath<br>Ø in Fr |
|------------|-----------------|--------------------------------|-----------------------------------|------------------------------|
| ASL-25     | 25              | 130/150                        | 110                               | 8                            |

Packaging Unit: 1

## **Andramed GmbH**

Schiesswiesenstr. 18  
72766 Reutlingen, Germany  
Tel.: +49 (0) 7127 811 57 15  
Fax: +49 (0) 7127 811 57 11  
[info@andramed.com](mailto:info@andramed.com)  
[www.andramed.com](http://www.andramed.com)

Made in Germany  
Rev. 02/2019

