Rüsch
TracheoQuick
Emergency Coniotomy Set

Safe quick airway access

Where there is a danger of asphyxiation, prompt, resolute, professional and safe action is essential to save lives. In this situation, the Rüsch TracheoQuick Emergency Coniotomy Sets from Teleflex are your reliable partners. Safe, short-term ventilation can be achieved to stabilise the patient until a long-term solution can be implemented in a controlled environment.

The sterile TracheoQuick Sets are clever, convenient units containing all necessary tools for a fast intervention when intubation or ventilation are no longer possible.

The TracheoQuick Coniotomy Tube is available in two sizes, for adults and children.

Safe and gentle

The TracheoQuick Coniotomy Set consists of a pre-assembled coniotomy unit with a one-way syringe attached to a comfort neck band, allowing fast attachment to the flange. The stainless steel introduction dilator is conical for a safe and precise coniotomy. A pre-mounted stopper prevents the needle from being too deeply inserted which minimises the risk of posterior tracheal wall perforation.

Safety Plus

The TracheoQuick Plus Set offers a coniotomy tube with cuff, which seals the trachea securely for efficient ventilation with aspiration protection.

Benefits

The Clinician

Quick, safe and efficient intervention to secure short-term airway access


The Patient

Immediate, life saving ventilation in the presence of acute respiratory distress

TracheoQuick Set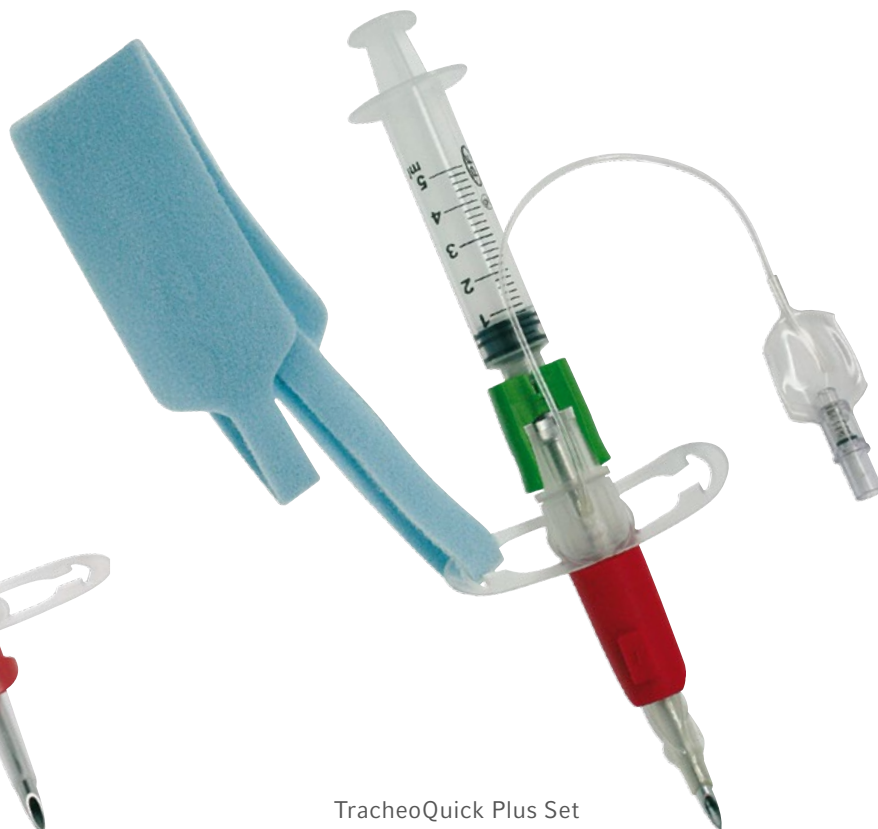

TracheoQuick Plus Set

TracheoQuick Set Fast Facts

Pre-assembled emergency coniotomy set, with all necessary components, ready to use

Conical stainless steel introduction dilator for a safe and precise coniotomy

Pre-mounted stopper to minimise risk of posterior wall perforation

Not made of natural rubber latex

Sterile
Single use

TracheoQuick Set

REF.	ORDER SIZE/I.D.	O.D.	QTY
120600-000020	2.0	2.8	1
120600-000040	4.0	4.8	1

TracheoQuick Plus Set

REF.	ORDER SIZE/I.D.	O.D.	QTY
120604-000040	4.0	5.0	1

Set Components: TracheoQuick Set

- 1 x Coniotomy tube (PVC)
- 1 x Flexible silicone tube extension with 15 mm standard connectors
- 1 x Scalpel
- 1 x Pre-assembled unit of disposable syringe with tube, fixation flange and attached comfort neck band

Set Components: TracheoQuick Plus Set

- 1 x Coniotomy tube with cuff (PVC)
- 1 x Flexible silicone tube extension with 15 mm standard connectors
- 1 x Scalpel
- 1 x Pre-assembled unit of disposable syringe with tube, fixation flange and attached comfort neck band

Teleflex, the Teleflex logo, Rüsch and TracheoQuick are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries. Information in this document is not a substitute for the product Instructions for Use. The products in this catalogue may not be available in all countries. Please contact your local representative. All data current at time of printing (03/2016). Subject to technical changes without further notice. © 2016 Teleflex Incorporated. All rights reserved. 94 08 25 - AP 00 01 • REV A • 03 16 PDF

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
Simplified procedure

Simplified procedure for using the RÜSCH™ TracheoQuick™


The RÜSCH™ TracheoQuick™ is a percutaneous cricothyrotomy set for securing the airway in case of upper airway obstruction. This document does not substitute the instruction for use. Read the IFU before using.

Use size 2.00mm (I.D.) for children and size 4.00mm (I.D.) for adults.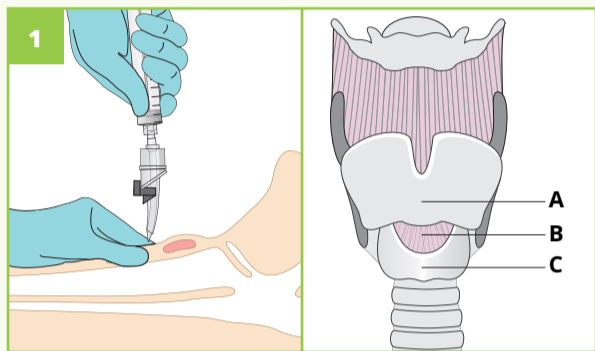


- Extend the neck of the patient
- Locate the cricothyroid membrane (b) by the palpable depression between the thyroid cartilage (a) and cricoid cartilage (c)
- Stabilise this site with index finger and thumb
- Puncture the cricothyroid membrane (b) perpendicular to the trachea


The posterior tracheal wall can be injured by inserting the RÜSCH™ TracheoQuick™ too deeply or by removing the stopper too early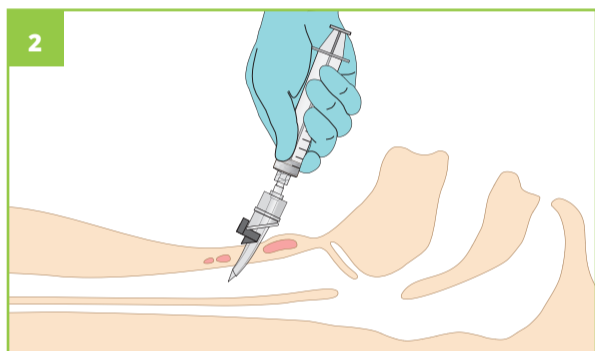

- Insert the RÜSCH™ TracheoQuick™ in caudal direction to the stopper
- Aspirate air with the syringe to determine the position of the cannula. A syringe pre-filled with e.g. normal saline (2 - 3 ml) can be helpful. If aspiration of air is possible, the needle tip is in the trachea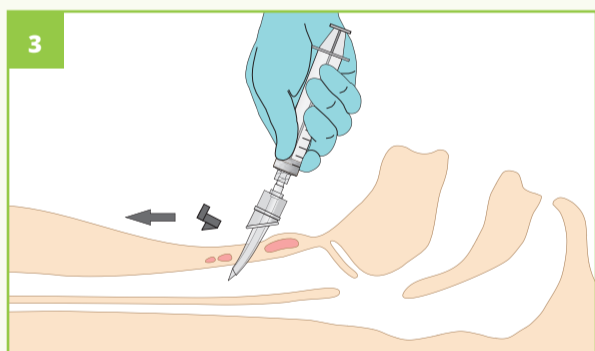


- Remove the stopper from the cannula

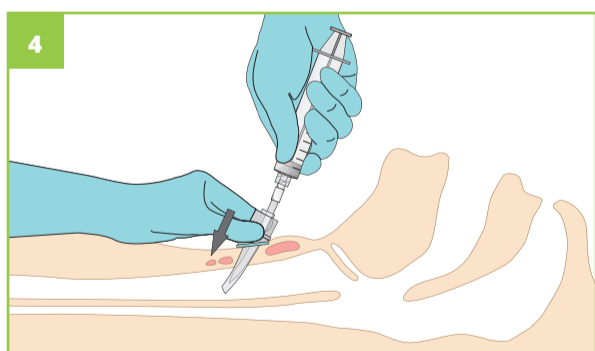- While holding the needle in position, advance the cannula along the needle until the flange is flush with the patient's neck
- Remove the needle, while holding the cannula in position
- Ventilate the patient directly via the 15 mm connector or via the included extension tubing for enhanced flexibility

The needle serves as a guide and must not be inserted further

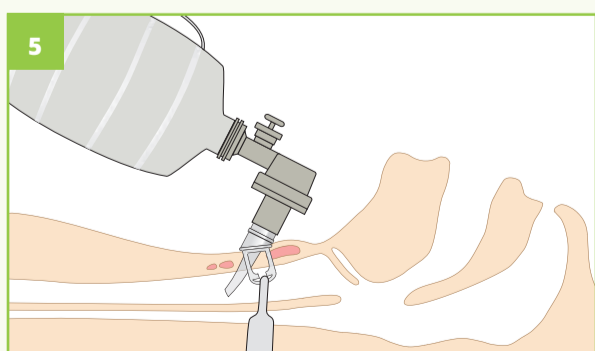- Secure the cannula with the neckband

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Sediul Teleflex International, Irlanda · Teleflex Medical Europe Ltd. · IDA Business & Technology Park Dublin Road ·