


THE STRUCTURE OF THE CHESTPIECE

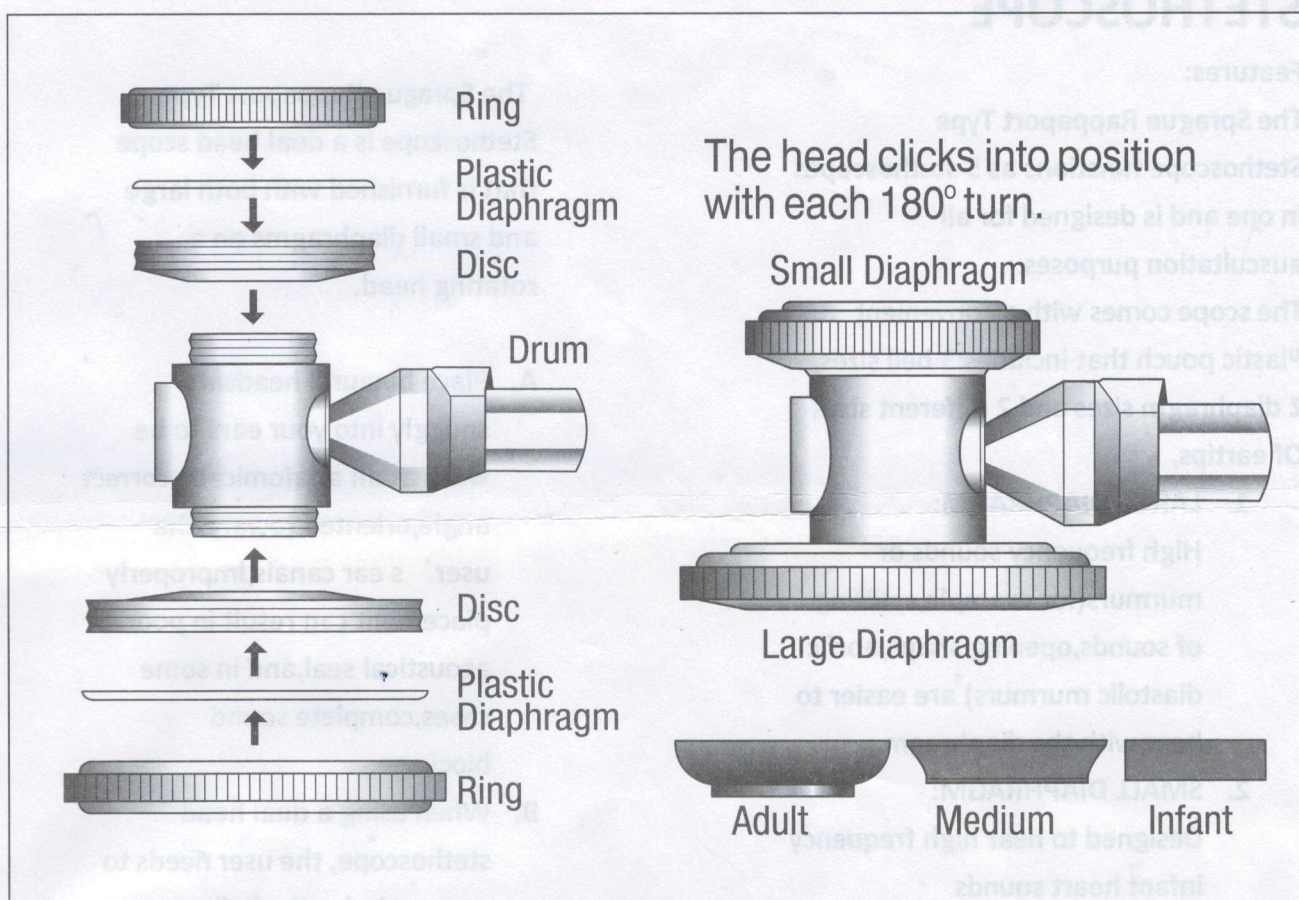

Sprague Rappaport Type Improved Stethoscope comes as a Complete set with the following components:

1. Chestpiece with large and small diaphragm.....1 pc.
2. Binaurals, chrome plated with white plastic medium size eartips.....1 pc.
3. Heavy wall tubing with metal clip.....1 pc.
4. Interchangeable bell type chestpiece, adult size.....1 pc.
5. Interchangeable bell type chestpiece, medium size.....1 pc.
6. Interchangeable bell type chestpiece, infant size.....1 pc.
7. Spare eartips, large and small size.....2 pairs
8. Spare transparent plastic diaphragms, large and small.....2 pcs.
9. Plastic container for accessories.....1 pc.

5-WAY CONVERTIBLE STETHOSCOPE

HOW TO USE

Features:

The Sprague Rappaport Type

Stethoscope functions as 5 stethoscopes in one and is designed for all auscultation purposes.

The scope comes with a convenient

Plastic pouch that includes: 3 bell sizes, 2 diaphragm sizes and 2 different sizes

Of eartips,

1. LARGE DIAPHRAGM:

High frequency sounds or murmurs (for example, splitting of sounds, opening snaps, aortic diastolic murmurs) are easier to hear with the diaphragm,

2. SMALL DIAPHRAGM:

Designed to hear high frequency infant heart sounds

3. ADULT BELL:

The bell, which should be applied lightly to the chest, transmits low frequency sounds more effectively—for example, diastolic murmur of mitral stenosis and third and fourth heart sounds,

4. MEDIUM SIZE BELL:

Used to auscultate low and medium pitched sounds in narrow locations such as between ribs

5. INFANT BELL:

Used for infants and young children to auscultate low and medium pitched sounds and murmurs

The Sprague Rappaport Type

Stethoscope is a dual head scope that is furnished with both large and small diaphragms on a rotating head.

A. Place binaural headsets snugly into your ears to be worn at an anatomically correct angle, oriented toward the user's ear canals. Improperly placement can result in poor acoustical seal, and in some cases, complete sound blockage.

B. When using a dual head stethoscope, the user needs to open, or index the bell or diaphragm by rotating the chest piece, if the diaphragm is open, the bell will be closed, preventing sound from coming in through the bell, and vice versa. The

Sprague Rappaport Type stethoscope ships with the adult and pediatric diaphragms. Bells are in the convenient plastic pouch,

C. Proceed with the client examination



KANGSHUN Medical Corporation

Sprague Rappaport Type Stethoscope

22' Double Tubing
2 Diaphragms
2 Pairs of eartips
3 Bells



CE