

## Infinity® BISx® SmartPod® Patient Monitoring Pods

Incorporating the extended platform (XP) technology from Aspect Medical Systems, the Infinity® BISx® SmartPod® provides sustained, reliable BIS measurement in the OR, ICU and other clinical settings.



## Benefits

---

### Uses Bispectral Index® (BIS) technology

The Infinity BISx SmartPod incorporates BISx smart technology into Infinity patient monitors. The Bispectral Index® (BIS), a processed EEG parameter, can indicate the effects of sedation on the brain. The BISx pod allows you to measure BIS index, Signal Quality Index, Burst Suppression Ratio, Electromyography (EMG), Total Power, Spectral Edge Frequency, and Burst Count. One EEG waveform can be displayed.

The pod easily interfaces to Infinity Delta, Delta XL, and Kappa monitors via a standard pod communication channel.

### Allows anesthesia to be delivered with more precision

Data gathered by the Infinity BISx SmartPod helps you deliver anesthesia with more precision, and enables you to assess and respond to a patient's changing condition during anesthesia.

### Supports patient care in the OR, ICU and more

Incorporating the extended platform (XP) technology from Aspect Medical Systems, the Infinity BISx SmartPod provides sustained, reliable BIS measurement in the OR, ICU and other clinical settings.

BISx® and Bispectral Index® are registered trademarks of Aspect Medical Systems, Inc.

## Technical Data

### Measuring Capabilities

Displayed parameters	BIS Index, Signal Quality Index, Burst Suppression Ratio, EMG, Total Power, Spectral Edge Frequency, Burst Count
----------------------	--

#### BIS Index (BIS)

Measurement range	0 to 100%
Display resolution	1%

#### Signal Quality Index (SQI)

Measurement range	0 to 100%
Display resolution	1%

#### Burst Suppression Ratio (BSR)

Measurement range	0 to 100%
Display resolution	1%

#### Electromyography (EMG)

Measurement range	0 to 100 dB in the frequency range from 70 to 110 Hz
Display resolution	1 dB

#### Total Power (PWR)

Measurement range	40 to 100 dB
Display resolution	1 dB in the frequency range 0.5 to 30 Hz

#### Spectral Edge Frequency (SEF)

Measurement range	0.5 to 30 Hz
Display resolution	0.1 Hz

#### Burst Count (BCT)

Measurement range	0 to 30
Display resolution	1

### PRODUCT SPECIFICATIONS

#### Physical Specifications

Size (H x W x D) (including clip)	95.3 x 114.3 x 63.5 mm (3.75 x 4.5 x 2.5 in.)
Weight	226.8 g (8 oz) without pod communication cable
Protection against electric shock	Type BF
Connections	PodCom for connection to monitor
Power source	Powered directly from monitor

#### Environmental Requirements

##### Temperature Range

Operating	10 °C to 40 °C (50 °F to 104 °F)
Storage	-20 °C to 40 °C (-4 °F to 104 °F)

##### Relative Humidity

Operating	20% to 90%, non-condensing
Storage	20% to 90% with packaging

##### Atmospheric Pressure

Operating	525 to 795 mmHg (70 to 106 kPa)
Storage	375 to 795 mmHg (50 to 106 kPa)

## Technical Data

### Standards

IEC 60601-1 and applicable particular and collateral standards,  
IEC 60601-1-2, Electromagnetic compatibility CISPR 11, Class B,  
EEG standard: IEC 60601-2-26. Infinity BISx SmartPod complies with the  
Medical Devices Directive (MDD) 93/42 EEC and bears the CE mark.

## Ordering Information

Infinity® BISx® SmartPod®	MS14796
BIS Quatro Sensor Starter Kit	MS15115
BIS Pediatric Sensor Starter Kit	MS15116

The BISx® pod is compatible with Infinity Delta, Delta XL or Kappa monitors with VF5 software or higher.

BIS, BISx and Bispectral Index are trademarks of Aspect Medical Systems, Inc., and are registered in the USA, EU and other countries.

The Infinity® BISx® SmartPod® is manufactured by Aspect Medical Systems.

Contact your local Draeger sales representative for further ordering information.

Not all products, features, or services are for sale in all countries.  
Mentioned Trademarks are only registered in certain countries and not necessarily in the country in which this material is released. Go to [www.draeger.com/trademarks](http://www.draeger.com/trademarks) to find the current status.

### CORPORATE HEADQUARTERS

Draegerwerk AG & Co. KGaA  
Moislinger Allee 53-55  
23558 Lübeck, Germany  
[www.draeger.com](http://www.draeger.com)

### Manufacturer:

Draegerwerk AG & Co. KGaA  
Moislinger Allee 53-55  
23558 Lübeck, Germany

### USA

Draeger, Inc.  
3135 Quarry Road  
Telford, PA 18969-1042, USA  
Tel +1 800 4DRAGER  
(+1 800 437 2437)  
Fax +1 215 723 5935  
[info.usa@draeger.com](mailto:info.usa@draeger.com)

### CANADA

Draeger Medical Canada Inc.  
2425 Skymark Avenue, Unit 1  
Mississauga, Ontario, L4W 4Y6  
Tel +1 905 212 6600  
Toll-free +1 866 343 2273  
Fax +1 905 212 6601  
[Canada.support@draeger.com](mailto:Canada.support@draeger.com)

Locate your Regional  
Sales Representative at:  
[www.draeger.com/contact](http://www.draeger.com/contact)

