


IFU for Sterilant RENO-SA10

* For use in RENO-S130 and RENO-S130D

Product Specifications


1. Intended Use: It is a sterilant when medical products are sterilized in the low temperature plasma sterilizers made by Renosem. Co., Ltd.
2. Storage condition: The sterilants should be stored in a room temperature of 5°C~30°C.
3. Expiry date: 12 months
4. Capacity: 4 ml/capsule(50%)

External Appearance and structure

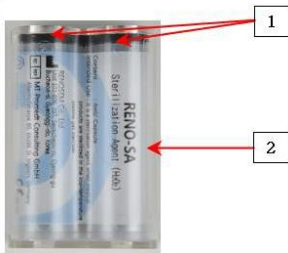

No.	Part name	Fuction
1	Syringe rubber cap	As an inlet for sterilant
2	Syringe body	To store sterlant

How to insert the sterilant

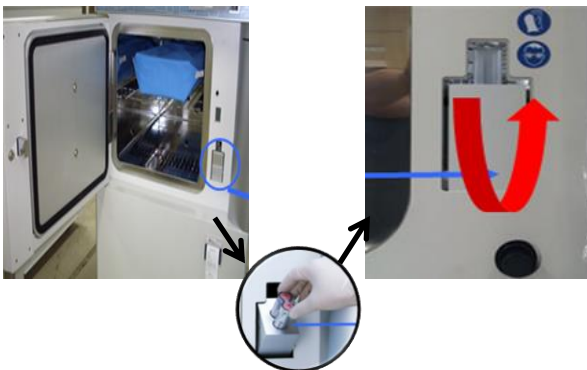

1. Take out the inlet of sterilant by pressing the button located on the right lower part of chamber.
2. Place the RENO-SA10 into the hole by holding the red rubber part.
3. Check if blue lamp turns on on the right upper part and push the inlet back to position.
4. Close the door.

Outer box Package

1. 15 ea (per box)
2. Material: Carton
3. Size: 185W x 155L x 119H (mm)



Caution



1. Hazard statement
2. Precautionary statement
3. Please read the RENO sterilizer operator's manual for details on warnings.
4. Do not use this sterilant for any uses other than the original purpose.
5. This sterilant is an accessory for the low-temperature plasma sterilizers for medical instruments.

Pictograms

1. Follow the instructions for use
2. To be used before YY/MM
3. Date of manufacture
4. EC representative

Renosem Co., Ltd.

Unit 103-806, 397, Seokcheon-ro
Bucheon, Gyeonggi
14449
South Korea

T: +82 32 222 0143
F: +82 32 222 0110