

Specification



Feature	Specification
Technology	Biosensor/Electrochemical, Glucose oxidase (GOD)
Result Calibration	Plasma-equivalent
Test Time	10 seconds
Sample Size	0.5 µL
Sample Type	Fresh capillary whole blood
Hematocrit Range	25 - 60%
Glucose Test Range	20 - 600 mg/dL (1.1 - 33.3 mmol/L)
Memory Storage	300 results with date and time
Test Averaging	7, 14, 30-day averages
Data Transfer	USB
Control Solution	3 levels
Audio Feature	Optional beep for sample detection, error messages
Automatic Shutoff	2 minutes after last action
Battery	One (1) CR 2032 3.0V coin cell battery
Battery Life	1,000 measurements
Operating Conditions	41 - 113 °F (5 - 45°C) and 10 - 90% relative humidity
Strip Storage Temperature	2-35°C
Expiration Date	24 months (6 months after first opening)


Catalog

Product Name	Catalog No.	Contents			
On-Call [®] Plus Blood Glucose Monitoring System	G113-111 v †	1 Meter 1 Manual 10 Lancets	10 Test Strips 1 Carrying Case 1 Code Chip	1 Control Solution 1 1 Quick Reference Guide 1 Clear Cap (for testing on forearm and palm)	1 Lancing Device 1 Warranty Card
On-Call [®] Plus Blood Glucose Meter	G113-211 v †	1 Meter 1 Manual	1 Control Solution 1 1 Warranty Card	1 Carrying Case 1 Quick Reference Guide	
	G113-214 v	1 Meter 1 Manual 10 Lancets	1 Lancing Device 1 Carrying Case 1 Warranty Card	1 Control Solution 1 1 Quick Reference Guide 1 Clear Cap (for testing on forearm and palm)	
On-Call [®] Plus Blood Glucose Test Strips	G133-111 v †	50 Test Strips (25/vial)		1 Code Chip	1 Package Insert
		50 Test Strips (50/vial)		1 Code Chip	1 Package Insert
	G133-112 v	100 Test Strips (25/vial)		1 Code Chip	1 Package Insert
	G133-114 v	10 Test Strips (10/vial)		1 Code Chip	1 Package Insert
	G133-115 v	25 Test Strips (Individually Foil Wrapped)		1 Code Chip	1 Package Insert
	G133-117 v	50 Test Strips (Individually Foil Wrapped)		1 Code Chip	1 Package Insert
G133-118 v	25 Test Strips (25/vial)		1 Code Chip	1 Package Insert	
On-Call [®] Plus Blood Glucose Test Strips and Lancets	G133-211 v	50 Test Strips (25/vial)	50 Lancets (25/bag)	1 Code Chip	1 Package Insert
On-Call [®] Plus Blood Glucose Control Solution	G123-311 v †	1 Control Solution 0	1 Control Solution 1	1 Control Solution 2	1 Package Insert
On-Call [®] Lancets	G124-10A v †	100 Lancets (25/bag)			
On-Call [®] Lancing Device	G124-11AV	1 Lancing Device		1 Package Insert	
On-Call [®] Diabetes Management Software Kit	G124-13A †	1 USB Data Transfer Cable		1 Installation Disk	

v CE Marked for sale in the European Community 0123 † US 510(k) Cleared and CLIA Waived



On-Call[®] Plus
Blood Glucose Monitoring System

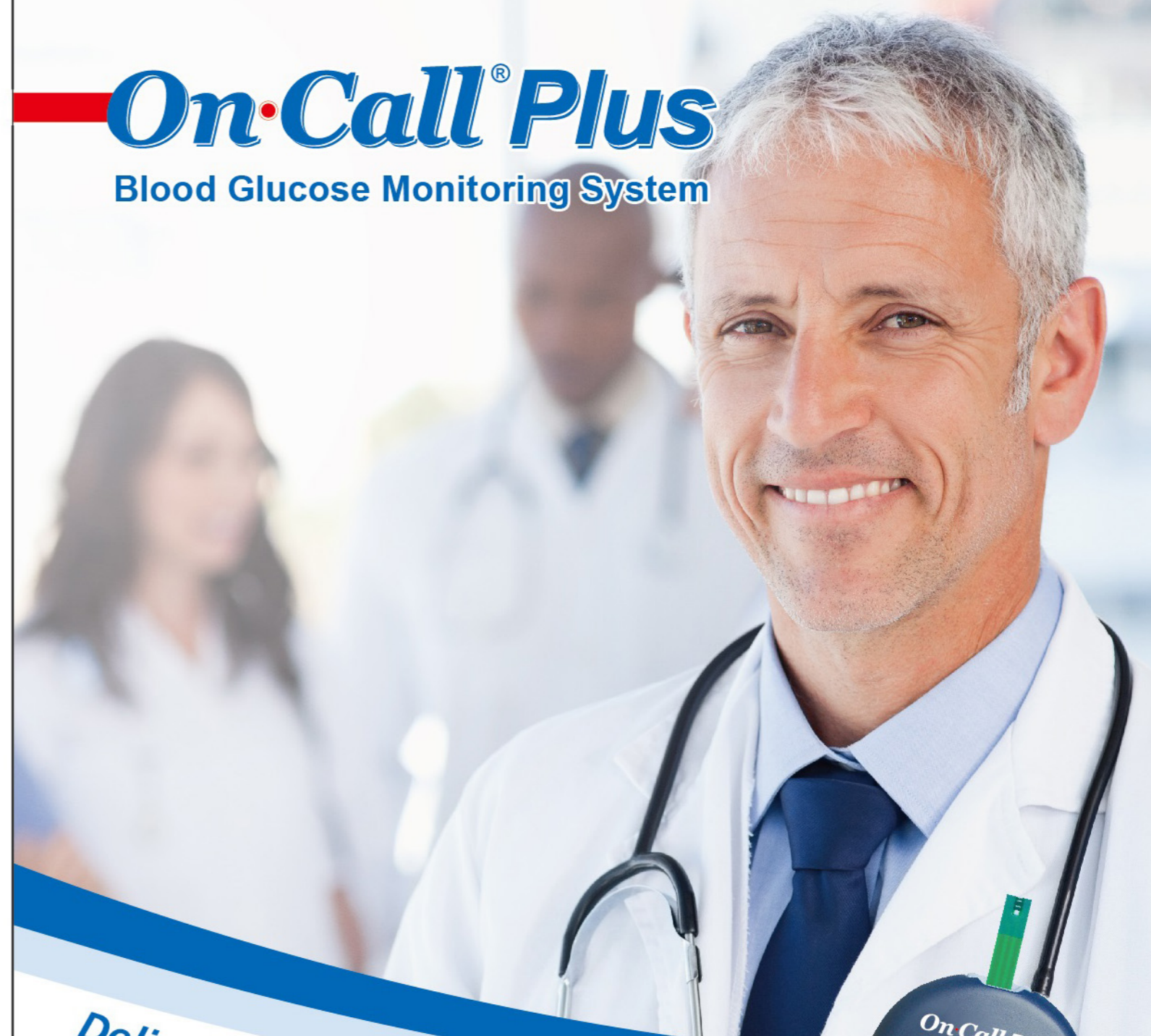


Delivers Value and Quality

- 0.5 µL Blood Sample
- Accurate & Reliable Results
- 25 - 60% HCT Range
- US 510(k) & CE




Accurate and Reliable


Extensive clinical studies proved the accuracy of *On-Call Plus* Blood Glucose Monitoring System with fresh capillary blood samples, which can comply with EN ISO 15197: 2015.

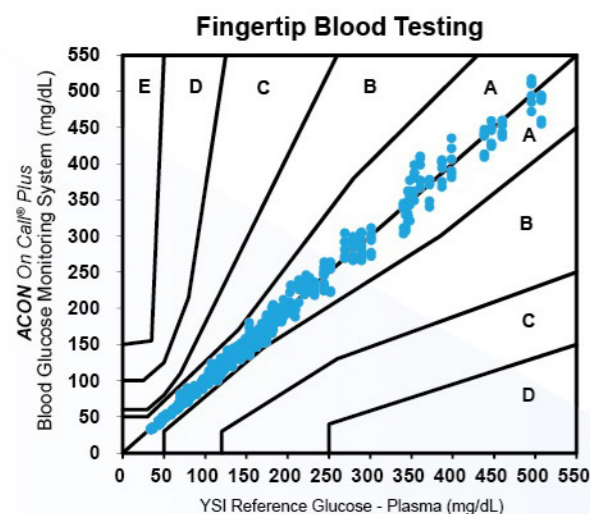

Consensus Error Grid Analysis Clinical Trial - Fingertip Capillary Blood, by Technican ACON On-Call Plus Blood Glucose Monitoring System vs. YSI		
System Accuracy Results for Glucose Concentration ≥ 100 mg/dL		
Within $\pm 5\%$	Within $\pm 10\%$	Within $\pm 15\%$
290 / 462 (62.8%)	432 / 462 (93.5%)	462 / 462 (100.0%)
System Accuracy Results for Glucose Concentration < 100 mg/dL		
Within ± 5 mg/dL	Within ± 10 mg/dL	Within ± 15 mg/dL
145 / 198 (73.2%)	193 / 198 (97.5%)	198 / 198 (100.0%)
System Accuracy Results for both Glucose Concentration ≥ 100 mg/dL and < 100 mg/dL		
Within $\pm 15\%$ or ± 15 mg/dL		
658 / 660 (99.7%)		

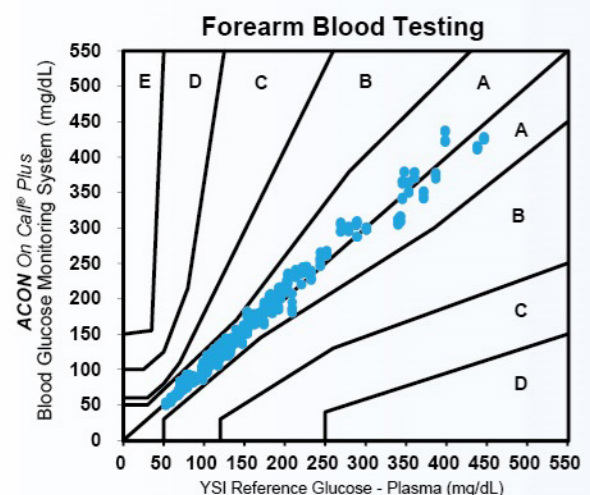

Consensus Error Grid Analysis Clinical Trial - Forearm Capillary Blood, by Technican ACON On-Call Plus Blood Glucose Monitoring System vs. YSI		
System Accuracy Results for Glucose Concentration ≥ 100 mg/dL		
Within $\pm 5\%$	Within $\pm 10\%$	Within $\pm 15\%$
202 / 444 (45.5%)	375 / 444 (84.5%)	440 / 444 (99.1%)
System Accuracy Results for Glucose Concentration < 100 mg/dL		
Within ± 5 mg/dL	Within ± 10 mg/dL	Within ± 15 mg/dL
110 / 168 (65.5%)	154 / 168 (91.7%)	168 / 168 (100.0%)
System Accuracy Results for both Glucose Concentration ≥ 100 mg/dL and < 100 mg/dL		
Within $\pm 15\%$ or ± 15 mg/dL		
608 / 612 (99.3%)		

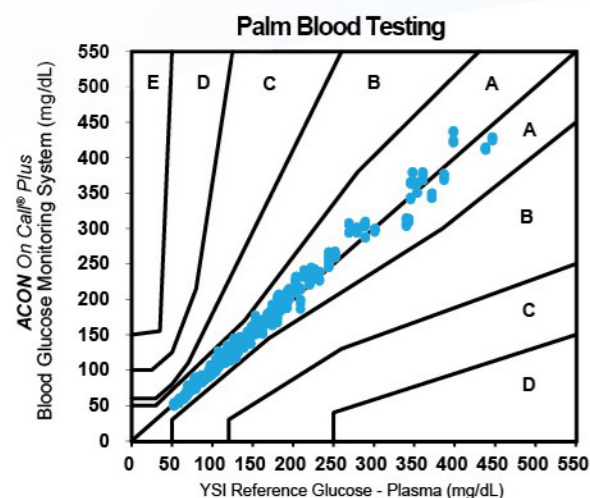

Consensus Error Grid Analysis Clinical Trial - Palm Capillary Blood, by Technican ACON On-Call Plus Blood Glucose Monitoring System vs. YSI		
System Accuracy Results for Glucose Concentration ≥ 100 mg/dL		
Within $\pm 5\%$	Within $\pm 10\%$	Within $\pm 15\%$
219 / 444 (49.3%)	395 / 444 (89.0%)	441 / 444 (99.3%)
System Accuracy Results for Glucose Concentration < 100 mg/dL		
Within ± 5 mg/dL	Within ± 10 mg/dL	Within ± 15 mg/dL
130 / 168 (77.4%)	166 / 168 (98.8%)	168 / 168 (100.0%)
System Accuracy Results for both Glucose Concentration ≥ 100 mg/dL and < 100 mg/dL		
Within $\pm 15\%$ or ± 15 mg/dL		
609 / 612 (99.5%)		

Key Features

0.5 μ L blood sample

HCT 25 - 60% HCT range

2 - 35°C strip storage temperature

Optional individually packaged test strips available

Alternative testing sites including fingertip, forearm and palm

Automatic detection of insufficient sample

300 test memory with date and time

7, 14, 30 - day averages calculation

Easy PC data transfer and smart App data analysis

Authority Certificate





Benannt durch/Designated by
Zentralstelle der Länder
für Gesundheitsschutz
bei Arzneimitteln und
Medizinprodukten
www.zlg.de
ZLG-BS-245.10.07



Product Service

EC Certificate

Full Quality Assurance System
Directive 98/79/EC on In Vitro Diagnostic Medical Devices (IVDD), Annex IV excluding (4, 6)
(List A and B and devices for self-testing)

No. V1 104507 0003 Rev. 01

For the product(s)/product category (ies):

On Call Plus Blood Glucose Monitoring System,
On Call Plus Blood Glucose Test Strips,
On Call EZ II Blood Glucose Monitoring System,
On Call Redi Blood Glucose Monitoring System,
On Call Redi II Blood Glucose Test Strips,
On Call Advanced Blood Glucose Monitoring System,
On Call Advanced Blood Glucose Test Strips,
On Call Platinum Blood Glucose Monitoring System,
On Call Platinum Blood Glucose Test Strips,
On Call Chosen Blood Glucose Monitoring System,
On Call Chosen Blood Glucose Test Strips,
On Call Vivid Blood Glucose Monitoring System (OGM-101),
On Call Vivid Blood Glucose Test Strips (OGS-101),
On Call Vivid Pal Blood Glucose Monitoring System (OGM-102),
On Call Sharp Blood Glucose Monitoring System (OGM-121),
On Call Sharp Blood Glucose Test Strips (OGS-121)
On Call Plus II Blood Glucose Monitoring System (OGM-171),
On Call Plus II Blood Glucose Test Strips (OGS-171),
On Call Extra Blood Glucose Monitoring System (OGM-191),
On Call Extra Blood Glucose Test Strips (OGS-191),
On Call GK Dual Blood Glucose & Ketone Monitoring System (OGM-161),
On Call Blood Ketone Test Strips (OGS-161),
D-ONE Blood Glucose Monitoring System,
D-ONE Blood Glucose Test Strips,
Urinalysis Reagent Strips (Urine),
UTI Urinary Tract Infection Test Strips,
Toxoplasma IgG EIA Test Kit,
Toxoplasma IgM EIA Test Kit,
Rubella IgG EIA Test Kit,
Rubella IgM EIA Test Kit,
CMV IgG EIA Test Kit,

Page 3 of 4

TÜV SÜD Product Service GmbH is Notified Body with identification no. 0123

TÜV SÜD Product Service GmbH • Certification Body • Ridlerstraße 65 • 80339 Munich • Germany

TÜV®

