



Technical Data Sheet

**Sterile Hypodermic Syringes for Single Use**

Revision : 5

Date : 2019-07-01

Page 1 of 2


Manufacturer:  
Changzhou Jiafeng Medical  
Equipment Co., Ltd

**2ML Syringe**  
**\*With needle**  
\*\* without needle

Product Structure  
1. Core Pin  
2. Barrel  
3. Plunger  
4. Needle Seat  
5. Needle tube  
6. Protective cover



Product variants	Ref	Description	Luer type
	<b>SS311022325</b>	<b>2ML with needle 23G*1"</b>	<b>Luer slip</b>
	<b>SS311022330</b>	<b>2ML with needle 23G*1 1/4"</b>	<b>Luer slip</b>
Intended use	For hypodermic drug injection giving , extraction.		
Materials	PP, NR, stainless steel, silicone oil (Syringes are PVC- and DEHP-free)		
Needle	Designed with optimal point geometry for maximum comfort and reliability, thin-walled stainless steel, trilateral sharpened and ultrasound polished. Needle surface coated with special grease.		
Sterilization	<b>Ethylene oxide (E.O)</b>		
Shelf life	60months		
Product standards	ISO 7886-1: 2017 , ISO 7864: 2016		
Classification	<b>MDD 93/42/EEC, Class IIa</b>		
Certificates	<b>CE,ISO13485, CFDA, FSC</b>		
Packaging	Single PE bag, 120pcs per middle bag, 20middle bags per carton		
Storage	Store in relative humidity less than 80%, no corrosive gas, cool, dry, well ventilated, clean environment.		

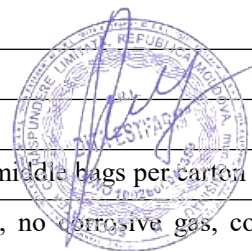



# Technical Data Sheet

Revision : 5

## Sterile Hypodermic Syringes for Single Use

Date : 2019-07-01

Page 1 of 2

### Operation Instruction

- 1.Open the individual packing and take out the syringe
- 2.Take off the needle cap, avoid touching with the needle tube
- 3.Extract the solution and exhaust the air
- 4.Push the plunger to the top of the barrel to finish fluid injection
- 5.Throw the used syringe in to to the medical waste box

### Cautions

- \*Check the batch number and validity before use.
- \*Tighten the needle before use.
- \*It is strictly prohibited to use if the single packing damaged or needle cap drop off.
- \*For single use only, use immediately after unsealed, destroy after use. It is strictly prohibited to reuse.

### Marks



### Packing

- \* Primary PE bag
- Sample PE bag design

