


Chapter 4 Preparation for use

4.1 Castors


1. The compact trolley has 4 braked castors. Ensure all brakes are applied prior to use and released before moving the compact trolley.

NOTE

Brakes should always be applied when the compact trolley is not being maneuvered.

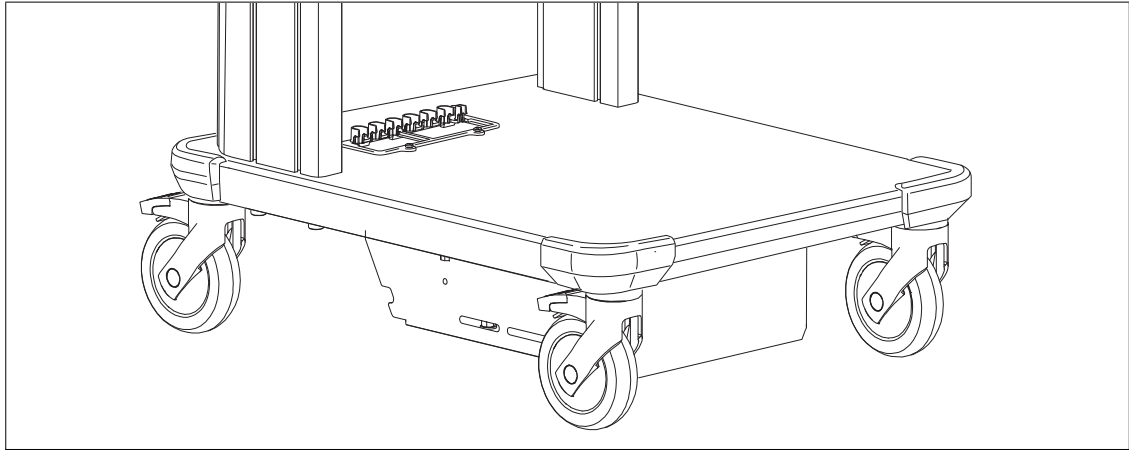**Figure 4.1**

2. To lock and unlock the castor brake, refer to Figure 4.2.

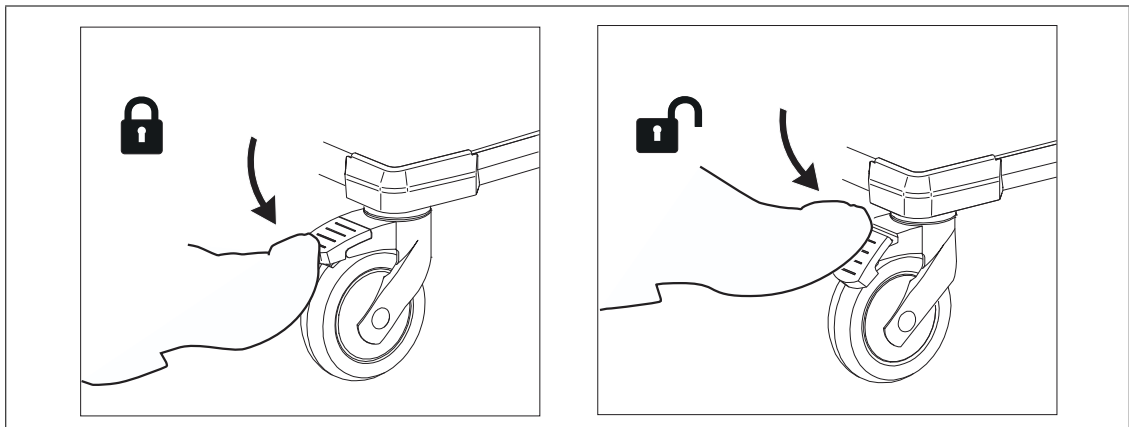**Figure 4.2**

4.2 Fixing positions of intermediate shelf

The compact trolley columns contain additional sets of fixing holes for the intermediate shelf. The shelf may be repositioned to suit the size of the equipment being mounted.

○ On delivery

The product is delivered with shelves fitted in the positions shown in Figure 4.3.

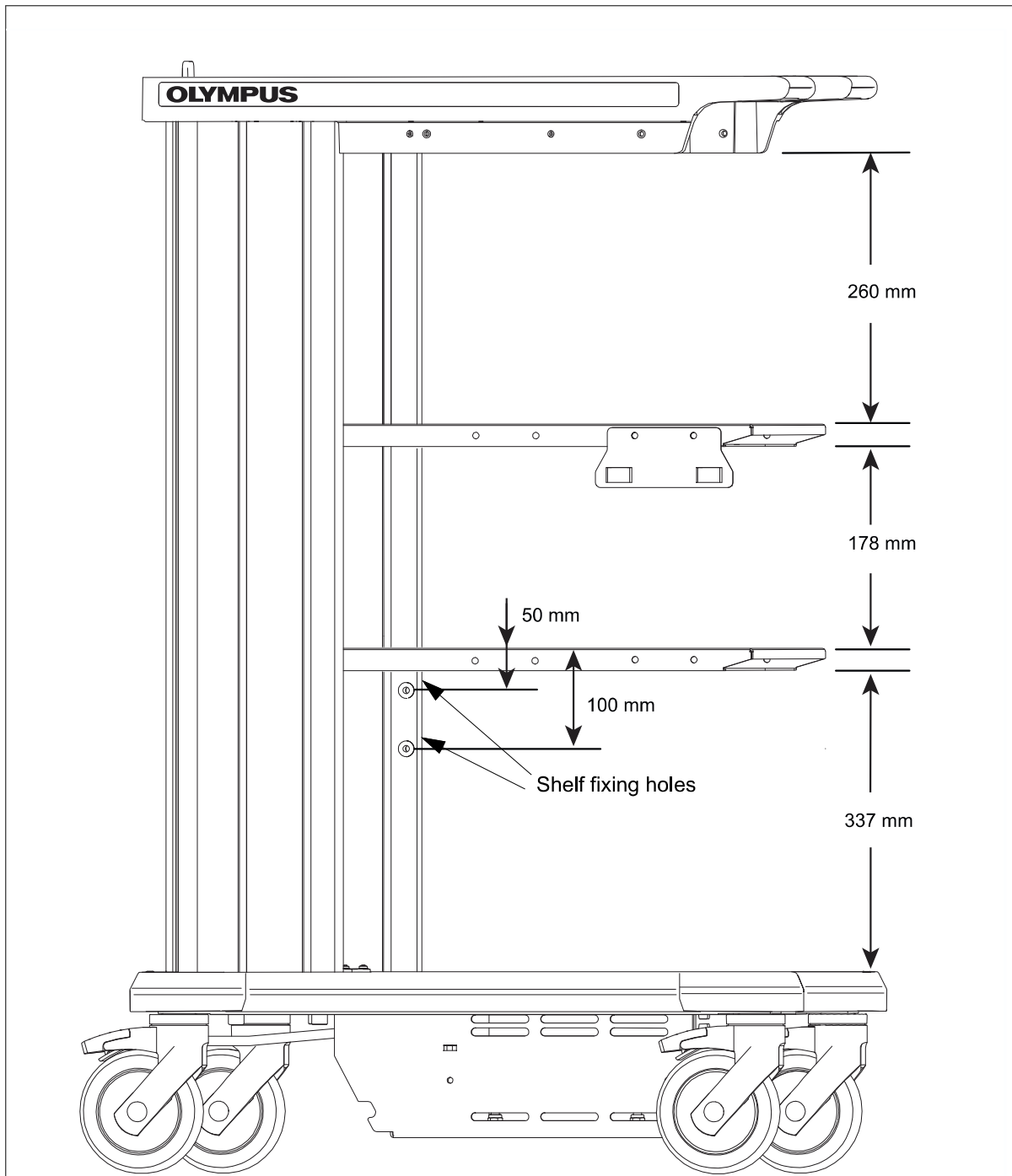


Figure 4.3

○ Adjusting the shelves

CAUTION

- Before commencing shelf adjustment, ensure equipment is disconnected and removed from the shelf.
- To avoid damaging the shelf or paintwork, it is recommended that two people adjust the shelf.

1. Position the compact trolley on a level surface.
2. Apply all brakes.
3. Remove any equipment that is on the shelf.
4. Referring to Figure 4.4, loosen the four M8 screws securing the shelf.
5. Take a firm hold of the shelf, then remove the four M8 screws.
6. Move the shelf to align with the required fixing holes and replace the four M8 screws, ensuring each screw is fully tightened.

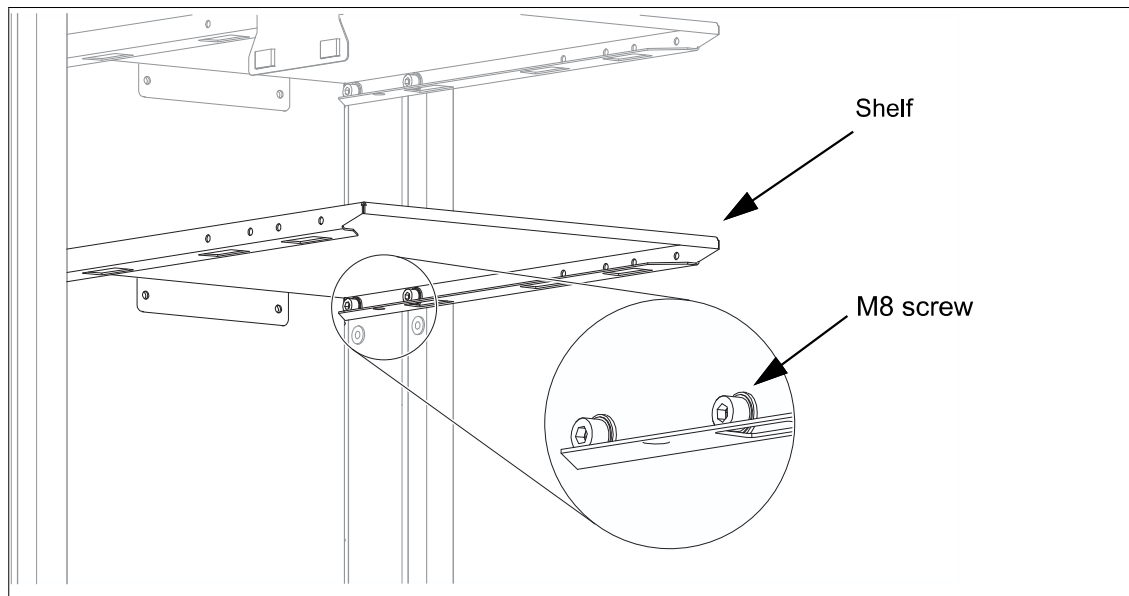


Figure 4.4

4.3 Assembling the scope pole

NOTE

The scope pole can be mounted on either the left hand side or right hand side of the compact trolley. Only one person is needed to install the scope pole.

1. Position the compact trolley on a level surface.
2. Apply all brakes.