

**SAM**  
MEDICAL

# **SAM**® Pelvic Sling II

**SLIDE.  
CLICK.  
SECURE.**

## **OPTIMAL FORCE STABILIZATION**

SAM® Pelvic Sling is the first and only force-controlled circumferential pelvic belt. Featuring an optimized one-piece design, it's scientifically proven to safely and effectively reduce and stabilize open-book pelvic ring fractures.



## **FEATURED TECHNOLOGY**

### **AUTO-STOP BUCKLE**

The patented SAM® AUTO-STOP buckle acts to avoid over- or under-tightening of SAM® Pelvic Sling. Once optimal compressive force is reached, two prongs activate, clicking to confirm correct application. The sling is secured in place with its hook and loop system strap.



## ENGINEERED TO PRESERVE LIFE

### 1 LOW-FRICTION POSTERIOR SLIDER

Easier transfers following severe trauma made possible by low-friction materials covering backside of the pelvic sling.

### 2 SAM® AUTO-STOP BUCKLE

Tactile and audible “click” provides clear feedback, confirming proper application and preventing over-tightening while maintaining optimal force.

### 3 PRECISION STRAP

Precisely-engineered holes lock with AUTO-STOP Buckle, allowing for quick application. Secure fastening with sturdy hook and loop strap.

### 4 TAPERED DESIGN


Narrow and tapered front facilitates urinary catheterization, interventional radiology, external fixation, and abdominal surgery without removal.

### 5 RADIOLOGENT MATERIAL

All materials are radiolucent except the buckle's two stainless steel springs, allowing X-rays and CT-scans without removal.

### 6 3 SIZES AVAILABLE

The Standard size fits 95% of the population. Complete the set with the Small and Large sizes to fit almost any body type.

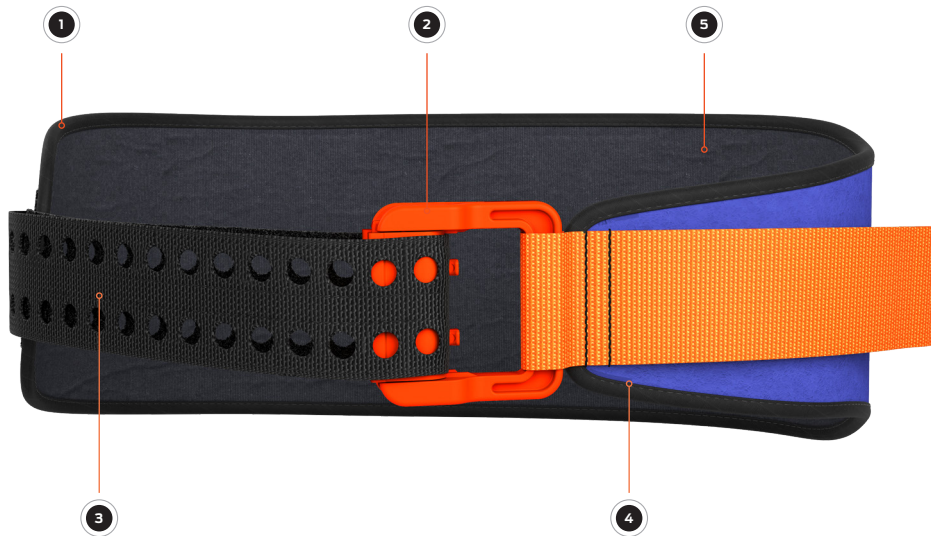

## MODELS



CIVILIAN  
ORANGE/BLUE



MILITARY  
OD GREEN /BLACK

## PRODUCT SPECIFICATIONS

### PACKAGED SIZE & WEIGHT\*

Height: 5.5 in (14 cm)  
Width: 7.5 in (19 cm)  
Depth: 3.75 in (9.5 cm)  
Weight: 8.6 - 9.2 oz (244 - 250g)\*\*

### OPERATING PRINCIPLE

Optimal force is reached when 150N of circumferential pressure is attained, which extends prongs from SAM® AUTO-STOP Buckle through the Precision Strap.

### MODELS AVAILABLE

Civilian - Small (SL556652-SM)  
Civilian - Standard (SL556652)  
Civilian - Large (SL556652-LG)  
Military - Standard (SL556652-OD)

### GLOBAL SALES

U.S. Toll Free: 800.818.4726  
U.S. Local: 503.639.5474  
U.S. Fax: 503.639.5425  
**sammedical.com**



Emergo Europe, Prinsessegracht 20,  
2514 AP, The Hague, The Netherlands



SAM Medical Products®  
27350 SW 95th Ave, Ste 3038  
Wilsonville, OR 97070 USA

\* Size and weight vary by configuration and manufacturing process.

\*\*Weight varies by size (Small, Standard, Large)

SAM® is a registered trademark in the United States and other countries.

©2017 SAM Medical Products. All rights reserved.



**European Declaration of Conformity to the Medical Device Directive, 93/42/EEC**



**Product Family Name:** SAM<sup>®</sup> Pelvic Sling II

**Catalog Numbers:**

SL556652-SM	SAM Pelvic Sling II, Civilian; Size: Small – Hip Circumference 27"-45" (69cm-114cm)
SL556652	SAM Pelvic Sling II, Civilian; Size: Standard – Hip Circumference 32"-50" (81cm-127cm)
SL556652-LG	SAM Pelvic Sling II, Civilian; Size: Large – Hip Circumference 36"-54" (91cm-137cm)
SL55652-MIL-OD	SAM Pelvic Sling II, Military – Olive Drab; Size: Standard – Hip Circumference 32"-50" (81cm-127cm)

**Description:** The SAM Pelvic Sling<sup>™</sup> is a non-invasive, circumferential pelvic belt intended for application by pre-hospital and medical professionals or trained personnel to individuals who have, or are suspected of having, a pelvic ring fracture where hemorrhage might occur.

**Classification Name:** Sling, Pelvic  
**GMDN Code:** 41155  
**Classification/Rule:** Class I by Rule 1, Annex IX  
**Conformity Assessment Route:** Annex VII  
**Notified Body:** Self-Certified

**Declaration:**

SAM Medical Products declares under its sole responsibility that the above products to which this declaration relates, and which bear the CE Marking, are in conformity with the applicable requirements of EC Directive 93/42/EEC of 14 June 1993 as transposed in the national laws of the Member States.

**Manufacturer:**

SAM Medical Products  
27350 SW 95<sup>th</sup> Ave. Suite 3038  
Wilsonville, OR 97070, U.S.A.  
TEL: 503-639-5474  
FAX: 503-639-5425

**EU Authorized Representative:**

Emergo Europe  
Prinsessegracht 20  
2514 AP, The Hague, The Netherlands  
TEL: +31 (0)70 345-8570  
FAX: +31 (0)70 346-7299

**Signature:**

  
Jeff Lipps, Director RA/QA

**Date:** 2017-12-08

