Surgical Drapes

STERILE EO

Microscope Drape

Intended Use

- Nowadays lots of surgery is made by surgical microscopes. During these procedures the risk of contamination from microscope to patient with open wound and to the media is high. Microscope cover is used for isolating nonsterile surgery microscopes which is taken into the sterile area during the surgical procedures and kept in. Microscope cover's centre is placed on the microscope's objective and its special ends are placed on the binocular. The cover is fixed to the microscope via stickers and belts on the cover.

Components of Product

Medbar Brand Microscope Drape consists of:

- Microscope cover film,
- A lens,
- A Lens center,
- 6 double-sided tape (small)
- A small blue arrow sticker,
- A small blue right hand sticker,
- A small blue left hand sticker,
- 3 tear here sticker,
- 3 green belt,
- A double-sided tape (larger).

In some models (204-01, 204-03, 204-05) there are

- Inner and Outer ring parts around the lens centre.
- Microscope cover (204-04) only consists of cover.

Product Features

- Specially folded for easy application.
- Super transparent lens system.
- Adhesive straps secure the drape safely in place.
- Accessories are easy to use when the drape is in place.
- Compatible with Carl Zeiss, Leica and Moller-Wedel operating microscope models.
- More than 50 different sizes available.
- Customized sizes are available upon request.

Quality

- CE marked according to 93/42/EEC Medical Device Directive and classified as class I sterile medical device.
- Manufactured under ISO 13485 Quality Management System standard.
- Notified Body: UDEM Ltd. Şti. (2292)
- EC Certification No: M.2016.106.7000
- GMDN Code: 12535

Sterilization

- Sterilized with Ethylene Oxide.
- Validated for ISO 11135 standard.

Biocompatibility

- Biologic evaluation of Microscope Drape is done according to ISO 10993-1.
- The products do not incorporate a substance of a human blood derivative, animal originated tissues, phthalates, medicinal product, radioactive materials and electromagnetic waves.
- Latex-free.

Shelf Life

- 5 years.

Manufacturing Methods

Main manufacturing processes for Microscope Drape are consists of

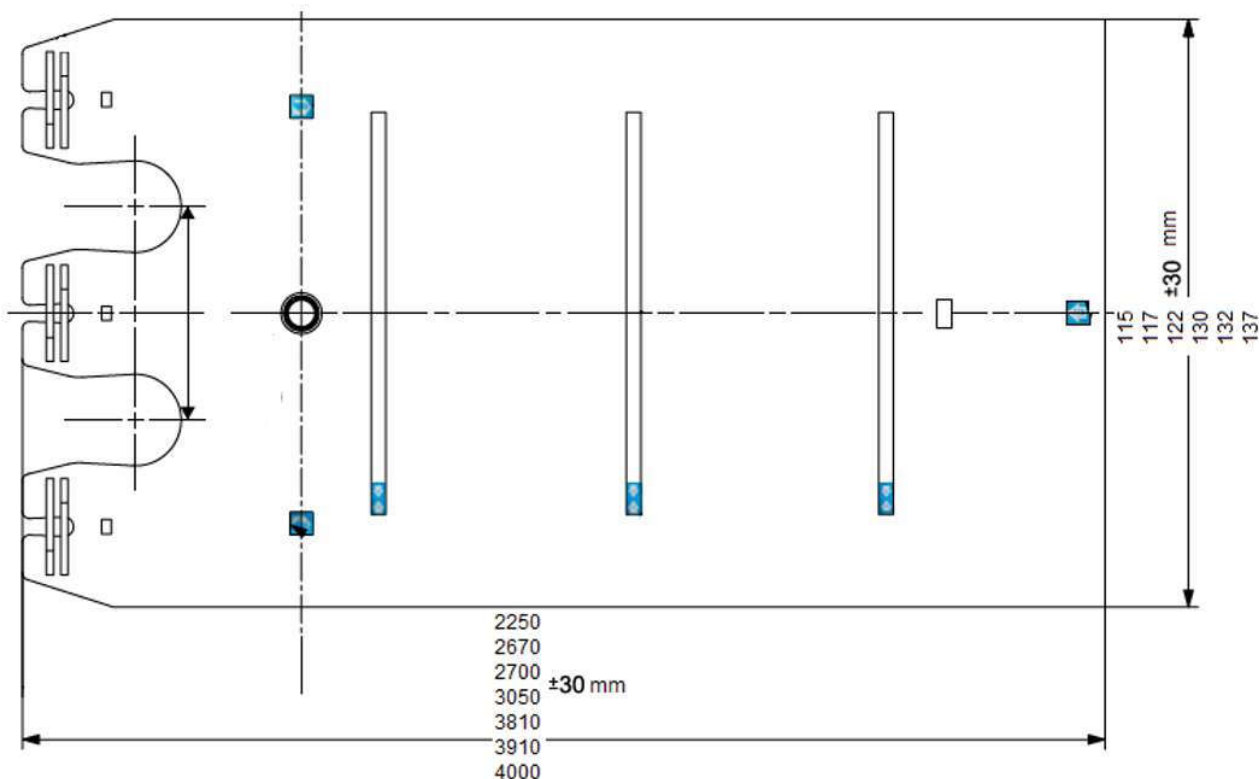
- Nylon production,
- Injection
- Assemble,
- Cutting-Sticking,
- Folding,
- Packaging,
- Parcelling and
- EO sterilization.

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