

SHORT INSTRUCTIONS


FOR THE
COMPLETE
HANDLING
INSTRUCTIONS

please refer to the
Instructions for Use
for each product on
www.gbo.com.

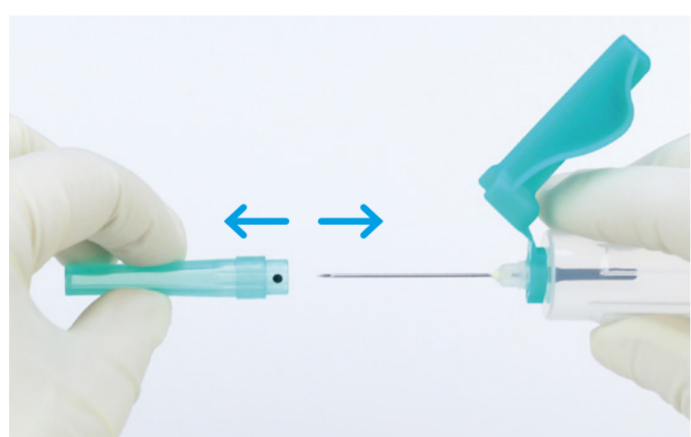
VACUETTE® QUICKSHIELD

Safety Tube Holder

The VACUETTE® QUICKSHIELD Safety Tube Holder, together with the VACUETTE® Blood Collection Tubes and needles, is suitable as a safe single-use system for everyday venous blood collection.

- 

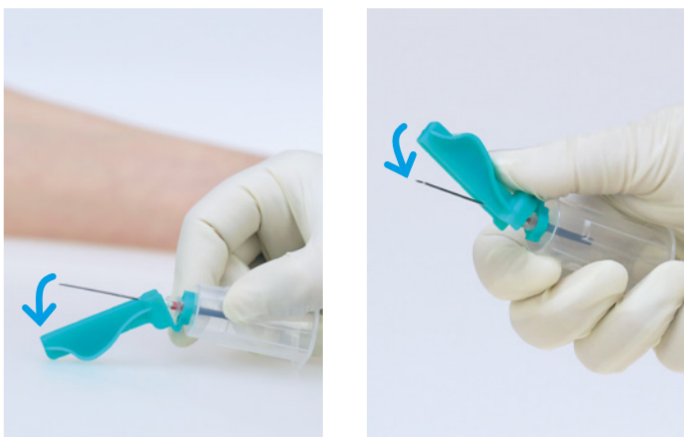
1
THREAD THE NEEDLE
- 

2
IF DESIRED, RE-POSITION
SAFETY SHIELD
- 

3
REMOVE PROTECTIVE
NEEDLE CAP
- 

4
VENIPUNCTURE
- 

5
INSERT THE TUBE
- 

6
REMOVE THE TUBE
- 

7
ACTIVATE THE SAFETY
MECHANISM
with the aid of solid surface or thumb
- 

8
// click
AUDIBLE CLICK INDICATES
ACTIVATION
- 

9
DISPOSAL

NOTE

Always follow general effective and safe working procedures and precautions during use and disposal of the products.

Legal Disclaimer: This product information is addressed exclusively to healthcare professionals. Devices of Greiner Bio-One are to be used by properly trained healthcare professionals only in accordance with the relevant Instructions for Use (IFU). For a listing of indications, contraindications, precautions and warnings, please refer to the Instructions for Use which accompanies each product or is available for download from our website at www.gbo.com (Download Center). For more information contact your local Greiner Bio-One sales representative or visit our website. All information is provided without guarantee despite careful processing. Any liability, warranty or guarantee of Greiner Bio-One GmbH is excluded. All rights, errors and changes are reserved. If not stated otherwise, Greiner Bio-One GmbH has all copyrights and/or other (user-)rights in this documents, in particular to signs such as the mentioned (word-picture-)brands and logos. Any use, duplication or any other use of the rights of Greiner Bio-One GmbH is expressly prohibited. Media owner: Greiner Bio-One GmbH

www.gbo.com

Greiner Bio-One GmbH / Bad Haller Straße 32 / 4550 Kremsmünster / Austria
PHONE +43 7583 6791-0 / FAX +43 7583 6318 / E-MAIL office@gbo.com

Poster Short Instructions Quickshield en [rev.01, 07.2022]


greiner
BIO-ONE