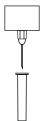
100
buc./pics.

STERILE EO CE 0344

32G 0,23mm
4mmAc stilou insulină
Insulin pen needle

iDiab

Ac stilou insulină
Insulin pen needle100
buc./pics.4mm
32G 0,23mm

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EN REMEMBER!

PEN NEEDLES SHOULD BE USED ONLY ONCE AND DISPOSED OF IN A SHARPS CONTAINER.

Weigao Medical International Co., Ltd.
No.1 Weigao Road High-Tech Industrial Development
Zone 264210 Weihai, Shandong Province
PEOPLE'S REPUBLIC OF CHINAMedNet EC-Rep GmbH
Borkstrasse 10, 48163 Muenster, Germany

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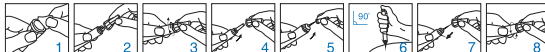
EN

How to use pen needles:

- patients should remember to rotate injections sites, according to applicable insulin delivery recommendations;
- pen needles are sterile and for single use only. Sterility is guaranteed if the needle seal is unbroken.

REMEMBER!

PEN NEEDLES SHOULD BE USED ONLY ONCE AND DISPOSED OF IN A
SHARPS CONTAINER.

Re-use can:

- cause lipohypertrophy;
- dull or bend up the tip, causing bleeding, bruising or scarring;
- increase the risk of needle breaking off in the skin;
- increase the risk of infection because the needle is no longer sterile.

iDiab

Ac stilou insulină
Insulin pen needle

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EN

Single use.
Use immediately after opening.
Sterile, non toxic, latex free.
DON'T REUSE.4mm
32G 0,23mm100
buc./pics.

STERILE EO CE 0344

Characteristics:

- needles for use with the pen for subcutaneous insulin injection;
- fine three-facet grinding and siliconizing of needle minimizes tissue trauma;
- extra thin wall allows for a larger inner diameter and an optimized insulin flow and reduce injection time;
- integrated plunger backstop;
- latex, PVC and DEHP free;
- EO sterilization.

Insulin needles are compatible with the following pen models
for insulin administration:Novo Nordisk: NovoPen® 3, NovoPen® 4, NovoPen® 5, NovoPen®
Junior, NovoRapid® FlexPen®;

Lilly: Humapen® luxura, Humapen® luxura Hd;

Owen Mumford: Autopen® 24, Autopen® classic 2;

Sanofi: Aventis Lantus, Optipen® Pro, TactiPen®;

BD: Puregon pen®

STERILE EO CE 0344

100
buc./pics.