## GARANZIA

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## WARRANTY

This shoulder crutch has **2-YEAR** warranty from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. **The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.** In the case you need to get your shoulder crutch repaired and you fulfil the conditions of warranty, send the it to the nearest retailer together with the purchase receipt.

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## SHOULDER CRUTCHES INSTRUCTION MANUAL

Moretti S.p.A.  
Via Bruxelles 3 - Melegnano  
52022 Cavriglia (Arezzo)  
Telefono +39 055 96 21 11

Fax +39 055 96 21 200  
[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)

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I Class Medical Device according  
to EEC directive 93/42

### Code RP710 - RP713

Thank you for purchasing shoulder crutch, **MOPEDIA** line by Moretti manufactured to support people with movement difficulty. It consists of lightweight anodised aluminium tubing, non-slip soft rubber tips, anatomic non-slip armrests and handgrips made of resistant material to give you comfort and security.  
It is height adjustable every 2,5 cm:

- **RP710** ..... from cm 112,5 a 140 h
- **RP713** ..... from cm 93 to 114 h

#### To adjust the height:

To adjust the height press the push button and slide the cane to the desired height. Be sure the push button is fully protruded through the desired hole when finished. To adjust the height of the handgrips, loosen the screw and draw it out. Insert the screw in the desired hole and tighten firmly the nut.  
The maximum endurable weight is 130kg.

#### ATTENTION: USE EXTRA CAUTION ON WET AND SLIPPERY SURFACES

Avoid to leave the shoulder crutch in contact with water for long periods and control its sturdiness periodically. Keep it away from children to avoid any possible risks. Replace rubber tips every time they are worn out.