Edta Tube
血常规管

Usage: applying to clinical haematology test, such as blood routine examination, all kinds of blood cells analysis (such as blood cell morphology examination, blood cell count, hematocrit, hemoglobin, sickling test, etc.)

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Heparin Tube
肝素管

Usage: applying to clinical biochemistry, blood specimen collection for emergency biochemical tests and anticoagulation(blood rheology measurement, biochemical test, genetic diagnosis, enzymology, enzyme immunoassay, etc.)

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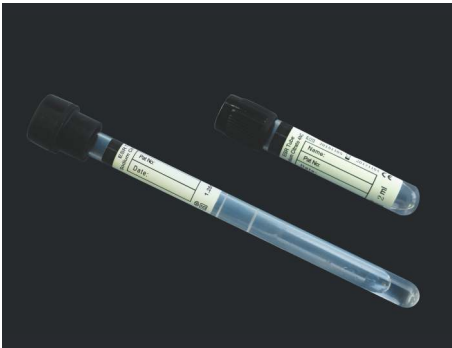
Glucose Tube
血糖管

Usage: Applying to clinical determination of blood glucose. The original characters of blood specimen can be maintained for a long time.

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ESR Vacuum Tube
血沉管

Usage: 1.28ml tube with a total capacity of 1.6ml, is equipped with 0.129mol/L sodium citrate 0.32ml. Collect blood 1.28ml to the central scale and after blood collection, we immediately invert the tube gently for 4-5 times so as to mix blood completely with anticoagulant, then put the tube into a D2010 automatic ESR analyzer and after 30 minutes read the data. This product can save time and effort and at the same time prevent cross infection.

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