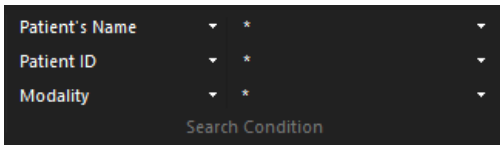


## 3.2 Search Condition

### 3.2.1 Search Conditions of Database

You can search the stored studies flexibly by combining the search conditions. The supported conditions are as follows.

Image	Search Conditions
	<ul style="list-style-type: none"> <li>• Study Status</li> <li>• Verification</li> <li>• Patient's Name, Patient ID</li> <li>• Patient Birth Date, Patient Sex, Patient Comments</li> <li>• Referring Physician</li> <li>• Study ID, Accession Number, Study Description</li> <li>• Modality, Body Parts, Operators</li> <li>• Institution, Source AE Title</li> <li>• Responsible Person (Demo &amp; Veterinary License Only)</li> </ul>

#### Study Status

The studies are classified into the following status.


Name	Status
Unread	When no report exists.
Preliminary	If an unauthorized report exists. (Any user can change the report).
Approved	If an approved report exists.
Approval Confirmed	If an approved report exists that has been verified.

#### Verification

This item indicates the verification status of each study. The status of study is changed when you click on the **Save** button to save the status of image after you diagnosed it.

Name	Status
Unverified	The status of image is not changed since it has been stored in the server.
Verified	The status of image is changed after checking it.

### 3.2.2 Search Conditions of DICOM Q/R

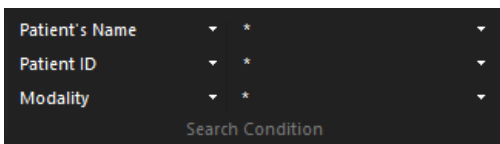Image	Search Condition
	<ul style="list-style-type: none"> <li>• Patient's Name, Patient ID</li> <li>• Patient Birth Date, Patient Sex, Patient Comments</li> <li>• Referring Physician</li> <li>• Study ID, Accession Number, Study Description</li> </ul>