


7.23.2 Description of APR Modification Window

7.23.2.1 APR Modification Window

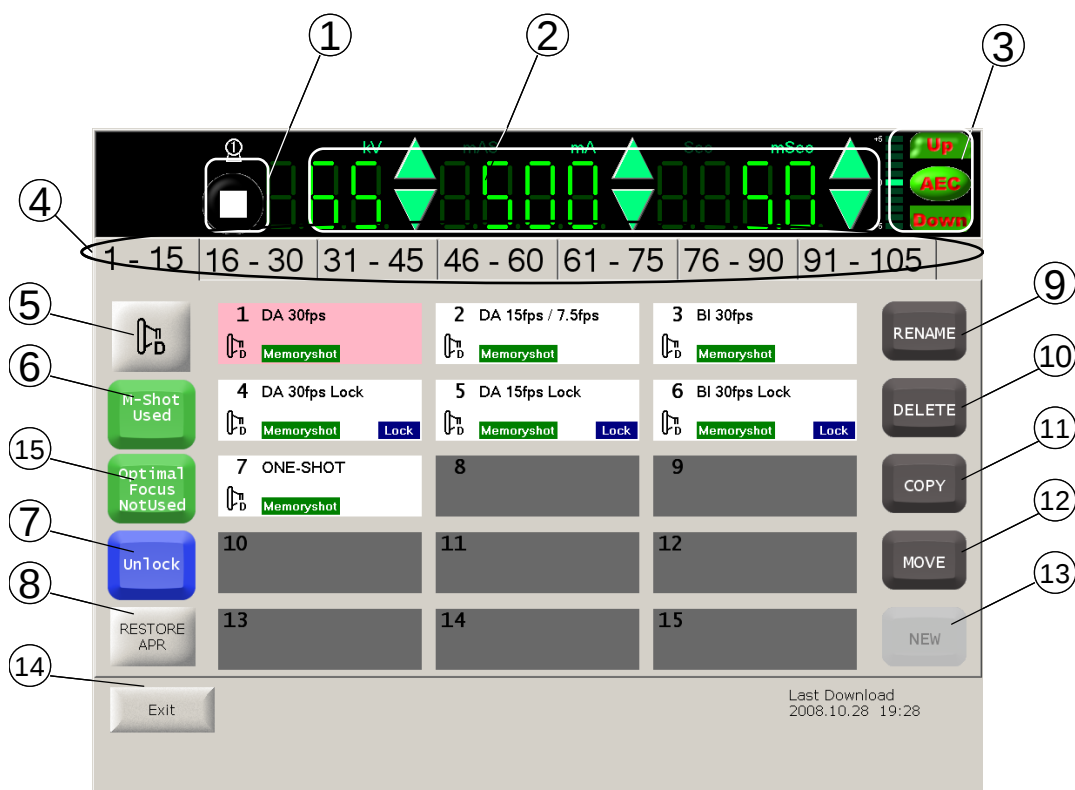

- | | |
|---------------------------|--------------------------|
| ① Focus Switch Button | ② X-ray Condition Button |
| ③ AEC Button | ④ APR Tab |
| ⑤ Radiography Mode Button | ⑥ Memoryshot Button |
| ⑦ Lock Button | ⑧ RESTORE Button |
| ⑨ RENAME Button | ⑩ DELETE Button |
| ⑪ COPY Button | ⑫ MOVE Button |
| ⑬ NEW Button | ⑭ Exit Button |
| ⑮ Optimal Focus Button | |

① Focus Switch Button

Switch the focus that is selected when changing APR.

② X-ray Condition Button

Change X-ray condition which is registered in APR. The value is not related if Memoryshot button is used in APR.

③ AEC Button

Configure AEC default mode when changing APR.