

## 9.2 Print Type

### 9.2.1 DICOM Print

| Button                                                                            | Name                       | Description                                                                                                                    |
|-----------------------------------------------------------------------------------|----------------------------|--------------------------------------------------------------------------------------------------------------------------------|
|  | <b>Print All Sheets</b>    | Transfers all images from the <b>Print</b> window to the DICOM printer. You can stop DICOM printing while printing out images. |
|  | <b>Print Current Sheet</b> | Transfers the image of the currently selected sheet to the DICOM printer.                                                      |
|  | <b>Print Current Page</b>  | Transfers the image of current page to the DICOM printer.                                                                      |



- Activated when the current DICOM printer is selected.
- When the images are transferred to the DICOM printer normally, the mark **Printed** will be displayed on the right top corner of the image.

### 9.2.2 Paper Print

| Button                                                                              | Name                       | Description                                                   |
|-------------------------------------------------------------------------------------|----------------------------|---------------------------------------------------------------|
|  | <b>Print All Sheets</b>    | Prints all images of all sheets in the print window to paper. |
|  | <b>Print Current Sheet</b> | Prints the image of the currently selected sheet on paper.    |
|  | <b>Print Current Page</b>  | Prints the image of the currently selected page on paper      |



- Activated when the current paper printer is selected.

### 9.2.3 Echo

| Button                                                                              | Name        | Description                                    |
|-------------------------------------------------------------------------------------|-------------|------------------------------------------------|
|  | <b>Echo</b> | Checks the activation status of DICOM Printer. |