

MEDIVATORS™ ISA™

Endoscope Reprocessor



SERVICE MANUAL

WASHER-DISINFECTOR

MEDIVATORS™

is a trademark of Medivators Inc.

INTERCEPT™

is a trademark of Medivators Inc.

RAPICIDE™

is a trademark of Medivators Inc.

ISA™

is a trademark of Cantel Medical (Italy) S.r.l.

ISASPOR™

is a trademark of Cantel Medical (Italy) S.r.l.

ISAZONE™

is a trademark of Cantel Medical (Italy) S.r.l.

ISACLEAN™

is a trademark of Cantel Medical (Italy) S.r.l.

All other company or product names referenced
are trademarks of their respective owners.

50098-1091-EN REV F

© 2018 All rights reserved. This publication is
protected by copyright laws.

The copying, distribution or use of this publication
without the express written permission of Cantel
Medical (Italy) are prohibited.

Cantel Medical (Italy) reserves the right to make
changes in specifications shown herein without notice
or obligation. For more information, contact your
Cantel representative.

CONTENTS

USE OF THE MANUAL

Important notes	04
-----------------------	----

CHAPTER 1

Advertisement for Safety	06
Intended Use for MEDIVATORS™ ISA™ Endoscope Reprocessor	11
Regulatory Compliance	12
Description of the Validated Chemical Solution.....	13
Handling and Storage of 70% Isopropyl Alcohol	15
MEDIVATORS ISA endoscope channel connections.....	16

CHAPTER 2

Dimensions MEDIVATORS ISA System	18
Technical Specification MEDIVATORS ISA System	20
External Connection Supplies	22
Hydraulic-Pneumatic Circuit MEDIVATORS ISA System	23
Layout MEDIVATORS ISA System	28
Hookup Connection Scheme.....	39

CHAPTER 3

Software Procedure - Technical / Administrator Function	41
--	----

CHAPTER 4

Accessories for Installation	88
Installation Procedure for MEDIVATORS ISA Endoscope Reprocessor ..	93
Setting for MEDIVATORS ISA Endoscope Reprocessor	96
TP2 Calibration: Leak Test Pressure Transducer	96
Leak Test Valve Check	99
Pressure Transducer Calibration: TP3, TP4, TP7	100
Pressure Transducer Calibration: TP5	102
R3 Calibration: Air Purge Pressure Reducer	104

TP1 Calibration : Inlet water Transducer	105
Setting of Inlet Water Flowmeters	107
Setting of Drain Flowmeter	110
Setting FLOW Reducer for Leak Tester R4....	111
R2 Calibration: Recirculation Channel Reducer	112
Setting of Detergent (Ft-13)	113
Setting of Alcohol (Ft-19)	114
Setting of HLD Solution A.....	115
Setting of HLD Solution B	116
Lid Calibration	117
Set Endoscopes.....	119
Performance Qualification.....	122

CHAPTER 5

Scheduled Maintenance	126
Water Filter Replacement.....	127
Air Filter Replacement.....	129
Replacement of Interlocked Endoscopic Channel Connection Gaskets	131

CHAPTER 6

Electrical Protections	134
Fuse Replacement.....	134

CHAPTER 7

Warranty.....	138
---------------	-----

CHAPTER 8

Troubleshooting	142
-----------------------	-----

Table 2. Input

CODE	DESCRIPTION	TYPE
SL11	Level Sensor Dosing Sol. A	Switch
SL12	Level Sensor Dosing Sol. B	Switch
FT-13	Detergent Flowmeter	Flow meter
FT-19	Alcohol Flowmeter	Flow meter
FT-1	Water load Flowmeter	Flow meter
FT-2	Water load Flowmeter	Flow meter
FT-14	Drain Flowmeter	Flow meter
FT-5	Channel 1 Flowmeter	Flow meter
FT-6	Channel 2 Flowmeter	Flow meter
FT-7	Channel 3 Flowmeter	Flow meter
FT-8	Channel 4 Flowmeter	Flow meter
FT-9	Channel 5 Flowmeter	Flow meter
FT-10	Channel 6 Flowmeter	Flow meter
TP4	Air Purge Pressure Transducer	Pressure transducer4...20mA
T1	Temperature Sensor	PT-1000
T2	Temperature Sensor	PT-1000
TP2	Leak Test Pressure Transducer	Pressure transducer4...20mA
SL1	Basin Level Sensor	Switch
SL2	Basin Level Sensor	Switch
SL3	Leakage Sensor	Pressure transducer4...20mA
TP7	Channel 7 Pressure Transducer	Reed Sensor
FC	Switch Sensor Closed Lid	Reed Sensor
FO	Switch Sensor Open Lid	Pressure transducer4...20mA
TP1	Pressure Switch Water	Switch
PED	Footswitch	Switch
TP5	Pressure Transducer Compressed Air Lid	Pressure transducer4...20mA
SGR	Spray Arm Sensor	Magnetic Sensor