
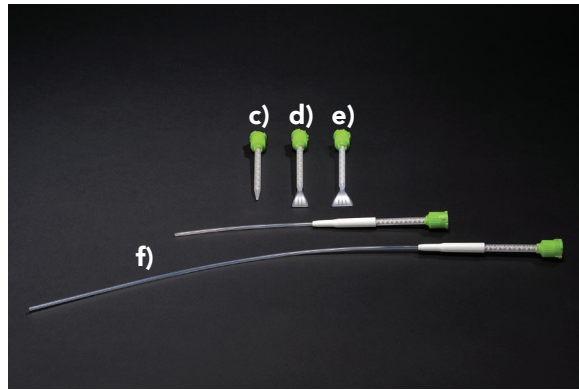


Syringe Assembly Procedure

Safe | Simple | Effective

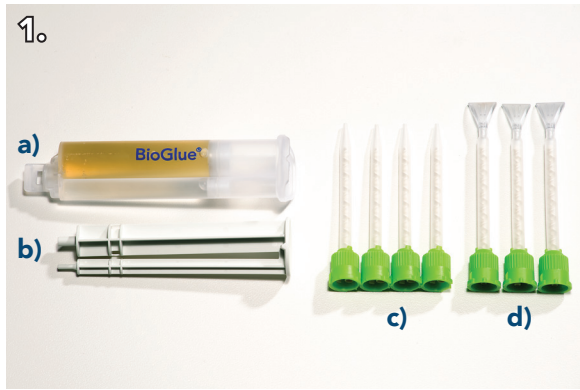



1. System Components:

- a) Pre-filled BioGlue Syringe
- b) Syringe Plunger
- c) Syringe Applicator Tips
- d) 12mm Spreader Tips
- e) 16mm Spreader Tips
- f) Applicator Tips with Flexible Extensions (10cm and 27cm)


NOTE: Hold syringe upright during assembly.

- 2. Hold syringe upright and tap until air bubbles rise to the top.
- 3. Remove cap by twisting 90° counter-clockwise and gently lift it from syringe.

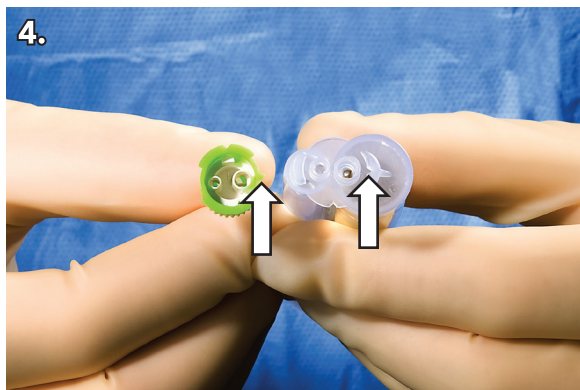

- 4. Align the triangular notch on the collar of the applicator tip with the corresponding notch on the syringe nose.
- 5. Seat the tip firmly on syringe and rotate collar 90° clockwise to lock in place.



- 6. Align the barrels of the syringe with the plunger heads and slide the plunger into the syringe until resistance is felt.



- 7. Remove the residual air space using one of two methods:
 - a) Compress the plunger only until the solutions are even with top of the syringe body. The syringe is now ready for priming and immediate use.
 - b) Compress the plunger until both solutions can be visibly seen in the base of the tip. Change the tip prior to priming and use.

Keys To Success

- 1. **De-Air** remove all air from the syringe.
- 2. **Dry** the target site (by suction and/or gauze).
- 3. **Prime** the syringe.
- 4. **Apply** a thin layer in a slow and steady manner.
- 5. **Wait** 2 minutes for full strength.

To Order, Contact
ARTIVION
artivion.com/contact

BioGlue Syringe Delivery System

Catalogue #	Product	Contains
BG3510-5-US	Syringe 10mL 5-Pack Kit	Five single packs – contains one 10mL syringe and syringe plunger, four standard syringe tips
BG3515-5-US	Syringe 5mL 5-Pack Kit	Five single packs – Each contains one 5mL syringe and syringe plunger, and four standard syringe tips
BG3502-5-US	Syringe 2mL 5-Pack Kit	Five single packs – Each contains one 2mL syringe and syringe plunger, and four standard syringe tips
BGAT-SY	Syringe Applicator Tip	Ten single packs – Each contains four standard syringe tips
BGAT-10-SY	Syringe 10cm Applicator Tip	Ten single packs – Each contains four 10cm syringe tips
BGAT-27-SY	Syringe 27cm Applicator Tip	Ten single packs – Each contains four 27cm syringe tips
BGAT-10-SY	Syringe 10cm Applicator Tip	Ten single packs – Each contains four 10cm syringe tips
BGST-12	Syringe 12mm Spreader Tip	Ten single packs – Each contains three 12mm spreader tips
BGST-16	Syringe 16mm Spreader Tip	Ten single packs – Each contains three 16mm spreader tips

ARTIVION™

Learn more at [artivion.com](https://www.artivion.com)

All products and indications are not available/approved in all markets. Artivion and BioGlue are registered trademarks owned by Artivion, Inc. or its subsidiaries. On-X Life Technologies, Inc. Jotec GmbH and Ascyrus Medical GmbH are wholly owned subsidiaries of Artivion, Inc. All other trademarks are owned by their respective owners and their use herein does not reflect affiliation with or endorsement by the respective owners. © 2022 Artivion, Inc. All rights reserved.

Artivion, Inc.
1655 Roberts Blvd., NW, Kennesaw, GA 30144 USA
Phone: 888-427-9654 | Fax: 770-590-3573 | E-mail: inquiries@artivion.com
For contact information by region, please visit www.artivion.com/contact

 CryoLife, Inc., 1655 Roberts Blvd., NW, Kennesaw, GA 30144 USA

MLENG0161.004 (2022-07)