



STERILE CATHETER GEL

At Turkuaz Healthcare, we continue unabated product and technology development speed in our R&D Center and that has led us to developing our range of catheterisation gels.

Our new formulations are designed to offer clinicians a choice while providing solutions to different needs of patients...

LIDO C Sterile Catheter Gel

Lidocaine 2% & Chlorhexidine 0,05%

NEW LIDO Sterile Catheter Gel

*Lidocaine 2%
(Without CHG)*

NEW C Sterile Catheter Gel

*Chlorhexidine 0,05%
(Without Lidocaine)*



CE
2292
Clas III
Certified



EUROPE'S LEADING MEDICAL GEL MANUFACTURER, THE CHOICE OF MEDICAL PROFESSIONALS!



Lido C

Sterile Catheter Gel

ANAESTHETIC & ANTISEPTIC



KONIX Lido C Catheter Gel is a sterile, transparent, water soluble lubricant gel used during insertion of urethral catheter, cystoscopy and other medical devices.



PROPERTIES

- Effective and long-lasting lubrication
- Anaesthetizing & antiseptic effects
- Easy to apply
- Ideal viscosity



INGREDIENTS

- Deionized water
- Lidocaine (Local anaesthetic)
- Chlorhexidine Gluconate (Antiseptic)
- Hydroxyethyl Cellulose (Lubricative)
- Mono Propylene Glycol (Moisturizer)
- Methyl Hydroxybenzoate (Preservative)
- Propyl Hydroxybenzoate (Preservative)



KONIX Lido C Sterile Catheter Gel has local anaesthetic effect with Lidocaine and antiseptic effect with Chlorhexidine Gluconate.

Lubricating the catheter is the easier and painless way of inserting the catheter into a body orifice.

KONIX Lido C Sterile Catheter Gel, because it...

- is a lubricant for catheters, endoscopes, probes, tubes or other medical instruments for insertion in the urethra
- is applied when no anaesthesia is needed
- reduces the risk of urethral trauma on catheter insertion
- reduces the possibility of infection by reducing friction and discomfort
- reliefs pain through anaesthetic effect of Lidocaine
- has antiseptic effect due to Chlorhexidine Gluconate
- is latex free and hypoallergenic
- helps dilation of urethra or target body orifice

KONIX Lido C Sterile Catheter Gel has been CE marked as a Class III Medical Device under the Medical Device Directive 93/42 EEC

STERILE R

Sterilized with Gamma

NEW



Lido

Sterile Catheter Gel



ANAESTHETIC

KONIX Lido Sterile Catheter Gel has been CE marked as a Class III Medical Device under the Medical Device Directive 93/42 EEC

STERILE

Sterilized with Gamma

KONIX Lido Sterile Catheter Gel is a sterile, water-soluble, non-irritating lubricant gel with supportive anesthetic properties used to facilitate insertion of catheters and other medical instruments (catheterization, cystoscopy) into the urethra and bladder.



PROPERTIES

- Effective and long-lasting lubrication
- Anaesthetizing effect
- Easy to apply
- Ideal viscosity



INGREDIENTS

- Deionized water
- Lidocaine (Local anaesthetic)
- Mono Propylene Glycol (Moisturizer)
- Hydroxyethyl Cellulose (Lubricative)
- Methyl Hydroxybenzoate (Preservative)
- Propyl Hydroxybenzoate (Preservative)



KONIX Lido Sterile Catheter Gel has local anaesthetic effect with Lidocaine.

Lubricating the catheter is the easier and painless way of inserting the catheter into a body orifice. KONIX Lido Sterile Catheter Gel, because it...

- is a lubricant for catheters, endoscopes, probes, tubes or other medical instruments for application in the urethra
- is applied when no anaesthesia is needed
- reduces the risk of urethral trauma on catheter insertion
- reduces the possibility of infection by reducing friction and discomfort
- relieves pain through anaesthetic effect of Lidocaine
- is latex free and hypoallergenic
- helps dilation of urethra or target body orifice

NEW

KONIX®

C

Sterile Catheter Gel



ANTISEPTIC

KONIX C Sterile Catheter Gel has been CE marked as a Class III Medical Device under the Medical Device Directive 93/42 EEC

STERILE !!

Sterilized with Steam

KONIX C Sterile Catheter Gel is a sterile, transparent, water-soluble, non-irritating lubricant gel with physiological conductivity and supportive antiseptic properties to facilitate the insertion of catheters and other medical instruments (catheterization, cystoscopy) into the urethra and bladder, especially for endourological procedures in electrocoagulation and resection.



PROPERTIES

- Effective and long-lasting lubrication
- Antiseptic effect
- Easy to apply
- Ideal viscosity



INGREDIENTS

- Deionized water
- Chlorhexidine Gluconate (Antiseptic)
- Mono Propylene Glycol (Moisturizer)
- Hydroxyethyl Cellulose (Lubricative)
- Methyl Hydroxybenzoate (Preservative)
- Propyl Hydroxybenzoate (Preservative)



KONIX C Sterile Catheter Gel has antiseptic effect with Chlorhexidine Gluconate.


Lubricating the catheter is the easier and painless way of inserting the catheter into a body orifice KONIX C Sterile Catheter Gel, because it...

- is a lubricant for catheters, endoscopes, probes, tubes or other medical instruments for application in the urethra
- is applied when no anaesthesia is needed
- reduces the risk of urethral trauma on catheter insertion
- reduces the possibility of infection by reducing friction and discomfort
- has antiseptic effect due to Chlorhexidine Gluconate
- is latex free and hypoallergenic
- helps dilation of urethra or target body orifice



Sterile Catheter Gel

APPLICATION

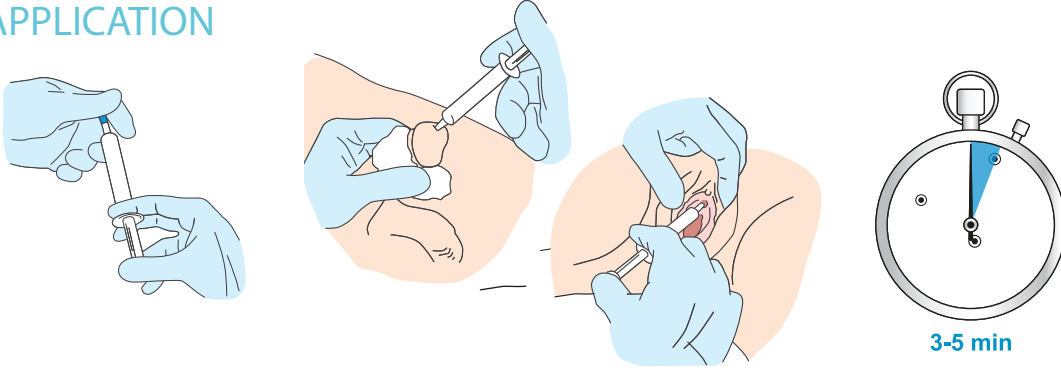

1. Wash your hands and prepare an area that is aseptic as possible to perform the operation. Prepare a clean serving tray and place required materials.
2. Remove the syringe (6ml, 12ml and 12.5g) from its sterile package by tearing off. 6ml syringe is suitable for use in women, while the 12ml syringe and 12.5g accordion syringe are suitable for use in men.
3. Lightly press on the plunger before removing the blue cap to provide more even and gentle application. For the accordion syringe, break off the applicator.
4. Carefully insert the cone of the syringe into the urethral opening and slowly inject the gel by applying gentle and even pressure.
5. Apply the remaining gel to the surface of the catheter or to the surface of another medical device.
6. KONIX Sterile Catheter Gels shows lubricative effect as soon as the application starts. KONIX Lido C & KONIX Lido's anesthetic effect starts 3-5 minutes later.
7. If more gel has to be used, condition of applied tissue, age of the patient and any adverse condition of the patient which may be related to sensitivity of Lidocaine and chlorhexidine must be taken into care by the physician.
8. When the whole product is not used, the syringe and the remaining gel should be evaluated and discarded like other medical wastes.
9. KONIX Sterile Catheter Gels should only be used under the supervision of specialist health personnel.

PRODUCT LINE

KONIX Sterile Catheter Gel is available in sterile, single-use syringes of 6 ml, 12 ml and 12.5g accordion syringe. Healthcare professional should take care of instructions of use, as well as individual patients' medical history and experiences, before choosing catheter gel to use.

Product Name	Model	Packaging	Product Code
Konix Lido C Sterile Catheter Gel	6 ml	6ml Syringe	Y0000.107.0005
Konix Lido C Sterile Catheter Gel	12 ml	12ml Syringe	Y0000.107.0007
Konix Lido C Accordion Syringe	12.5g	Accordion Syringe	Y0000.107.0015
Konix Lido Sterile Catheter Gel	6 ml	6ml Syringe	Y0000.107.0020
Konix Lido Sterile Catheter Gel	12 ml	12ml Syringe	Y0000.107.0021
Konix C Sterile Catheter Gel	6 ml	6ml Syringe	Y0000.107.0016
Konix C Sterile Catheter Gel	12 ml	12ml Syringe	Y0000.107.0017



Konix Lido C Sterile Catheter Gel Certificate



Konix Lido Sterile Catheter Gel Certificate



Konix C Sterile Catheter Gel Certificate

