

## DATA & ESSENTIALS SHEET

### VACCUM BLOOD COLLECTION TUBE HOLDERS



1. General Information		
1.1.	Description:	Holders for blood collection from vacuum tubes.
1.2.	Specifications:	Universal design. Perfectly suitable for use with TTT WORLD vacuum blood collection tubes and needles.
1.3.	Advantages for user:	Ergonomic design. Optimal size for transportation. Enables multiple blood collection.
1.4.	How to Use:	Attach the female luer tip of the holder to the vacuum blood collection needle. Push the blood collection tube through the tip of the needle inside of the holder. After the application, click the safety cap if provided. The safety and luer adapter versions are for single use only. Dispose according to the medical waste rules. The application shall be made by authorized health care practitioners only.

<b>2</b>	<b>Design</b>	
2.1.	Variations:	Can be supplied with safety lock, luer tip or as standard form.
2.2.	Luer Tip:	Complies with ISO 80369 Luer Connectors standard.
2.3.	Package Details:	Can be supplied as individually packed or bulk according to the demand.
2.6.	Shelf Life:	5 years
<b>3</b>	<b>Complied Standards and Certificates</b>	
3.1.	CE Marking	Yes
3.2.	Directive	98/79/EC Directive on In Vitro Diagnostic Medical Devices
3.3.	Class	IVD, others
3.4.	Other Certificates	ISO 13485
3.5.	GMDN	37566 – Adaptor, blood collection tube, single use

