

RELIABLE HEARING IN ANY SITUATION

PRODUCT CATALOG



REXTON


OUR LATEST INNOVATIONS FOR RELIABLE HEARING



MotionCore technology



M-Core hearing aids

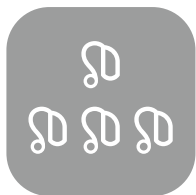

Hearing aid model overview



M-CORE B

Rechargeable BTE power, now with MotionCore

8



M-CORE R

Updated design, with Rechargeable, CROS, and Telecoil options

12



M-CORE SR

Award-winning design – now with exchangeable receivers

18



M-CORE iX

Our smallest instant-fit CIC, now with MotionCore

19

Model overview

Proven technology

6

MOTIONCORE TECHNOLOGY



Today's hearing aids analyze hearing situations solely on acoustic criteria, then classify them into a small range of predefined "acoustic environments". But when people rely on their senses to keep them safe and sound, hearing clearly in every situation is a must – especially when they're on the job and moving through the day, where environments can change quickly.

Our latest range of hearing aids use proven MotionCore technology to automatically recognize and adapt to more hearing situations than ever. This ensures users automatically hear speech clearer and get the most reliable impression of their environment. M-Core hearing aids are designed to be durable, practical and easy-to-use, so your customers can rely on Rexton in any situation.

UP TO 20 TIMES MORE HEARING SITUATIONS THAN BEFORE, AUTOMATICALLY



High Resolution Soundpro

Using a combination of inputs including real-time motion data, signal to noise ratio, and sound directionality, High Resolution Soundpro can continuously adjust the sound settings to provide the most reliable impression of the user's environment, automatically.



Motion Sensor

MotionCore technology features an integrated real-time motion sensor that detects a user's movement, and uses the data to even more accurately determine their sound situation. With this, M-Core hearing aids can automatically detect up to 127 different sound situations.



My Voice

The hearing aid recognizes the user's specific voice pattern, and integrates it naturally into what they hear. In turn, this provides the user with a natural-sounding voice that leads to high acceptance rates. And for that the user will communicate more and be more socially active, greatly improving their quality of life.



Streaming

Audio signals from smartphones, TVs or devices can be streamed wirelessly to the hearing aids; directly or via transmitter. Direct Audio Streaming is available from iOS smartphones, and Android smartphones can stream with the addition of the Smart Mic accessory.











REXTON APP: A HEARING AID WEARER'S ALL-IN-ONE SMARTPHONE COMPANION

The Rexton App allows wearers to control their hearing aids conveniently and discreetly, e.g. change programs, volume, tone.

Download the Rexton App now!



A FULL RANGE OF RELIABLE HEARING OPTIONS FOR YOUR CUSTOMERS

	Custom				Instant Fit	Slim-RIC	RIC	
								
	Sterling IIC	Sterling CIC	Sterling ITC	Sterling ITE	M-CORE iX-CIC (inoX)	M-CORE SR	Emerald XS	M-CORE R312 (Emerald S)
Battery size	10	10	312	312	10/312	Li-Ion	10	312
PL80								
PL60								
PL40								
PL30	New	New	New	New	New	New		New
PL20	New	New	New	New	New	New		New
PL10								

BTE



**M-CORE
R-Li**
(Stellar RIC)

**Emerald
M**



**Mosaic
S**



**Mosaic
M**



**M-CORE
B-Li M**
(Stellar M)



**Mosaic
P**



**M-CORE
B-Li P**



**Mosaic
HP**



**M-CORE
B-Li HP**

Li-Ion	13	312	13	Li-Ion	13	Li-Ion	675	Li-Ion
● ●	● ●		● ●	● New	● ●	New	●	New
● ●	● ●		● ●	● New	● ●	New	●	New
● ●	● ●	●	● ●	● New	● ●	New	●	New
● New	●	●	● ●	● New	● ●	New	●	New
● New	●	●	● ●	● New	● ●	New	●	New
		●	●		●		●	



RECHARGEABLE BTE POWER

NOW WITH MOTIONCORE

Our latest generation of BTE hearing aids comes completely rechargeable, even up to our High Power models with an unbelievable runtime of up to 61 hours on a single charge. They also feature MotionCore technology, which automatically recognizes and adapts to more hearing situations than ever. With new performance levels and our first rechargeable Superpower BTE, more users can automatically get the most reliable impression of their environment than ever. M-Core B-Li hearing aids are designed to be durable, practical and easy-to-use, so your customers can rely on Rexton in any situation.

Lithium-ion power for every need

- Up to 61 hours wear time on a single full charge
- 30-minute fast charge delivers up to six hours of wear time
- Automatic drying function integrated in the optional Dry&Clean charging station



Available colors



Silver



Pearl White



Beige



Rose Gold



Gray



Dark Champagne



Black



Deep Brown



Sandy Brown



Granite



With a compact size that is comparable to a 312 battery device, the new M-CORE B-Li M combines a new, modern design with rechargeability and Bluetooth connectivity.

60/133 dB FOG/OSPL (Earhook)

60/125 dB FOG/OSPL (Life Tube)



M-CORE B-Li M

Performance Level	80, 60, 40, 30, 20
Battery	Lithium-ion battery
FOG in dB	60
Accessories	Connex Basic Charging Station Connexx Charging Station
	Smart Mic
	Smart Transmitter 2,4
	New Rexton App
	New Life Tube and tips
	Color Conversion Kits
Features	Integrated motion sensor
	Hi-Res Soundpro
	Up to 120 hearing situations
	Direct Audio Streaming
	Programming via: NoahLink Wireless

Chargeable via:



Connex Basic Charging Station



Connex Charging Station

RECHARGEABLE BTES

WITH THE POWER OF MOTIONCORE

With a new smaller size, and rechargeable battery, the new M-CORE B-Li P remains a powerful device that now also offers the latest MotionCore technology.

77/135 dB FOG/OSPL (Earhook)

66/130 dBFOG/OSPL (Life Tube)

M-CORE B-Li P



Performance Level	80, 60, 40, 30, 20
Battery	Lithium-ion battery
FOG in dB	77
Accessories	Standard or Dry&Clean Charger
	Smart Mic
	Smart Transmitter 2,4
	New Rexton App
	New Life Tube and tips
	Color Conversion Kits
Features	Integrated motion sensor
	HiRes SoundPro
	Up to 120 hearing situations
	Direct Audio Streaming
	Programming via: NoahLink Wireless

Chargeable via:



B-Li P Dry&Clean Charger



Standard P Charger



Our most powerful device, the M-CORE B-Li HP is the smart choice for those with severe to profound hearing loss, combining the latest MotionCore technology with rechargeability.

82/140 dB FOG/OSPL (Earhook)



M-CORE B-Li HP

Performance Level	80, 60, 40, 30, 20
Battery	Lithium-ion battery
FOG in dB	82
Accessories	Standard or Dry&Clean Charger
	Smart Mic
	Smart Transmitter 2,4
	New Rexton App
	New Life Tube and tips
	Color Conversion Kits
Features	Integrated motion sensor (TL 8 - 16)
	HiRes SoundPro
	Up to 120 hearing situations
	Direct Audio Streaming
	Programming via: NoahLink Wireless

Chargeable via:



B-Li HP Dry&Clean Charger



Standard HP Charger

M-CORE B-Li M



Performance Level 80 | 60 | 40 | 30 | 20



VERSATILE AND RECHARGEABLE

Our most popular BTE device now features MotionCore technology and a rechargeable lithium-ion battery, for an accurate impression of your environment in every situation. Made for a wide range of hearing losses, M-Core B-Li M is newly designed and is available in Performance Levels 20-80. Your customers can choose from 10 colors, and have their device fitted open or closed depending on their needs.

Highlights

- › Integrated motion sensor
- › Lithium-ion battery
- › Programmable rocker switch
- › ThinTube 3.0
- › Color Conversion Kits
- › Available as tune T2.0 device
- › Programming via:
 - › NoahLink Wireless

Receiver	Ear hook	ThinTube 3.0
FOG/OSPL in dB	60/133	60/125
Battery current drain	up to 21 h	up to 21 h

Colors



Features / Performance Level	80	60	40	30	20
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4	16/8/4
Soundpro	HighRes	HighRes	HighRes	-	-
My Voice	•	•	•	-	-
Direct Streaming/ Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume
Wireless sync	•	•	•	•	•
Directionality	Automatic/Adaptive iOmni Front/Back Left/Right Narrow	Automatic/Adaptive iOmni Front/Back	Automatic/Adaptive iOmni	Automatic/Adaptive	Automatic/Adaptive
Noise Reduction	Noise Management Sound Smoothing Directional	Noise Management Sound Smoothing Directional	Noise Management Sound Smoothing Directional	Noise Management Sound Smoothing	Noise Management
Wind Noise Reduction	Binaural	Binaural	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	•	•	-	-	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	-
Xphone	•	•	•	-	-
Acclimatization/Data Logging	• / •	• / •	• / •	• / •	• / •
Small ear hook (optional)	•	•	•	•	•
Accessories & Apps					
Connex Basic Charging Station	•	•	•	•	•
Connexx Charging Station	•	•	•	•	•
Smart Mic	•	•	•	•	•
Smart Transmitter 2.4	•	•	•	•	•
Smart Key	•	•	•	•	•
Rexton App	•	•	•	•	•

M-CORE B-Li P



Performance Level 80 | 60 | 40 | 30 | 20



POWERFUL AND RECHARGEABLE

With a new design and rechargeable battery, the new M-Core B-Li P remains a powerful device that now also offers the latest motion technology. Available in Performance Levels 20-80 for a wide range of applications.

Highlights

- › Integrated motion sensor
- › Lithium-ion battery
- › Programmable rocker switch
- › ThinTube 3.0
- › Color Conversion Kits
- › Available as tune T2.0 device
- › Programming via:
 - › NoahLink Wireless

Receiver	Ear hook	ThinTube 3.0
FOG/OSPL in dB	77/135	66/130
Battery current drain	up to 27 h	up to 27 h

Colors



Features / Performance Level	80	60	40	30	20
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4	16/8/4
Soundpro	HighRes	HighRes	HighRes	-	-
My Voice	•	•	•	-	-
Direct Streaming/ Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume
Wireless sync	•	•	•	•	•
Directionality	Automatic / Adaptive iOmni Front & Back Left & Right Narrow	Automatic / Adaptive iOmni Front & Back	Automatic / Adaptive iOmni	Automatic / Adaptive	Automatic / Adaptive
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management
Wind Noise Reduction	Standard Binaural	Standard Binaural	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	•	•	-	-	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	-
Xphone	•	•	•	-	-
Acclimatization/Data Logging	• / •	• / •	• / •	• / •	• / •
Small ear hook (optional)	•	•	•	•	•
Accessories & Apps					
Standard Charger	•	•	•	•	•
Dry&Clean Charger	•	•	•	•	•
Smart Mic	•	•	•	•	•
Smart Transmitter 2,4	•	•	•	•	•
Smart Key	•	•	•	•	•
Rexton App	•	•	•	•	•

M-CORE B-LI HP



Performance Level 80 | 60 | 40 | 30 | 20



EXTRA POWERFUL AND RECHARGEABLE

The M-Core B-Li HP is our most powerful device, suitable for those with severe to profound hearing loss, and combines the latest motion technology with rechargeability. Available in Performance Levels 20-80.

Highlights

- › Integrated motion sensor
- › Lithium-ion battery
- › Programmable rocker switch
- › Color Conversion Kits
- › Available as tune T2.0 device
- › Programming via:
 - › NoahLink Wireless

Receiver	Ear hook
FOG/OSPL in dB	82/140
Battery current drain	up to 57 h

Colors



Features / Performance Level	80	60	40	30	20
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4	16/8/4
Soundpro	HighRes	HighRes	HighRes	-	-
My Voice	•	•	•	-	-
Direct Streaming/ Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume
Wireless sync	•	•	•	•	•
Directionality	Automatic/Adaptive iOmni Front/Back Left/Right Narrow	Automatic/Adaptive iOmni Front/Back	Automatic/Adaptive iOmni	Automatic/Adaptive	Automatic/Adaptive
Noise Reduction	Noise Management Sound Smoothing Directional	Noise Management Sound Smoothing Directional	Noise Management Sound Smoothing Directional	Noise Management Sound Smoothing	Noise Management
Wind Noise Reduction	Binaural	Binaural	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	•	•	-	-	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	-
Xphone	•	•	•	-	-
Acclimatization/Data Logging	• / •	• / •	• / •	• / •	• / •
Small ear hook (optional)	•	•	•	•	•
Accessories & Apps					
Standard Charger	•	•	•	•	•
Dry&Clean Charger	•	•	•	•	•
Smart Mic	•	•	•	•	•
Smart Transmitter 2,4	•	•	•	•	•
Smart Key	•	•	•	•	•
Rexton App	•	•	•	•	•

FITTING RANGES

MOTIONCORE B-LI HEARING AIDS

M-CORE B-LI M



M-CORE B-LI P








M-CORE B-LI HP



Battery size

Rechargeable battery (Li-Ion)

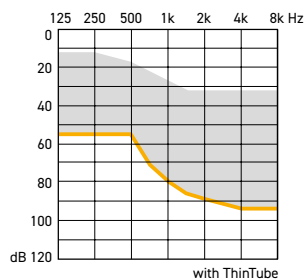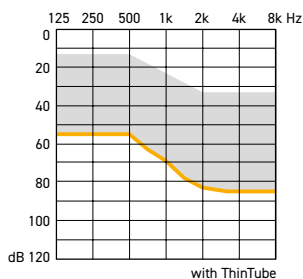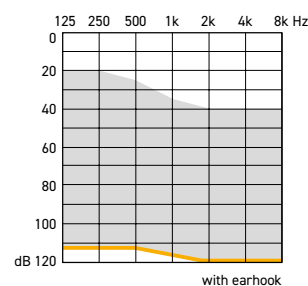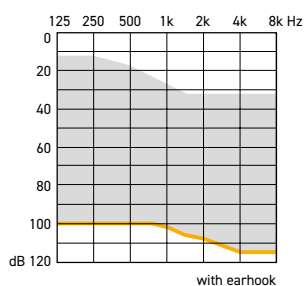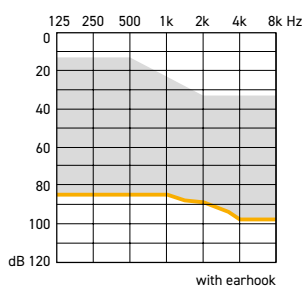

OUR MOST TRUSTED RIC

NOW WITH MOTIONCORE TECHNOLOGY

NOW IN
PERFORMANCE
LEVELS
30 & 20

Our most trusted RIC hearing aid now comes with proven MotionCore technology that automatically recognizes and adapts to more hearing situations than ever. This ensures users automatically hear speech clearer and get the most reliable impression of their environment. M-Core RIC hearing aids are designed to be durable, practical and easy-to-use – with either 312 battery or rechargeable option.

Available colors



Silver



Pearl White



Beige



Rose Gold



Gray



Dark Champagne



Black



Deep Brown



Sandy Brown



Granite



M-CORE R-Li



Performance Level	80, 60, 40 New 30, 20
Battery	Lithium-ion battery
Accessories	Charging Station
	Smart Mic
	Smart Transmitter 2,4
	Rexton App
Features	Integrated Motion Sensor
	Programmable rocker switch
	New Life Tube and tips
	Color Conversion Kits
	Programming via: NoahLink Wireless

- M-Core R-Li features a rechargeable lithium-ion battery format. The rechargeable batteries have a running time of up to 21 h incl. 5 h streaming on one full charge, and the quick charge feature delivers up to 6 hours extra running time in just 30 minutes charging.



Connexx Charging Station

M-CORE R312



Performance Level	80, 60, 40 New 30, 20
Battery	312
Accessories	Smart Mic
	Smart Transmitter 2,4
	Rexton App
Features	Integrated Motion Sensor
	Programmable rocker switch
	New Life Tube and tips
	Color Conversion Kits
	Color Conversion Kits with design integrated telephone coil (M-Core R312T)
	Programming via: NoahLink Wireless

- M-Core R312 is optionally available with an integrated Telecoil for optimal hearing where loop systems are available.



M-Core R-312T with optional telecoil

M-CORE R-LI



Performance Level 80 | 60 | 40 | 30 | 20



THE NEW ULTIMATE RIC DEVICE

Our best sound impression ever! With M-Core R-Li we combine the best hearing comfort, thanks to over 120 hearing situations with a new housing which is more than 15% smaller than its predecessor. All powered with a strong, environment friendly lithium-ion battery with up to 23h runtime. Of course with all connectivity options you value from our Rexton portfolio.

Highlights

- › Integrated motion sensor
- › Lithium-ion battery
- › Programmable rocker switch
- › Mini Receivers 2.0
- › Smart Demo device available
- › Color Conversion Kits
- › Programming via:
 - › NoahLink Wireless

Receiver	S	M	P	HP
FOG/OSPL in dB	45/108	60/119	70/124	75/130
Batt. runtime w/o streaming	up to 23 h	up to 23 h	up to 23 h	up to 23 h
Batt. runtime w. 5h streaming	up to 21 h	up to 20 h	up to 20 h	up to 20 h

Colors



Features / Performance Level	80	60	40	30 New	20 New
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4	16/8/4
Soundpro	HighRes	HighRes	HighRes	-	-
My Voice	•	•	•	-	-
Direct Streaming/ Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume
Wireless sync	•	•	•	•	•
Directionality	Automatic / Adaptive iOmni Front & Back Left & Right Narrow	Automatic / Adaptive iOmni Front & Back	Automatic / Adaptive iOmni	Automatic / Adaptive	Automatic / Adaptive
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management
Wind Noise Reduction	Standard Binaural	Standard Binaural	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	•	•	-	-	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	-
Xphone	•	•	•	-	-
Acclimatization/Data Logging	• / •	• / •	• / •	• / •	• / •
Accessories & Apps					
Connexx charging station	•	•	•	•	•
Smart Mic	•	•	•	•	•
Smart Transmitter 2,4	•	•	•	•	•
Smart Key	•	•	•	•	•
Rexton App	•	•	•	•	•

M-CORE R312



Performance Level 80 | 60 | 40 | 30 | 20



THE ENTIRELY NEW DESIGNED UNIVERSAL DEVICE

Taking hearing to a new level. With the new MotionCore technology, we set a new standard for hearing comfort. With over 120 hearing situations M-Core R312 automatically offers the right setting in every situation, thanks to the integrated motion sensor. Optionally available with a completely integrated telephone coil which can be easily fitted with a color conversion kit.

Highlights

- › Integrated motion sensor
- › Battery size 312
- › Programmable rocker switch
- › Mini Receivers 2.0
- › Smart Demo device available
- › Color Conversion Kits with completely integrated telephone coil
- › Programming via: NoahLink Wireless

Receiver	S	M	P	HP
FOG/OSPL in dB	45/108	60/119	70/124	75/130
Battery current drain	1.7 mA	1.9 mA	1.8 mA	1.8 mA

Colors



Features / Performance Level	80	60	40	30 New	20 New
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4	16/8/4
Soundpro	HighRes	HighRes	HighRes	-	-
My Voice	•	•	•	-	-
Direct Streaming/ Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume	Made for iPhone, Android via Smart Mic/Auto Volume
Wireless sync	•	•	•	•	•
Directionality	Automatic / Adaptive iOmni Front & Back Left & Right Narrow	Automatic / Adaptive iOmni Front & Back	Automatic / Adaptive iOmni	Automatic / Adaptive	Automatic / Adaptive
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management
Wind Noise Reduction	Standard Binaural	Standard Binaural	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	•	•	-	-	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	-
Xphone	•	•	•	-	-
Acclimatization/Data Logging	• / •	• / •	• / •	• / •	• / •
Accessories & Apps					
Smart Mic	•	•	•	•	•
Smart Transmitter 2,4	•	•	•	•	•
Smart Key	•	•	•	•	•
Rexton App	•	•	•	•	•



FITTING RANGES

MOTIONCORE RIC HEARING

M-CORE R-LI



M-CORE R312











RX
M-CORE

Battery size

Rechargeable battery (Li-Ion)

312

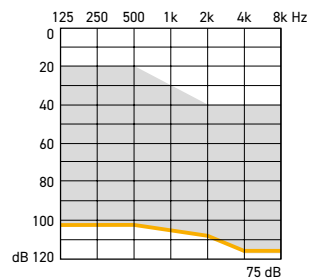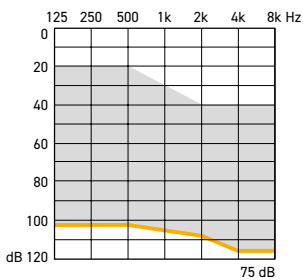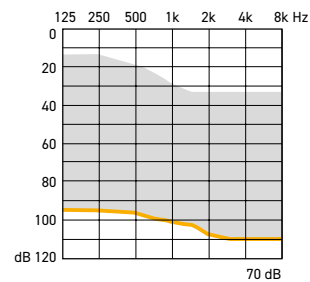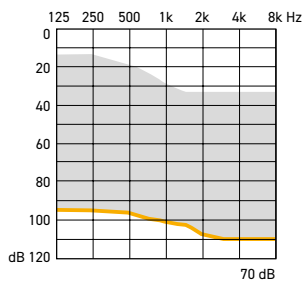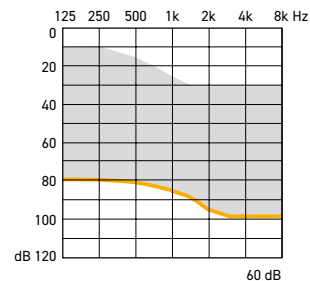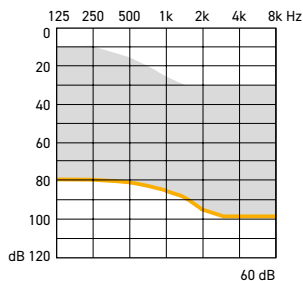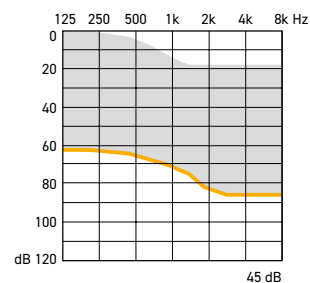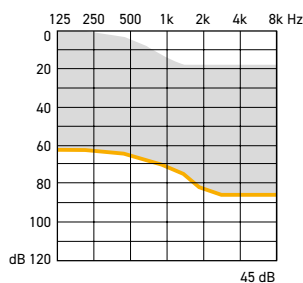

AWARD-WINNING DESIGN

NOW WITH EXCHANGEABLE RECEIVERS

NOW IN
PERFORMANCE
LEVELS
30 & 20

Our award-winning Slim-RIC design now features proven MotionCore technology that automatically recognizes and adapts to up to 20 times more hearing situations than before. This ensures users hear speech clearer and get the most reliable impression of their environment, automatically. And now with exchangeable receivers, you can easily fit M-Core SR hearing aids to more users than ever.

Choose from 5 distinct color combinations



M-CORE SR

Performance Level	80, 60, 40 New 30, 20
Battery	Lithium-ion battery
Accessories	Connexx portable charging case
	Smart Mic
	Smart Transmitter 2,4
	Rexton App
Features	Integrated Motion Sensor
	127 sound situations
	Exchangeable Receivers
	Direct Streaming
	Programming via: NoahLink Wireless

Updated portable charger for power on-the-go

M-Core SR hearing aids feature rechargeable lithium-ion batteries for hassle-free power. The included charging case enables on-the-go top-ups, for up to 4 days of use – so your customers can enjoy a relaxing weekend without having to be near a power socket.





INSTANT FIT

WITH THE HIDDEN POWER OF MOTIONCORE



Our virtually invisible CIC design now features proven MotionCore technology that automatically recognizes and adapts to up to 10 times more hearing situations than before. This ensures users hear speech clearer and get the most reliable impression of their environment, automatically. M-Core iX hearing aids are designed to be discreet, practical and easy-to-use.

M-CORE iX



Performance Level	80, 60, 40 New 30, 20
Battery	10
Accessories	Smart Key Rexton App
Features	HD Soundpro 70 sound situations Instant-Fit Sleeves Programming via: Flex connector HiPro, Connexx Air

Accessories



Smart Transmitter 2.4
Allows for direct streaming of Dolby Digital® stereo sound.



Smart Mic
Remote microphone that allows for streaming from almost any Blue-tooth-enabled phone. May also be used as a remote control.



Smart Key
Discreet remote control to change volume and programs.

M-CORE SR



Performance Level 80 | 60 | 40 | 30 | 20



AWARD-WINNING DESIGN - NOW WITH EXCHANGEABLE RECEIVERS

Our award-winning Slim-RIC design now features proven MotionCore technology that automatically recognizes and adapts to up to 20 times more hearing situations than before. This ensures users hear speech clearer and get the most reliable impression of their environment, automatically. And now with exchangeable receivers, you can easily fit M-Core SR hearing aids to more users than ever.

Highlights

- > Lithium-ion battery
- > Smart Demo device available
- > Programming via:
- > NoahLink Wireless

Receiver	S	M	P
FOG/OSPL in dB	45/108	60/119	70/124
Battery runtime	up to 19 h	up to 19 h	up to 19 h

Colors

Black | Silver



Black | Gloss Black



Cosmic Blue | Rose Gold



Snow White | Rose Gold



Snow White | Gloss White



Features / Performance Level	80	60	40	30	New	20	New
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4		16/8/4	
HD Soundpro	•	•	•	-		-	
My Voice	•	•	•	-		-	
Direct Streaming/ Auto Volume	• / •	• / •	• / •	• / •		• / •	
Wireless sync	•	•	•	•		•	
Directionality	Automatic/Adaptive iOmni Front/Back Left/Right Narrow	Automatic/Adaptive iOmni Front/Back	Automatic/Adaptive iOmni	Automatic/Adaptive		Automatic/Adaptive	
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing		Noise Management	
Wind Noise Reduction	Standard Binaural	Standard Binaural	Standard	Standard		-	
Reverb Reducer	•	-	-	-		-	
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •		- / •	
Music Enhancer: Live / Recording / Acting	• / • / •	• / • / •	-	-		-	
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy		-	
Xphone	•	•	-	-		-	
Acclimatization/Data Logging	• / •	• / •	• / •	• / •		• / •	
Accessories & Apps							
Slim-RIC Travel Charger	•	•	•	•		•	
Smart Key	•	•	•	•		•	
Rexton App	•	•	•	•		•	
Smart Mic	•	•	•	•		•	
Smart Transmitter 2,4	•	•	•	•		•	

M-CORE iX



Performance Level 80 | 60 | 40 | 30 | 20



INSTANT FIT WITH THE HIDDEN POWER OF MOTIONCORE


Our virtually invisible CIC design now features proven MotionCore technology that automatically recognizes and adapts to up to 10 times more hearing situations than before. This ensures users hear speech clearer and get the most reliable impression of their environment, automatically. M-Core iX hearing aids are designed to be discreet, practical and easy-to-use.

Highlights

- > 10 Size battery
- > Smart Demo device available
- > Programming via:
ConnexxAir
Flex connector

FOG/OSPL in dB	60/125
----------------	--------

Colors

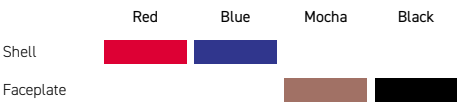

Features / Performance Level	80	60	40	30 New	20 New
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4	16/8/4
HD Soundpro	•	•	•	•	•
My Voice	-	-	-	-	-
Direct Streaming/ Auto Volume	-	-	-	-	-
Wireless sync	•	•	•	•	•
Directionality	Automatic/Adaptive iOmni Front	Automatic/Adaptive iOmni Front	Automatic/Adaptive iOmni	-	-
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management
Wind Noise Reduction	Standard	Standard	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	• / • / •	• / • / •	-	-	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	-
Xphone	•	•	•	-	-
Acclimatization/Data Logging	• / •	• / •	• / •	• / •	• / •
Accessories & Apps					
Rexton App	•	•	•	•	•
Smart Key	•	•	•	•	•



FITTING RANGES

SLIM-RIC AND INSTANT FIT CIC HEARING AIDS

M-CORE SR







M-CORE iX



Battery size

Rechargeable battery (Li-Ion)

10

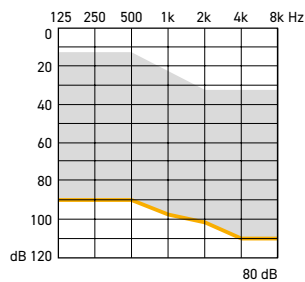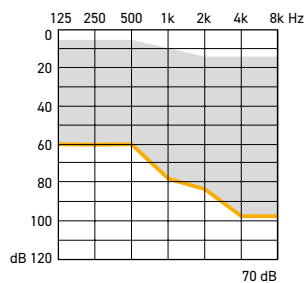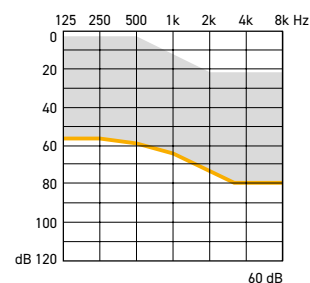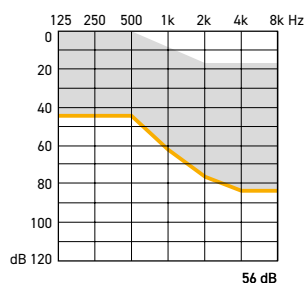

MYCORE

A PERFECT COMBINATION OF NATURAL SOUND AND DIRECT STREAMING



DIRECT STREAMING

Optimally connected to smartphones, TV's and more



MY VOICE

Natural perception of wearer's own voice



STEREO ILOCK

Improves the signal-to-noise ratio in difficult listening situations



VOICE RANGER

Easier one-on-one listening in noisy situations



DYNAMIC EXTENDER

Automatically optimizes the dynamic bandwidth of sounds

DISCREET AND READY TO FIT

inoX combines the discretion of a custom model without having to take an ear impression. It is not only 20% smaller than its predecessor but also offers enormous sound thanks to the MyCore technology. Even directional characteristics are imbedded in a binaural fitting. With its high flexibility inoX can be fitted to nearly every ear.



Highlights

- › Battery size 10
- › Smart Demo device available
- › Programming via:
 - › Flex connector › HiPro
 - › ConnexxAir

FOG/OSPL in dB	50/113
Battery current drain	1.1 mA

Colors

	Red	Blue	Mocha	Black
Shell				
Faceplate				

Features / Performance Level	80	60	40	30	20
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4	16/8/4
Soundpro	•	•	•	-	-
My Voice	-	-	-	-	-
Voice Ranger	•	•	•	•	•
Direct Streaming/ Auto Volume	- / -	- / -	- / -	- / -	- / -
Wireless sync	•	•	•	•	•
Directionality	Automatic / Adaptive bin. Single-mic Directionality	Automatic / Adaptive bin. Single-mic Directionality	Automatic / Adaptive bin. Single-mic Directionality	-	-
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management
Wind Noise Reduction	Standard	Standard	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	• / • / •	• / - / -	-	-	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy	-
Xphone	•	•	•	-	-
Acclimatization/Data Logging	• / •	• / •	• / •	• / •	• / •

Accessories & Apps

Smart Key	•	•	•	•	•
CR0S inoX 8C	•	•	•	-	-
Rexton App	•	•	•	•	•

STERLING IIC & CIC



Performance Level 80 | 60 | 40 | 30 | 20

ULTRA-DISCREET WITH MYCORE FEATURES

Our smallest custom models to offer customers discretion and performance. Maximum discretion ensured by small components and custom-made housing. Binaural signal processing ensures easier listening in loud environments. Discreet Smart key or Rexton App allows wearers to change volume, programs and more with discretion.



Highlights

- > Battery size 10
- > Programming via:
 - > ConnexxAir
 - > FlexConnector (CIC only)

	50 dB	55db (CIC only)	65 dB (CIC only)
FOG/OSPL in dB	50/113	55/118	65/124
Battery current drain	1.3 mA	1.3 mA	1.3 mA

Colors

	Mocha	Black	Brown
IIC			
CIC			

Features / Performance Level	80	60	40	30 New	20 New
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4	16/8/4
Soundpro	•	•	•	-	-
My Voice	-	-	-	-	-
Voice Ranger	•	•	•	•	•
Direct Streaming/ Auto Volume	- / -	- / -	- / -	- / -	- / -
Wireless sync	•	•	•	•	•
Directionality	Automatic / Adaptive bin. Single-mic Directionality	Automatic / Adaptive bin. Single-mic Directionality	Automatic / Adaptive bin. Single-mic Directionality	-	-
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management
Wind Noise Reduction	Standard	Standard	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	• / • / •	• / - / -	-	-	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy	-
Xphone	•	•	•	-	-
Acclimatization/Data Logging	• / •	• / •	• / •	• / •	• / •

Accessories & Apps					
Smart Key	•	•	•	•	•
CROS inoX 8C	•	•	•	-	-
Rexton App	•	•	•	•	•



PROVEN DESIGN - CUTTING TECHNOLOGY

The most popular custom design with MyCore technology. This hearing system provides a pleasant sound experience with excellent listening comfort and high amplification power. Wireless functionality up to 65 dB possible. Binaural signal processing ensures an easier listening in loud environments. Small sized remote or Rexton App allows wearers to change volume, programs, and more with discretion.

Highlights

- › Battery size 312
- › T-coil option (non-BT)
- › Programming via:
 - › NoahLink Wireless (BT)
 - › ConnexxAir / › HiPro

	50 dB	55 dB	65 dB
FOG/OSPL in dB	50/113	55/118	65/124
Battery current drain	1.3 mA	1.3 mA	1.3 mA

Colors



Features / Performance Level	80 / 80 BT	60 / 60 BT	40 / 40 BT	30 New	20 New
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4	16/8/4
Soundpro	•	•	•	-	-
My Voice	-	-	-	-	-
Voice Ranger	•	•	•	•	•
Direct Streaming/ Auto Volume (BT version only)	Made for iPhone via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone via TV Transmitter & Smart Mic / Auto Volume	- / -	- / -
Wireless sync	•	•	•	•	•
Directionality	Automatic / Adaptive Front & Back Left & Right Narrow - Stereo iLock	Automatic / Adaptive Front & Back Narrow - iLock	Automatic / Adaptive Narrow - iLock	-	-
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management
Wind Noise Reduction	Standard Binaural (BT)	Standard Binaural (BT)	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	• / • / •	• / - / -	-	-	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy	-
Xphone	•	•	•	-	-
Acclimatization/Data Logging	• / •	• / •	• / •	• / •	• / •
Accessories & Apps					
Smart Mic	•	•	•		
Smart Transmitter 2,4	•	•	•		
Smart Key	•	•	•		
CROS inoX 8C	-	-	-		
CROS RIC 8C	•	•	•		
Rexton App	•	•	•		



THE CUSTOM MADE IN-THE-EAR PRODUCT FEATURING DIRECT AUDIO STREAMING

The Sterling ITE is Rexton's first hearing aid with Direct Audio Streaming technology via Bluetooth® and standard wireless. This makes the hearing aid a universally networked in-ear communication center. Binaural signal processing ensure easier listening in loud environments. The small-sized remote or App enable wearers to discreetly change volume, programs and more.

Highlights

- › Battery size 312
- › T-coil option (non-BT)
- › Programming via:
 - › NoahLink Wireless (BT)
 - › ConnexxAir / › HiPro

	55 dB	65 dB
FOG/OSPL in dB	55/118	65/124
Battery current drain	1.3 mA	1.3 mA

Colors



Features / Performance Level	80 / 80 BT	60 / 60 BT	40 / 40 BT	30 New	20 New
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4	16/8/4
Soundpro	•	•	•	-	-
My Voice	-	-	-	-	-
Voice Ranger	•	•	•	•	•
Direct Streaming/ Auto Volume (BT version only)	Made for iPhone via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone via TV Transmitter & Smart Mic / Auto Volume	- / -	- / -
Wireless sync	•	•	•	•	•
Directionality	Automatic / Adaptive Front & Back Left & Right Narrow - Stereo iLock	Automatic / Adaptive Front & Back Narrow - iLock	Automatic / Adaptive Narrow - iLock	-	-
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management
Wind Noise Reduction	Standard Binaural (BT)	Standard Binaural (BT)	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	• / • / •	• / - / -	-	-	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy	-
Xphone	•	•	•	-	-
Acclimatization/Data Logging	• / •	• / •	• / •	• / •	• / •
Accessories & Apps					
Smart Mic	•	•	•		
Smart Transmitter 2,4	•	•	•		
Smart Key	•	•	•		
CROS inoX 8C	-	-	-		
CROS RIC 8C	•	•	•		
Rexton App	•	•	•		

FITTING RANGES

CUSTOM HEARING AIDS



INOX

**STERLING
IIC & CIC**

IIC ONLY AVAILABLE MATRIX
113/50

**STERLING
ITC**

**STERLING
ITE**












Battery size

10

10

312

312

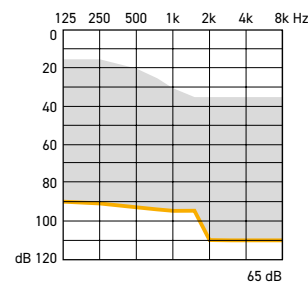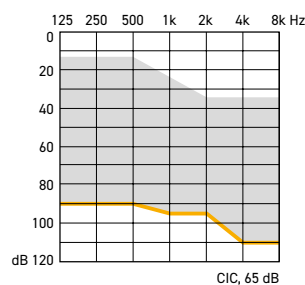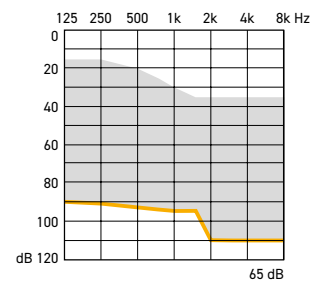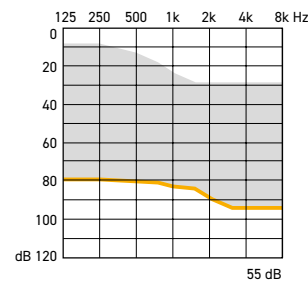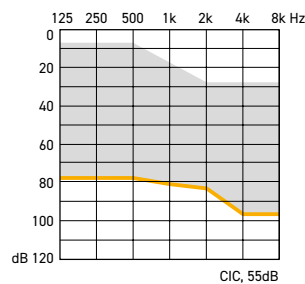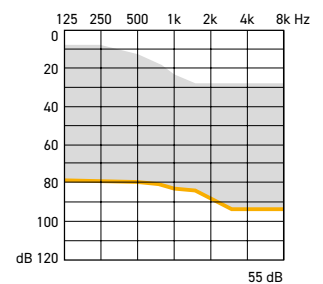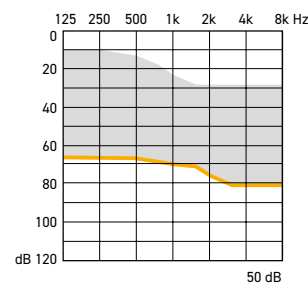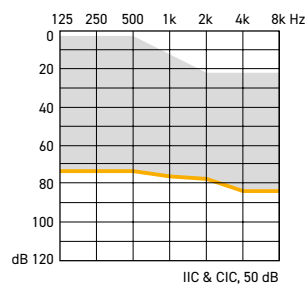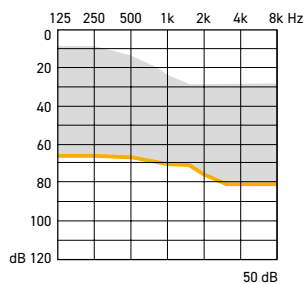



INNOVATIVE CHARGING CONCEPT MEETS SLIM-RIC FORM FACTOR AND MOBILE CHARGING CASE

Rexton StyleLine - proven to overcome the hearing aid stigma in a new, previously unmatched way. Higher consumer approval than any other hearing aid. the perfect approach for first-time wearers. MyCore technology, innovative design and a new mobile charging system that enables wearers to be on the go for up to four days without needing any power connection, this is the true choice for mobile and active wearers. And with Smart Key or Rexton App, the full functionality of StyleLine can be remotely set and superior hearing quality easily enjoyed.

Highlights

- > Lithium-ion battery
- > Smart Demo device available
- > Programming via: ConnexxAir

Receiver	M
FOG/OSPL in dB	60/119
Battery runtime	up to 19 h

Colors

Dark Granite | Silver



Snow White | Rose Gold



Cosmic Blue | Rose Gold



Features / Performance Level	80	60	40	30	20
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4	16/8/4
Soundpro	•	•	•	-	-
My Voice	•	•	•	-	-
Direct Streaming/ Auto Volume	- / -	- / -	- / -	- / -	- / -
Wireless sync	•	•	•	•	•
Directionality	Automatic/Adaptive iOmni Front/Back Left/Right Narrow - Stereo iLock	Automatic/Adaptive iOmni Front/Back Narrow - iLock	Automatic/Adaptive iOmni Narrow - iLock	Automatic/Adaptive iOmni	Automatic/Adaptive iOmni
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management
Wind Noise Reduction	Standard Binaural	Standard Binaural	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	• / • / •	• / - / -	-	-	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy	-
Xphone	•	•	•	-	-
Acclimatization/Data Logging	• / •	• / •	• / •	• / •	• / •

Accessories & Apps					
Mobile charging system	•	•	•	•	•
Smart Key	•	•	•	•	•
Rexton App	•	•	•	•	•

STYLELINE BT



Performance Level 80 | 60 | 40



ENJOY LEADING-EDGE MOBILITY AND FULL STREAMING WITH REXTON SLIM-RIC DEVICE

StyleLine BT offers the same reliable feature set as StyleLine, such as reliable MyCore technology and the truly innovative mobile charging system. On top, StyleLine BT integrates direct audio streaming via Bluetooth to iOS devices or other Bluetooth enabled Android systems (via Smart Mic)

Highlights

- > Lithium-ion battery
- > Smart Demo device available
- > Programming via:
- > NoahLink Wireless

Receiver	M
FOG / OSPL in dB	60/119
Battery runtime without streaming	up to 19 h
Battery runtime incl. 5h streaming	up to 16 h

Colors

Black | Silver



Snow White | Rose Gold



Cosmic Blue | Rose Gold



Features / Performance Level	80	60	40
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6
Soundpro	•	•	•
My Voice	•	•	•
Direct Streaming/ Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume
Wireless sync	•	•	•
Directionality	Automatic/Adaptive iOmni Front/Back Left/Right Narrow - Stereo iLock	Automatic/Adaptive iOmni Front/Back Narrow - iLock	Automatic/Adaptive iOmni Narrow - iLock
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing
Wind Noise Reduction	Standard Binaural	Standard Binaural	Standard
Reverb Reducer	•	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	• / • / •	• / - / -	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy
Xphone	•	•	•
Acclimatization/Data Logging	• / •	• / •	• / •
Accessories & Apps			
Mobile charging system	•	•	•
Smart Mic	•	•	•
Smart Transmitter 2,4	•	•	•
Smart Key	•	•	•
Rexton App	•	•	•



FITTING RANGES

SLIM-RIC HEARING AIDS

STYLELINE





STYLELINE BT



Battery size

Rechargeable battery (Li-Ion)

Rechargeable battery (Li-Ion)

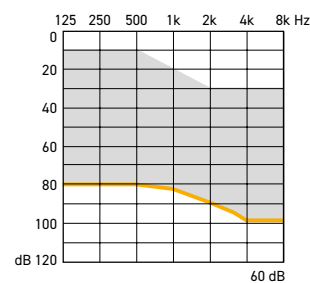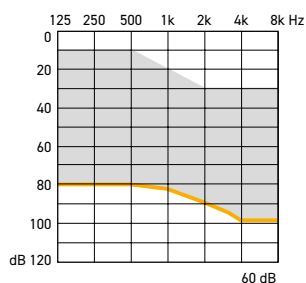

EMERALD XS



Performance Level 80 | 60 | 40 | 30 | 20



OUR SMALLEST RIC WITH NEAR INVISIBILITY BEHIND THE EAR

The Emerald XS is the ideal combination of small size and advanced functionality. Thanks to its durable design, you can enjoy the sounds of nature and music discreetly, no matter your lifestyle or degree or hearing loss. Perfect for first-time users demanding high discretion. Smart Key or Rexton App enable wearers to change volume, programs, and more.

Highlights

- > Battery size 10
- > Mini Receivers 2.0
- > Smart Demo device available
- > Programming via:
 - > HiPro

Receiver	S	M	P
FOG/OSPL in dB	45/108	55/113	60/118
Battery current drain	1.2 mA	1.3 mA	1.2 mA

Colors



Features / Performance Level	80	60	40	30	20
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4	16/8/4
Soundpro	•	•	•	-	-
My Voice	-	-	-	-	-
Direct Streaming/ Auto Volume	- / -	- / -	- / -	- / -	- / -
Wireless sync	-	-	-	-	-
Directionality	Automatic / Adaptive iOmni Front & Back	Automatic / Adaptive iOmni Front & Back	Automatic / Adaptive iOmni	Automatic / Adaptive iOmni	Automatic / Adaptive iOmni
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management
Wind Noise Reduction	Standard	Standard	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	• / • / •	• / - / -	-	-	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy	-
Xphone	-	-	-	-	-
Acclimatization/Data Logging	• / •	• / •	• / •	• / •	• / •
Accessories & Apps					
Smart Key	•	•	•	•	•
Rexton App	•	•	•	•	•

EMERALD S



Performance Level 80 | 60 | 40 | 30 | 20



A PERFECT COMBINATION OF SMALL SIZE AND PERFORMANCE

This RIC 312 boasts direct audio streaming from iPhone via Bluetooth® and streaming with most Bluetooth® – enabled phones via Smart Mic. Emerald S is small yet fully featured. Its variety of accessories offers full control and usability, e.g. TVs and other Bluetooth® enabled devices via Smart Transmitter 2.4. My Voice ensures a wearer's natural own voice experience by processing it differently than other voices and sounds.

Highlights

- › Battery size 312
- › Mini Receivers 2.0
- › Smart Demo device available
- › Programming via: NoahLink Wireless
- › HiPro

Receiver	S	M	P	HP
FOG/OSPL in dB	45/108	60/119	70/124	75/130
Battery current drain	1.2 mA	1.4 mA	1.3 mA	1.3 mA

Colors



Features / Performance Level	80	60	40	30	20
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4	16/8/4
Soundpro	•	•	•	-	-
My Voice	•	•	•	-	-
Direct Streaming/ Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume
Wireless sync	•	•	•	•	•
Directionality	Automatic / Adaptive iOmni Front & Back Left & Right Narrow - Stereo iLock	Automatic / Adaptive iOmni Front & Back Narrow - iLock	Automatic / Adaptive iOmni Narrow - iLock	Automatic / Adaptive iOmni	Automatic / Adaptive iOmni
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management
Wind Noise Reduction	Standard Binaural	Standard Binaural	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	• / • / •	• / - / -	-	-	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy	-
Xphone	•	•	•	-	-
Acclimatization/Data Logging	• / •	• / •	• / •	• / •	• / •

Accessories & Apps					
Smart Mic	•	•	•	•	•
Smart Transmitter 2.4	•	•	•	•	•
Smart Key	•	•	•	•	•
CROS RIC 8C	•	•	•	-	-
Rexton App	•	•	•	•	•

STELLAR RIC



Performance Level 80 | 60 | 40 | 30 | 20



LITHIUM-ION RECHARGEABLE RIC THAT PROVIDES MYCORE HEARING THAT LASTS ALL DAY

Stellar RIC is our lithium-ion rechargeable RIC that does not require any battery changes. Stellar RIC is suitable for up to severe hearing loss, and with an IP68 rating it can stand up to moisture, sweat and dirt. Contactless charging for easy, automatic charging that stops when fully charged. My Voice ensures a wearer's natural own voice experience.

Highlights

- > Lithium-ion battery
- > Mini Receivers 2.0
- > Smart Demo device available
- > Programming via: NoahLink Wireless

Receiver	S	M	P	HP
FOG/OSPL in dB	45/108	60/119	70/124	75/130
Batt. runtime w/o streaming	up to 21 h	up to 21 h	up to 21 h	up to 21 h
Batt. runtime w. 5h streaming	up to 19 h	up to 19 h	up to 19 h	up to 19 h

Colors



Features / Performance Level	80	60	40	30	20
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4	16/8/4
Soundpro	•	•	•	-	-
My Voice	•	•	•	-	-
Direct Streaming/ Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume
Wireless sync	•	•	•	•	•
Directionality	Automatic/Adaptive iOmni Front/Back Left/Right Narrow - Stereo iLock	Automatic/Adaptive iOmni Front/ Back Narrow - iLock	Automatic/Adaptive iOmni Narrow - iLock	Automatic/Adaptive iOmni	Automatic/Adaptive iOmni
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management
Wind Noise Reduction	Standard Binaural	Standard Binaural	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	• / • / •	• / - / -	-	-	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy	-
Xphone	•	•	•	-	-
Acclimatization/Data Logging	• / •	• / •	• / •	• / •	• / •

Accessories & Apps

Charging Station	•	•	•	•	•
Smart Li-Ion Power	•	•	•	•	•
Smart Mic	•	•	•	•	•
Smart Transmitter 2.4	•	•	•	•	•
Smart Key	•	•	•	•	•
CROS RIC 8C	•	•	•	-	-
CROS Li RIC 8C	•	•	•	-	-
Rexton App	•	•	•	-	-

EMERALD M



Performance Level 80 | 60 | 40 | 30 | 20



GREAT NEW FEATURES AND BENEFITS IN A SMALL HEARING AID

The Emerald M is the ideal combination of small size and long battery lifetime. With up to 75dB of gain, Emerald M fits for even severe hearing loss. The size 13 battery and low power consumption of the hearing aid ensure longest running time. Direct connectivity to iPhones, TVs and other Bluetooth® enabled devices via Smart Transmitter 2.4 and Smart Mic. My Voice ensures a wearer's natural own voice experience by processing it differently than other voices and sounds. And the optional battery door with integrated T-Coil ensures optimal understanding in locations equipped with loop system.

Highlights

- › Battery size 13
- › Mini Receivers 2.0
- › Smart Demo device available
- › Programming via:
 - › NoahLink Wireless
 - › HiPro

Receiver	S	M	P	HP
FOG/OSPL in dB	45/108	60/119	70/124	75/130
Battery current drain	1.2 mA	1.4 mA	1.3 mA	1.3 mA

Colors



Features / Performance Level	80	60	40	30	20
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4	16/8/4
Soundpro	•	•	•	-	-
My Voice	•	•	•	-	-
Direct Streaming/ Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume
Wireless sync	•	•	•	•	•
Directionality	Automatic / Adaptive iOmni Front & Back Left & Right Narrow - Stereo iLock	Automatic / Adaptive iOmni Front & Back Narrow - iLock	Automatic / Adaptive iOmni Narrow - iLock	Automatic / Adaptive iOmni	Automatic / Adaptive iOmni
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management
Wind Noise Reduction	Standard Binaural	Standard Binaural	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	• / • / •	• / - / -	-	-	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy	-
Xphone	•	•	•	-	-
Acclimatization/Data Logging	• / •	• / •	• / •	• / •	• / •

Accessories & Apps					
Smart Mic	•	•	•	•	•
Smart Transmitter 2,4	•	•	•	•	•
Smart Key	•	•	•	•	•
CROS RIC 8C	•	•	•	-	-
Rexton App	•	•	•	-	-

FITTING RANGES

MYCORE RIC HEARING AIDS

EMERALD XS










EMERALD S



Battery size

10

312

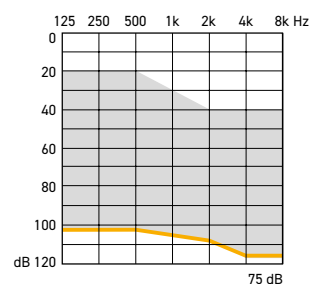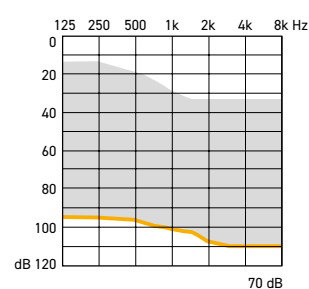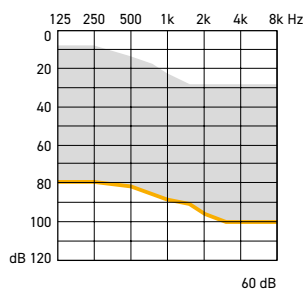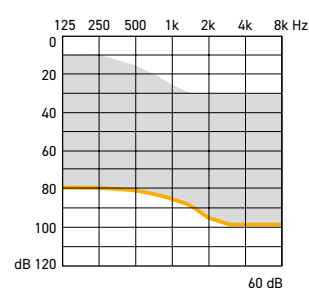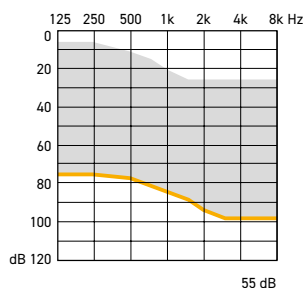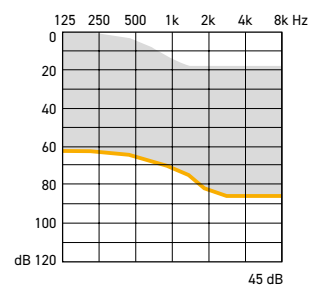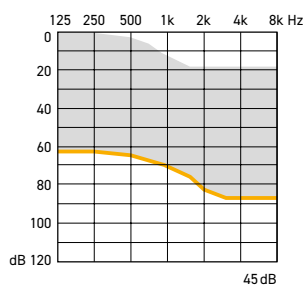

STELLAR RIC











EMERALD M



Rechargeable battery (Li-Ion)

13

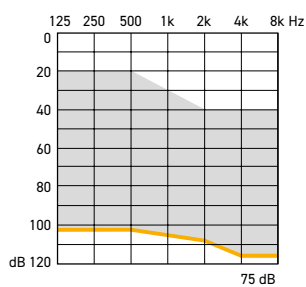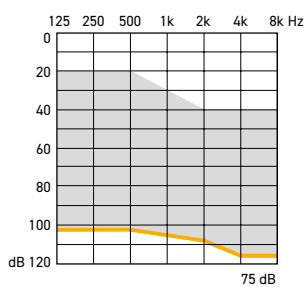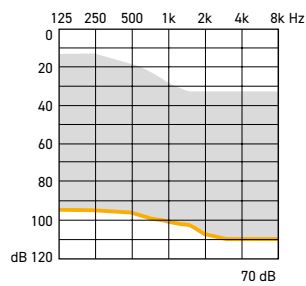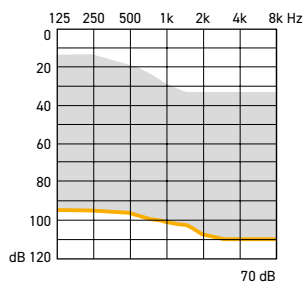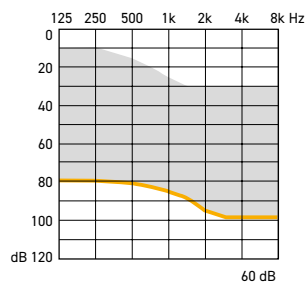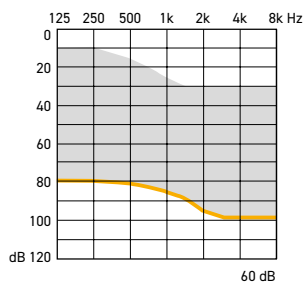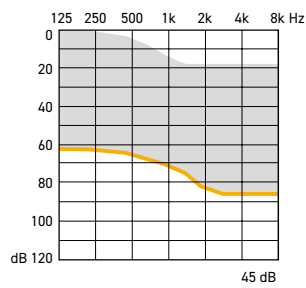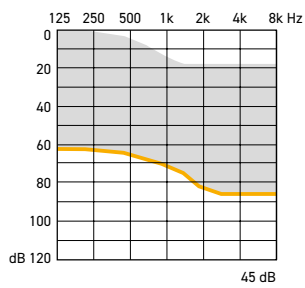

MOSAIC M



Performance Level 80 | 60 | 40 | 30 | 20



THE RELIABLE BTE THAT FITS ANY EAR

The Mosaic M is the ideal combination of small size and long battery lifetime. With up to 63 dB of gain, Mosaic M fits up to severe hearing loss. The size 13 battery and low power consumption ensures longest running time. It offers direct connectivity to iPhones, TVs and other Bluetooth® enabled devices. My Voice ensures a wearer's natural own voice experience by processing it differently than other voices and sounds. Mosaic M is also available with an optional telecoil, integrated in the battery door.

Highlights

- › Battery size 13
- › Smart Demo device available
- › Optional integrated telecoil
- › Programming via:
 - › NoahLink Wireless
 - › HiPro

	Ear hook	ThinTube
FOG/OSPL in dB	63/134	56/126
Battery current drain	1.4 mA	2.0 mA

Colors



Features / Performance Level	80	60	40	30	20
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4	16/8/4
Soundpro	•	•	•	-	-
My Voice	•	•	•	-	-
Direct Streaming/ Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume
Wireless sync	•	•	•	•	•
Directionality	Automatic / Adaptive iOmni Front & Back Left & Right Narrow - Stereo iLock	Automatic / Adaptive iOmni Front & Back Narrow - iLock	Automatic / Adaptive iOmni Narrow - iLock	Automatic / Adaptive iOmni	Automatic / Adaptive iOmni
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management
Wind Noise Reduction	Standard Binaural	Standard Binaural	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	• / • / •	• / - / -	-	-	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy	-
Xphone	•	•	•	-	-
Acclimatization/Data Logging	• / •	• / •	• / •	• / •	• / •

Accessories & Apps					
Smart Mic	•	•	•	•	•
Smart Transmitter 2,4	•	•	•	•	•
Smart Key	•	•	•	•	•
CROS RIC 8C	•	•	•	-	-
Rexton App	•	•	•	•	•

STELLAR M



Performance Level 80 | 60 | 40 | 30 | 20



COMPLETE MYCORE FEATURE SET IN EASY-TO-HANDLE BTE WITH LITHIUM-ION RECHARGEABILITY

With Stellar M, Rexton offers the MyCore feature set including My Voice and direct streaming via Bluetooth® in a robust, easy to handle BTE hearing instrument. Contactless charging for easy, automatic charging that stops when fully charged. Direct connectivity to iPhones, TVs and other Bluetooth®-enabled devices.

Highlights

- > Lithium-ion battery
- > Smart Demo device available
- > Programming via: NoahLink Wireless

	Ear hook	ThinTube
FOG/OSPL in dB	63/134	56/126
Battery runtime without streaming	up to 21 h	up to 21 h
Battery runtime incl. 5h streaming	up to 19 h	up to 19 h

Colors



Features / Performance Level	80	60	40	30	20
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4	16/8/4
Soundpro	•	•	•	-	-
My Voice	•	•	•	-	-
Direct Streaming/ Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume
Wireless sync	•	•	•	•	•
Directionality	Automatic / Adaptive iOmni Front & Back Left & Right Narrow - Stereo iLock	Automatic / Adaptive iOmni Front & Back Narrow - iLock	Automatic / Adaptive iOmni Narrow - iLock	Automatic / Adaptive iOmni	Automatic / Adaptive iOmni
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management
Wind Noise Reduction	Standard Binaural	Standard Binaural	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	• / • / •	• / - / -	-	-	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy	-
Xphone	•	•	•	-	-
Acclimatization/Data Logging	• / •	• / •	• / •	• / •	• / •

Accessories & Apps					
Connexx charging station	•	•	•	•	•
Smart Li-Ion Power	•	•	•	•	•
Smart Mic	•	•	•	•	•
Smart Transmitter 2,4	•	•	•	•	•
Smart Key	•	•	•	•	•
CROS RIC 8C	•	•	•	-	-
CROS Li RIC 8C	•	•	•	-	-
Rexton App	•	•	•	•	•

MOSAIC P



Performance Level 80 | 60 | 40 | 30 | 20



MOSAIC P: POWER - PURE AND SIMPLE

Mosaic P is perfect for patients who need extra amplification. The Mosaic P features a long-lasting battery with low power consumption and the flexibility you need to treat even severe hearing loss. Direct connectivity to iPhones, TVs and other Bluetooth® enabled devices via Smart Transmitter 2.4 and Smart Mic. My Voice ensures a wearer's natural own voice experience by processing it differently than other voices and sounds. Optional battery door with integrated T-Coil ensures optimal understanding in locations equipped with loop system.

Highlights

- › Battery size 13
- › Smart Demo device available
- › Programming via:
 - › NoahLink Wireless
 - › HiPro

	Ear hook	ThinTube
FOG/OSPL in dB	77/135	65/130
Battery current drain	2.0 mA	1.7 mA

Colors



Features / Performance Level	80	60	40	30	20
Channels/Controls/Programs	48/20/6	32/16/6	24/12/6	16/8/4	16/8/4
Soundpro	•	•	•	-	-
My Voice	•	•	•	-	-
Direct Streaming/ Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume	Made for iPhone, Android via TV Transmitter & Smart Mic / Auto Volume
Wireless sync	•	•	•	•	•
Directionality	Automatic / Adaptive iOmni Front & Back Left & Right Narrow - Stereo iLock	Automatic / Adaptive iOmni Front & Back Narrow - iLock	Automatic / Adaptive iOmni Narrow - iLock	Automatic / Adaptive iOmni	Automatic / Adaptive iOmni
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management
Wind Noise Reduction	Standard Binaural	Standard Binaural	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live / Recording / Acting	• / • / •	• / - / -	-	-	-
Tinnitus	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy Notch Therapy	Sound Therapy	-
Xphone	•	•	•	-	-
Acclimatization/Data Logging	• / •	• / •	• / •	• / •	• / •

Accessories & Apps					
Smart Mic	•	•	•	•	•
Smart Transmitter 2,4	•	•	•	•	•
Smart Key	•	•	•	•	•
CROS RIC 8C	•	•	•	-	-
Rexton App	•	•	•	•	•

FITTING RANGES

MYCORE BTE HEARING AIDS

MOSAIC M

STELLAR M

MOSAIC P









Battery size

13

Rechargeable battery (Li-Ion)

13

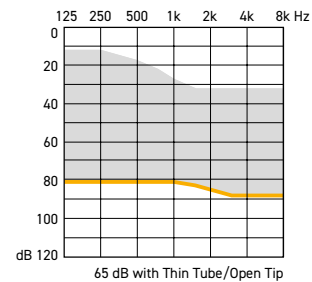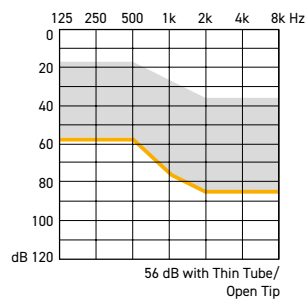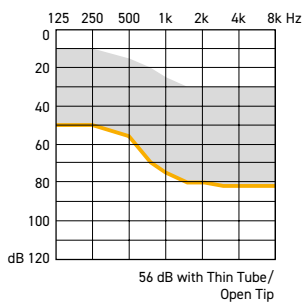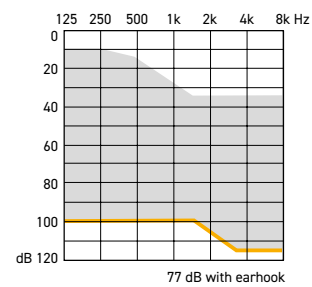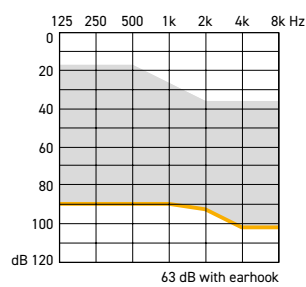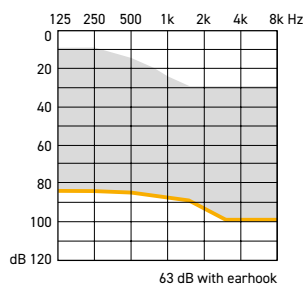



TRUCORE

TRULY CONVINCING TECHNOLOGY



VOICE RANGER

Delivering a more relaxed listening experience



MUSIC ENHANCER

Maximizing listening enjoyment for music lovers



XPHONE

Sending telephone signals automatically to both hearing aids



REVERB REDUCER

Pleasant communication in an echoing room



CUSTOM DISCRETION WITHOUT CUSTOM PRODUCTION

The shape of the inoX housing follows the natural auditory canal geometry and fits for nearly every ear. As discreet as custom CIC's, but without the hassle of custom production. An ideal solution for first-time wearers who favor discretion and convenience above all other. Also available as ITC.

Highlights

- › Battery size 10 (CIC)
- › Battery size 312 (ITC)
- › Programming via:
 - › Programming cable Flex-Connect
 - › ConnexxAir (PL80 to 20)

	inoX CIC	inoX ITC
FOG/OSPL in dB	50/113	55/116
Battery current drain	1.1 mA	1.1 mA

Colors

PL 80-20

PL 10

	Red	Blue	Mocha	Beige	Mocha
Shell					
Faceplate					

Features / Performance Level	80	60	40	30	20	10
Channels/Controls/Programs	40 / 20 / 6	32 / 16 / 6	24 / 12 / 6	16 / 8 / 4	16 / 8 / 4	12 / 6 / 4
Smart Automatic Equalizer	•	•	•	-	-	-
Wireless Sync	•	•	•	•	•	-
Directionality	Automatic / Adaptive bin. Single-mic Directionality	-	-	-	-	-
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management	Noise Management
Wind Noise Cancellation	Standard	Standard	Standard	Standard	-	-
Reverb Reducer	•	-	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •	- / -
Music Enhancer: Live/Recording/Acting	• / • / •	• / - / -	• / - / -	-	-	-
Tinnitus Function	Sound Therapy	Sound Therapy	Sound Therapy	Sound Therapy	-	-
Xphone	-	-	-	-	-	-
Acclimatization / Data Logging	• / •	• / •	• / •	• / •	• / •	- / •

Accessories & Apps						
Smart Connect	•	•	•	•	•	-
Smart Transmitter	•	•	•	•	•	-
Smart Remote	•	•	•	•	•	-
Smart Key	•	•	•	•	•	•
CROS RIC	•	•	•	-	-	-
Smart Remote App	•	•	•	•	•	•
Smart Connect App	•	•	•	•	•	-
Rexton App	•	•	•	•	•	•



PROVEN DESIGN










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
This hearing system provides a pleasant sound experience with excellent listening comfort and high amplification power. Wireless functionality up to 65 dB possible. Binaural signal processing ensures easier listening in loud environments. Smart Key or Rexton App enables wearers to discreetly change volume, programs, and more. Sterling ITC is available with an optional telecoil.

Highlights

- › Battery size 10
- › Programming via:
 - › Programming cable
 - Flex-Connect
 - › ConnexxAir (80 to 20 wireless)

	50 dB	55 dB	65 dB
FOG/OSPL in dB	50/113	55/118	65/124
Battery current drain	0.8 mA	0.8 mA	1.0 mA

Colors

	Beige	Brown	Tan	Mocha	Dark Brown
Shell					
Faceplate					

Features / Performance Level	80	60	40	30	20	10
Channels/Controls/Programs	40 / 20 / 6	32 / 16 / 6	24 / 12 / 6	16 / 8 / 4	16 / 8 / 4	12 / 6 / 4
Smart Automatic Equalizer	•	•	•	-	-	-
Wireless Sync	•	•	•	•	•	-
Directionality	Automatic / Adaptive bin. Single-mic Directionality	-	-	-	-	-
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management	Noise Management
Wind Noise Cancellation	Standard	Standard	Standard	Standard	-	-
Reverb Reducer	•	-	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •	- / -
Music Enhancer: Live/Recording/Acting	• / • / •	• / - / -	• / - / -	-	-	-
Tinnitus Function	Sound Therapy	Sound Therapy	Sound Therapy	Sound Therapy	-	-
Xphone	-	-	-	-	-	-
Acclimatization / Data Logging	• / •	• / •	• / •	• / •	• / •	- / •

Accessories & Apps						
Smart Connect	•	•	•	•	•	-
Smart Transmitter	•	•	•	•	•	-
Smart Remote	•	•	•	•	•	-
Smart Key	•	•	•	•	•	•
CROS RIC	•	•	•	-	-	-
Smart Remote App	•	•	•	•	•	•
Smart Connect App	•	•	•	•	•	-
Rexton App	•	•	•	•	•	•

MODERN AND COMFORTABLE

The high-performance model with programmable push button. Wireless functionality up to 65 dB possible. Binaural signal processing ensures easier listening in loud environments. Smart Key or the Rexton App enable wearers to discreetly volume, programs and more.













Highlights

- > Battery size 312 and 13
- > Optional telecoil
- > Programming via:
 - > Programming cable
 - Flex-Connect
 - > ConnexxAir (80 to 20 wireless)

	50 dB	55 dB	65 dB
FOG/OSPL in dB	50/113	55/118	65/124
Battery current drain	0.8 mA	0.9 mA	1.0 mA

Colors

	Beige	Brown	Tan	Mocha	Dark Brown
Shell					
Faceplate					

Features / Performance Level	80	60	40	30	20	10
Channels/Controls/Programs	40 / 20 / 6	32 / 16 / 6	24 / 12 / 6	16 / 8 / 4	16 / 8 / 4	12 / 6 / 4
Smart Automatic Equalizer	•	•	•	-	-	-
Wireless Sync	•	•	•	•	•	-
Directionality	•	•	•	•	•	-
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management	Noise Management
Wind Noise Cancellation	Premium	High	Standard	Standard	-	-
Reverb Reducer	•	-	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •	- / -
Music Enhancer: Live/Recording/Acting	• / • / •	• / - / -	• / - / -	-	-	-
Tinnitus Function	Sound Therapy	Sound Therapy	Sound Therapy	Sound Therapy	-	-
Xphone	•	•	•	-	-	-
Acclimatization / Data Logging	• / •	• / •	• / •	• / •	• / •	- / •

Accessories & Apps						
Smart Connect	•	•	•	•	•	-
Smart Transmitter	•	•	•	•	•	-
Smart Remote	•	•	•	•	•	-
Smart Key	•	•	•	•	•	•
CROS RIC	•	•	•	-	-	-
Smart Remote App	•	•	•	•	•	•
Smart Connect App	•	•	•	•	•	-
Rexton App	•	•	•	•	•	•

STERLING ITE



Performance Level 80 | 60 | 40 | 30 | 20 | 10



INNOVATIVE AND UNIQUE

Our first custom model with 2 microphones embedded in the faceplate delivers a new dimension of sound quality. And connected with the Rexton app, Sterling ITE is a most discreet hearing aid and more. Sterling ITE is available with an optional telecoil.

Highlights

- > Battery size 312 and 13
- > Optional telecoil
- > Programming via:
 - > Programming cable
 - Flex-Connect
 - > ConnexxAir (80 to 20 wireless)

	55 dB	65 dB
FOG/OSPL in dB	55/118	65/124
Battery current drain	0.9 mA	1.0 mA

Colors

	Beige	Brown	Tan	Mocha	Dark Brown
Shell					
Faceplate					

Features / Performance Level	80	60	40	30	20	10
Channels/Controls/Programs	40 / 20 / 6	32 / 16 / 6	24 / 12 / 6	16 / 8 / 4	16 / 8 / 4	12 / 6 / 4
Smart Automatic Equalizer	•	•	•	-	-	-
Wireless Sync	•	•	•	•	•	-
Directionality	•	•	•	•	•	-
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management	Noise Management
Wind Noise Cancellation	Premium	High	Standard	Standard	-	-
Reverb Reducer	•	-	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •	- / -
Music Enhancer: Live/Recording/Acting	• / • / •	• / - / -	• / - / -	-	-	-
Tinnitus Function	Sound Therapy	Sound Therapy	Sound Therapy	Sound Therapy	-	-
Xphone	•	•	•	-	-	-
Acclimatization / Data Logging	• / •	• / •	• / •	• / •	• / •	- / •

Accessories & Apps						
Smart Connect	•	•	•	•	•	-
Smart Transmitter	•	•	•	•	•	-
Smart Remote	•	•	•	•	•	-
Smart Key	•	•	•	•	•	•
CROS RIC	•	•	•	-	-	-
Smart Remote App	•	•	•	•	•	•
Smart Connect App	•	•	•	•	•	-
Rexton App	•	•	•	•	•	•

TRUCORE

CUSTOM HEARING AIDS OVERVIEW

INOX






STERLING CIC



Battery size

10 / 312

10

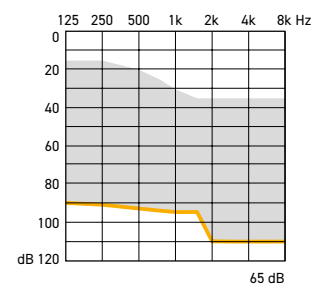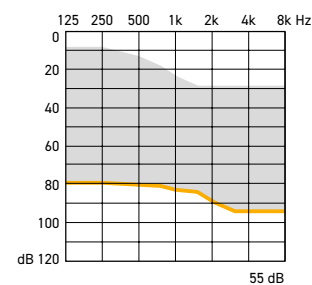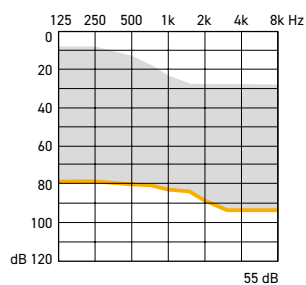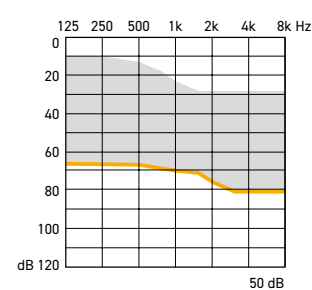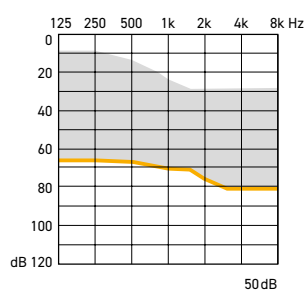

STERLING ITC








312

STERLING ITE



312

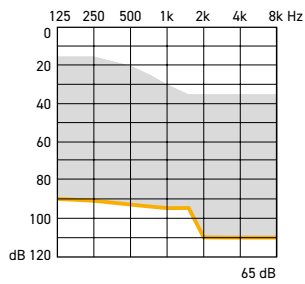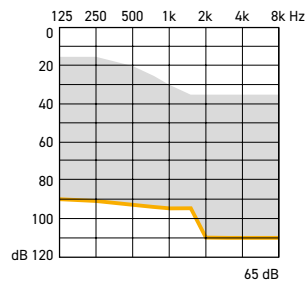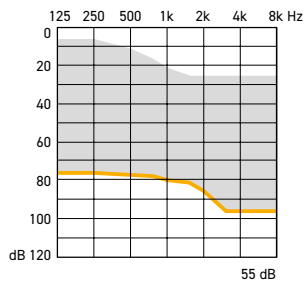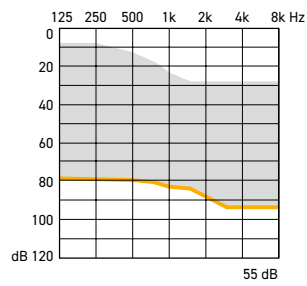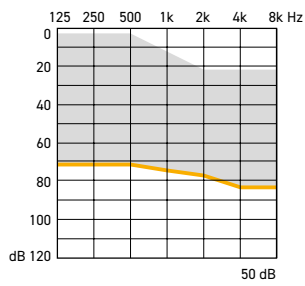

EMERALD XS



Performance Level 80 | 60 | 40 | 30 | 20



SMALL AND UNOBTRUSIVE

The Emerald XS hearing aids are equipped with many automatic functions which ensure outstanding hearing comfort. They adapt flexibly to each hearing situation. Perfect for first-time users demanding high discretion. Smart Key or Rexton App enables wearers to discreetly volume, programs and more.

Highlights

- › Battery size 10
- › Mini Receivers 2.0
- › Programming via:
- › Programming adaptor 10

Receiver	S	M	P
FOG/OSPL in dB	45/108	55/113	60/118
Battery current drain	0.9 mA	1.0 mA	0.9 mA


Colors

PL 80-40



Colors

PL 30-20

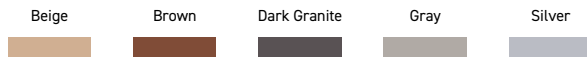

Features / Performance Level	80	60	40	30	20
Channels/Controls/Programs	40 / 20 / 6	32 / 16 / 6	24 / 12 / 6	16 / 8 / 4	16 / 8 / 4
Smart Automatic Equalizer	•	•	•	-	-
Wireless Sync	-	-	-	-	-
Directionality	Automatic / Adaptive iOmni Front & Back	Automatic / Adaptive iOmni Front & Back	Automatic / Adaptive iOmni	Automatic / Adaptive iOmni	Automatic iOmni
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management
Wind Noise Cancellation	Standard	Standard	Standard	Standard	-
Reverb Reducer	•	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •
Music Enhancer: Live/Recording/Acting	• / • / •	• / - / -	• / - / -	-	-
Tinnitus Function	Sound Therapy	Sound Therapy	Sound Therapy	Sound Therapy	-
Xphone	-	-	-	-	-
Acclimatization / Data Logging	• / •	• / •	• / •	• / •	• / •
Accessories & Apps					
Smart Connect	-	-	-	-	-
Smart Transmitter	-	-	-	-	-
Smart Remote	-	-	-	-	-
Smart Key	•	•	•	•	•
CROS RIC	-	-	-	-	-
Smart Remote App	•	•	•	•	•
Rexton App	•	•	•	•	•

EMERALD S



Performance Level 80 | 60 | 40 | 30 | 20 | 10



THE MULTITALENTED DEVICE FOR HIGHEST EXPECTATIONS

The versatile Emerald S is a small, inconspicuous hearing system. Various types of receiver mean that it can be used for a broad range of hearing loss. The Emerald S hearing aids in Performance Levels 80 to 40 can be used with either a standard battery OR rechargeable battery.

Highlights

- › Battery size 312
- › Mini Receivers 2.0
- › Rechargeable battery (80 / 60 /40)
- › Programming via: Programming adaptor 312
- › ConnexxAir (PL 80-20)

Receiver	S	M	P	HP
FOG/OSPL in dB	45/108	60/119	70/124	75/130
Battery current drain	0.9 mA	1.0 mA	1.0 mA	1.1 mA


Colors

PL 80-40



Colors

PL 30-20

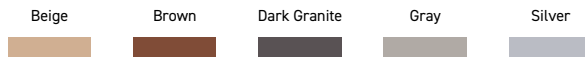

Features / Performance Level	80	60	40	30	20	10
Channels/Controls/Programs	40 / 20 / 6	32 / 16 / 6	24 / 12 / 6	16 / 8 / 4	16 / 8 / 4	12 / 6 / 4
Smart Automatic Equalizer	•	•	•	-	-	-
Wireless Sync	•	•	•	•	•	-
Directionality	Automatic / Adaptive iOmni Front & Back Left & Right Narrow - iLock	Automatic / Adaptive iOmni Front & Back Narrow - iLock	Automatic / Adaptive iOmni Narrow - iLock	Automatic / Adaptive iOmni	Automatic iOmni	Automatic iOmni
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management	Noise Management
Wind Noise Cancellation	Standard Binaural	Standard Binaural	Standard	Standard	-	-
Reverb Reducer	•	-	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •	- / -
Music Enhancer: Live/Recording/Acting	• / • / •	• / - / -	• / - / -	-	-	-
Tinnitus Function	Sound Therapy	Sound Therapy	Sound Therapy	Sound Therapy	-	-
Xphone	•	•	•	-	-	-
Acclimatization / Data Logging	• / •	• / •	• / •	• / •	• / •	- / •

Accessories & Apps						
Smart Connect	•	•	•	•	•	-
Smart Transmitter	•	•	•	•	•	-
Smart Remote	•	•	•	•	•	-
Smart Key	•	•	•	•	•	•
CROS RIC	•	•	•	-	-	-
Smart Remote App	•	•	•	•	•	•
Smart Connect App	•	•	•	•	•	-
Rexton App	•	•	•	•	•	•
Smart Power Charger	•	•	•	-	-	-

EMERALD M



Performance Level 80 | 60 | 40



COMPACT AND INTELLIGENT PREMIUM SOLUTIONS

The high-performance Emerald M is characterized by its versatility. The housing contains a highly effective hearing system which can alternatively be used with a standard or rechargeable battery. It offers smart connection options. Telephone coil ensures optimal understanding in locations equipped with loop system. The Emerald M hearing aids in Performance Levels 80 to 40 can be used with either a standard battery OR rechargeable battery.

Highlights

- › Battery size 13
- › Mini Receivers 2.0
- › Rechargeable battery
- › Programming via:
 - › Programming adaptor 13
 - › ConnexxAir

Receiver	S	M	P	HP
FOG/OSPL in dB	45/108	60/119	70/124	75/135
Battery current drain	0.9 mA	1.1 mA	1.0 mA	1.1 mA

Colors



Features / Performance Level	80	60	40
Channels/Controls/Programs	40 / 20 / 6	32 / 16 / 6	24 / 12 / 6
Smart Automatic Equalizer	•	•	•
Wireless Sync	•	•	•
Directionality	Automatic / Adaptive iOmni Front & Back Left & Right Narrow - iLock	Automatic / Adaptive iOmni Front & Back Narrow - iLock	Automatic / Adaptive iOmni Narrow
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Narrow - iLock
Wind Noise Cancellation	Standard Binaural	Standard Binaural	Standard
Reverb Reducer	•	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •
Music Enhancer: Live/Recording/Acting	• / • / •	• / - / -	• / - / -
Tinnitus Function	Sound Therapy	Sound Therapy	Sound Therapy
Xphone	•	•	•
Acclimatization / Data Logging	• / •	• / •	• / •

Accessories & Apps			
Smart Connect	•	•	•
Smart Transmitter	•	•	•
Smart Remote	•	•	•
Smart Key	•	•	•
CROS RIC	•	•	•
Smart Remote App	•	•	•
Smart Connect App	•	•	•
Rexton App	•	•	•
Smart Power Charger	•	•	•

STELLAR LI



Performance Level 80 | 60 | 40 | 30



POWER AND ENDURANCE FOR 24 HOURS

The Stellar Li has an integrated lithium-ion battery which is inductively charged in the charging station. The battery offers an operating time of 24 hours with full connectivity, with only 4 hours charging time. 30 minute fast-charging allows for 7 hours run time.

Highlights


- > Lithium-ion battery
- > Mini Receivers 2.0
- > Programming via:
 - > HiPro CS 44
 - > ConnexxAir

Receiver	S	M	P	HP
FOG/OSPL in dB	45/108	60/119	70/124	75/130
Battery current drain	0.9 mA	1.0 mA	1.0 mA	1.1 mA

Colors PL 80-40



Colors PL 30

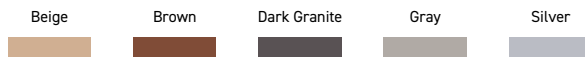

Features / Performance Level	80	60	40	30
Channels/Controls/Programs	40 / 20 / 6	32 / 16 / 6	24 / 12 / 6	16 / 8 / 4
Smart Automatic Equalizer	•	•	•	-
Wireless Sync	•	•	•	•
Directionality	Automatic / Adaptive iOmni Front & Back Left & Right Narrow - iLock	Automatic / Adaptive iOmni Front & Back Narrow - iLock	Automatic / Adaptive iOmni Narrow Narrow - iLock	Automatic / Adaptive iOmni
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing
Wind Noise Cancellation	Standard Binaural	Standard Binaural	Standard	Standard
Reverb Reducer	•	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •
Music Enhancer: Live/Recording/Acting	• / • / •	• / - / -	• / - / -	-
Tinnitus Function	Sound Therapy	Sound Therapy	Sound Therapy	Sound Therapy
Xphone	•	•	•	-
Acclimatization / Data Logging	• / •	• / •	• / •	• / •

Accessories & Apps				
Smart Connect	•	•	•	•
Smart Transmitter	•	•	•	•
Smart Remote	•	•	•	•
Smart Key	•	•	•	•
CROS RIC	•	•	•	-
Smart Remote App	•	•	•	•
Smart Connect App	•	•	•	•
Rexton App	•	•	•	•
Li-ion Charger	•	•	•	•

TRUCORE

TRUCORE RIC HEARING AIDS

EMERALD XS










EMERALD S



Battery size

10

312 / rechargeable battery

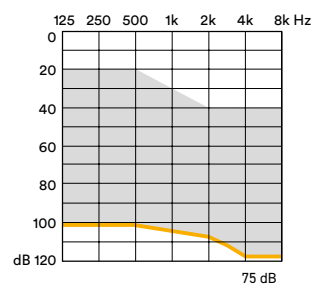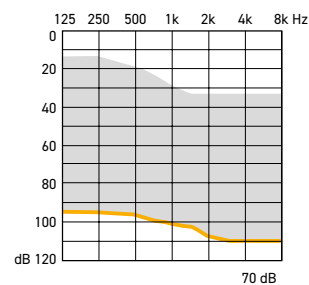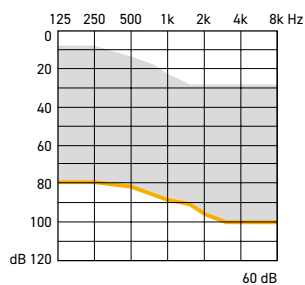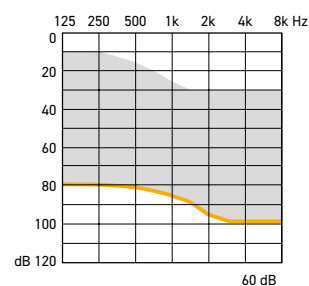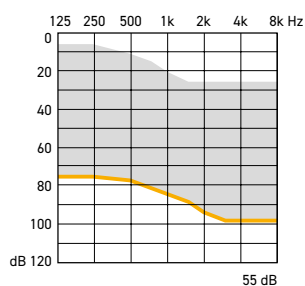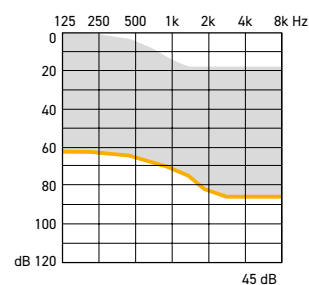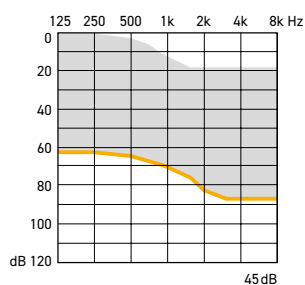

EMERALD M











STELLAR LI



13 / rechargeable battery

Rechargeable battery (Li-Ion)

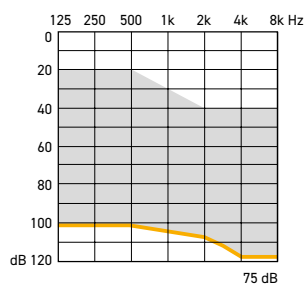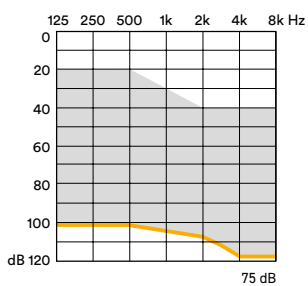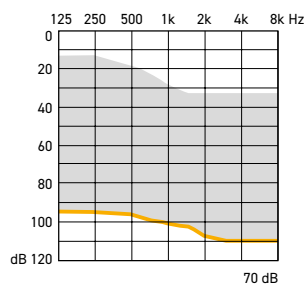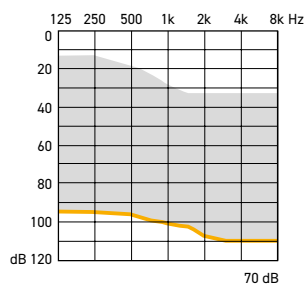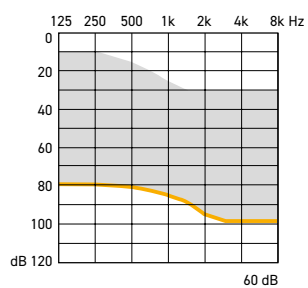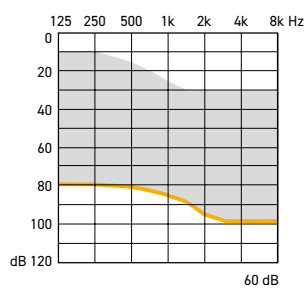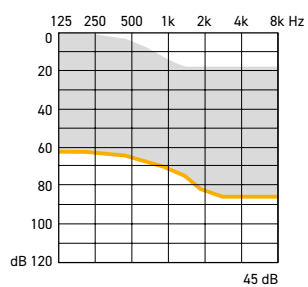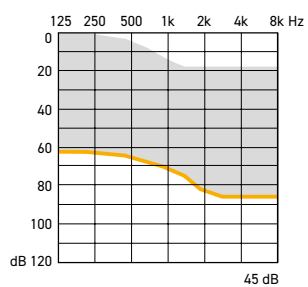

MOSAIC S



Performance Level 40 | 30 | 20 | 10



GRACEFUL AND VARIABLE

Mosaic S - the behind-the-ear hearing system for anyone who particularly values discretion. The Mosaic S is ideally suited for an open-fitting system. This hearing system is a perfect solution for mild to moderate hearing loss.

Highlights

› Battery size 312

› Programming via:

› ConnexxAir (PL 40-20)

› Programming adaptor 312

	Ear hook	ThinTube
FOG/OSPL in dB	55/124	45/124
Battery current drain	0.9 mA	0.9 mA

Colors



Features / Performance Level	40	30	20	10
Channels/Controls/Programs	24 / 12 / 6	16 / 8 / 4	16 / 8 / 4	12 / 6 / 4
Smart Automatic Equalizer	•	-	-	-
Wireless Sync	•	•	•	-
Directionality	Automatic / Adaptive iOmni Narrow - iLock	Automatic / Adaptive iOmni	Automatic iOmni	Automatic iOmni
Noise Reduction	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management	Noise Management
Wind Noise Cancellation	Standard	Standard	-	-
Reverb Reducer	-	-	-	-
Bandwidth: Extension / Compression	- / •	- / •	- / •	- / -
Music Enhancer: Live/Recording/Acting	• / - / -	-	-	-
Tinnitus Function	Sound Therapy	Sound Therapy	-	-
Xphone	-	-	-	-
Acclimatization / Data Logging	• / •	• / •	• / •	- / •
Accessories & Apps				
Smart Connect	•	•	•	-
Smart Transmitter	•	•	•	-
Smart Remote	•	•	•	-
Smart Key	•	•	•	•
Smart Remote App	•	•	•	•
Smart Connect App	•	•	•	-
Rexton App	•	•	•	•

MOSAIC M



Performance Level 80 | 60 | 40 | 30 | 20 | 10



UNIVERSAL APPLICATION

The Mosaic M hearing aids are suitable for various degrees of hearing loss and with their many features, they offer wide scope for meeting your customers' individual needs. Performance Level 80-40 can be used with standard or rechargeable batteries. For open and closed fitting.

Highlights


- > Battery size 13
- > Optional battery door with Audio Input
- > Rechargeable battery (80 / 60 / 40)
- > Programming via:
 - > ConnexxAir (PL 80-20)
 - > Programming adaptor 13

	Earhook damped	ThinTube
F0G/OSPL in dB	60/130	53/126 (PL10; 53/125)
Battery current drain	1.0 mA	1.1 mA (PL 10; 1,0 mA)

Colors PL 80-40



Colors PL 30-10

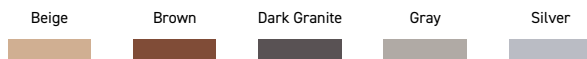

Features / Performance Level	80	60	40	30	20	10
Channels/Controls/Programs	40 / 20 / 6	32 / 16 / 6	24 / 12 / 6	16 / 8 / 4	16 / 8 / 4	12 / 6 / 4
Smart Automatic Equalizer	•	•	•	-	-	-
Wireless Sync	•	•	•	•	•	-
Directionality	Automatic / Adaptive iOmni Front & Back Left & Right Narrow - iLock	Automatic / Adaptive iOmni Front & Back Narrow - iLock	Automatic / Adaptive iOmni Narrow - iLock	Automatic / Adaptive iOmni	Automatic iOmni	Automatic iOmni
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management	Noise Management
Wind Noise Cancellation	Standard Binaural	Standard Binaural	Standard	Standard	-	-
Reverb Reducer	•	-	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •	- / -
Music Enhancer: Live/Recording/Acting	• / • / •	• / - / -	• / - / -	-	-	-
Tinnitus Function	Sound Therapy	Sound Therapy	Sound Therapy	Sound Therapy	-	-
Xphone	•	•	•	-	-	-
Acclimatization / Data Logging	• / •	• / •	• / •	• / •	• / •	- / •

Accessories & Apps						
Smart Connect	•	•	•	•	•	-
Smart Transmitter	•	•	•	•	•	-
Smart Remote	•	•	•	•	•	-
Smart Key	•	•	•	•	•	•
CROS RIC	•	•	•	-	-	-
Smart Remote App	•	•	•	•	•	•
Smart Connect App	•	•	•	•	•	-
Rexton App	•	•	•	•	•	•
Smart Power Charger	•	•	•	-	-	-

MOSAIC P



Performance Level 80 | 60 | 40 | 30 | 20 | 10



Highlights

- > Battery size 13
- > Optional battery door with Audio Input
- > Programming via:
 - > ConnexxAir (PL 80-20)
 - > Programming adaptor 13 (PL 80-20) and programming shoe (PL 10)

ADVANCED AND CONVINCING

Rexton's Mosaic P models are hearing aids for moderate to severe and severe hearing losses. The Mosaic P model with its rocker switch is very easy to use.

PL 80-20	Earhook damped	Earhook undamped	ThinTube
FOG/OSPL in dB	70/131	77/135	65/129
Battery current drain	1.0 mA	1.6 mA	1.2mA

PL 10	Earhook undamped
FOG/OSPL in dB	70/134
Battery current drain	1.4 mA


Colors

PL 80-40




Colors

PL 30-20

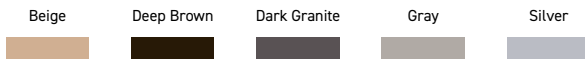

Colors

PL 10

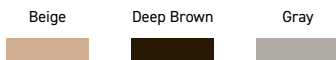

Features / Performance Level	80	60	40	30	20	10
Channels/Controls/Programs	40 / 20 / 6	32 / 16 / 6	24 / 12 / 6	16 / 8 / 4	16 / 8 / 4	12 / 6 / 4
Smart Automatic Equalizer	•	•	•	-	-	-
Wireless Sync	•	•	•	•	•	-
Directionality	Automatic / Adaptive iOmni Front & Back Left & Right Narrow - iLock	Automatic / Adaptive iOmni Front & Back Narrow - iLock	Automatic / Adaptive iOmni Narrow - iLock	Automatic / Adaptive iOmni	Automatic iOmni	Automatic iOmni
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management	Noise Management
Wind Noise Cancellation	Standard Binaural	Standard Binaural	Standard	Standard	-	-
Reverb Reducer	•	-	-	-	-	-
Bandwidth: Extension / Compression	• / •	- / •	- / •	- / •	- / •	- / -
Music Enhancer: Live/Recording/Acting	• / • / •	• / - / -	• / - / -	-	-	-
Tinnitus Function	Sound Therapy	Sound Therapy	Sound Therapy	Sound Therapy	-	-
Xphone	•	•	•	-	-	-
Acclimatization / Data Logging	• / •	• / •	• / •	• / •	• / •	- / •

Accessories & Apps						
Smart Connect	•	•	•	•	•	-
Smart Transmitter	•	•	•	•	•	-
Smart Remote	•	•	•	•	•	-
Smart Key	•	•	•	•	•	•
CROS RIC	•	•	•	-	-	-
Smart Remote App	•	•	•	•	•	•
Smart Connect App	•	•	•	•	•	-
Rexton App	•	•	•	•	•	•

MOSAIC HP



Performance Level 80 | 60 | 40 | 30 | 20 | 10



HIGH POWER FOR ADVANCED HEARING LOSS

Designed for users who need extra amplification. Mosaic HP offers a long lasting battery with low power consumption and the flexibility you need to treat up to profound hearing losses. Telephone-coil, audio streaming, and useful accessories provide the ultimate flexibility for the wearer, the Rexton App allows for discreet control.

Highlights


- › Battery size 675
- › Optional battery door with Audio Input
- › Child lock
- › Small ear hook
- › Programming via:
 - › ConnexxAir (PL 80-20)
 - › HiPro CS 44 (PL 80-20) and programming shoe (PL 10)

	Earhook undamped
FOG/OSPL in dB	82/140 (PL 10: 80/140)
Battery current drain	2.1 mA (PL10; 2.4 mA)


Colors PL 80-40



Colors PL 30-20

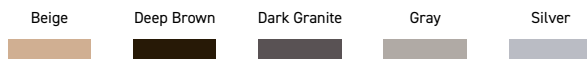

Colors PL 10

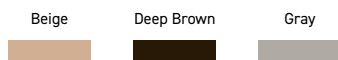

Features / Performance Level	80	60	40	30	20	10
Channels/Controls/Programs	24 / 16 / 6	24 / 16 / 6	24 / 12 / 6	16 / 8 / 4	16 / 8 / 4	12 / 6 / 4
Smart Automatic Equalizer	•	•	•	-	-	-
Wireless Sync	•	•	•	•	•	-
Directionality	Automatic / Adaptive iOmni Front & Back Left & Right Narrow - iLock	Automatic / Adaptive iOmni Front & Back Narrow - iLock	Automatic / Adaptive iOmni Narrow - iLock	Automatic / Adaptive iOmni	Automatic iOmni	Automatic iOmni
Noise Reduction	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing Directional	Noise Management SoundSmoothing	Noise Management SoundSmoothing	Noise Management	Noise Management
Wind Noise Cancellation	Standard Binaural	Standard Binaural	Standard	Standard	-	-
Reverb Reducer	•	-	-	-	-	-
Bandwidth: Extension / Compression	- / •	- / •	- / •	- / •	- / •	- / -
Music Enhancer: Live/Recording/Acting	• / • / •	• / - / -	• / - / -	-	-	-
Tinnitus Function	Sound Therapy	Sound Therapy	Sound Therapy	Sound Therapy	-	-
Xphone	•	•	•	-	-	-
Acclimatization / Data Logging	• / •	• / •	• / •	• / •	• / •	- / •

Accessories & Apps						
Smart Connect	•	•	•	•	•	-
Smart Transmitter	•	•	•	•	•	-
Smart Remote	•	•	•	•	•	-
Smart Key	•	•	•	•	•	•
CROS RIC	•	•	•	-	-	-
Smart Remote App	•	•	•	•	•	•
Smart Connect App	•	•	•	•	•	-
Rexton App	•	•	•	•	•	•

TRUCORE

TRUCORE BTE HEARING AIDS

MOSAIC S



MOSAIC M



MOSAIC P



MOSAIC HP














Battery size

312

13 / rechargeable battery

13

675

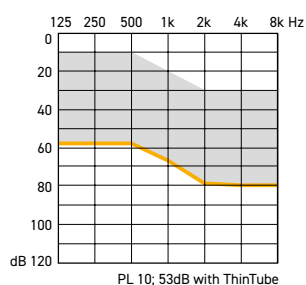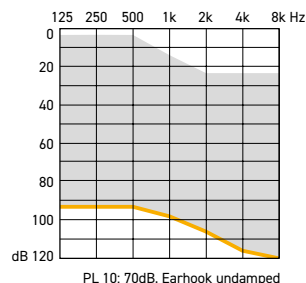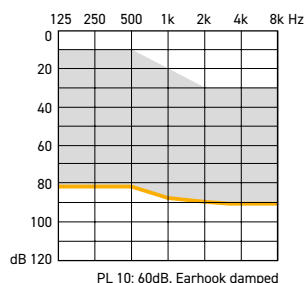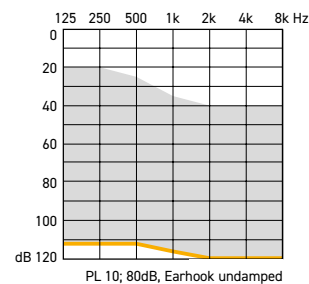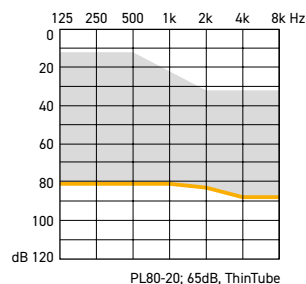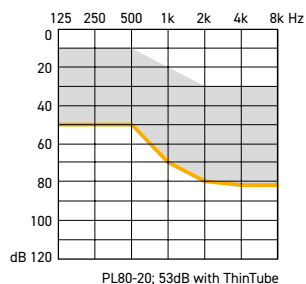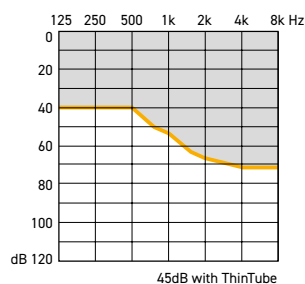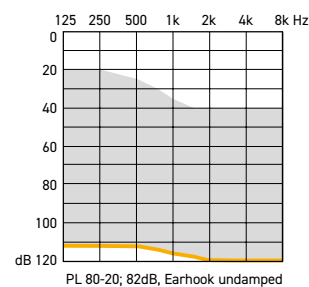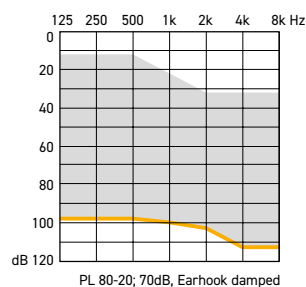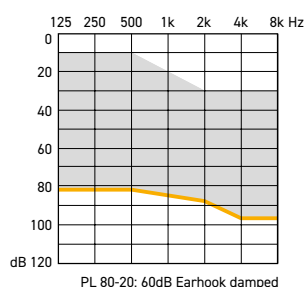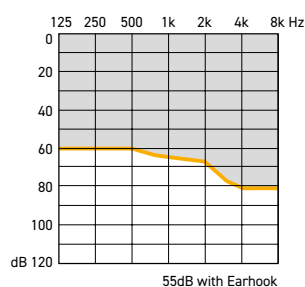





ACCESSORIES & APPS



THE COMPACT SMART MIC

**DIRECT AUDIO STREAMING FROM
PRACTICALLY ANY BLUETOOTH-ENABLED
ANDROID SMARTPHONE**



The new Smart Mic enables hands-free telephone calls without your customer needing to pick up his/her smartphone. In challenging hearing environments it can easily be used as an additional microphone.



SPECIAL FEATURES

- Direct streaming from Android smartphones
- Listen to music in stereo quality
- Can be used as a conference microphone

THE PRACTICAL SMART TRANSMITTER 2,4

**THE BLUETOOTH® TRANSMITTER FOR
TELEVISIONS AND STEREO SYSTEMS**



The Smart Transmitter 2,4 allows direct transfer of stereo sound from audio sources such as television sets and stereo systems to the hearing systems. Simply place the hearing systems on the transmitter and the pairing process begins. The transmitter can be paired with multiple hearing systems at the same time for shared enjoyment with friends.



SPECIAL FEATURES

- No separate remote control needed
- Connecting/pairing is particularly easy
- Several hearing system users can connect simultaneously
- Analog and digital audio inputs (Cinch and TOSLINK)

THE HANDY SMART KEY

SMALL AND PRACTICAL



With Smart Key, programs and volume can be easily adjusted. A keylock prevents buttons from being inadvertently pressed. A status LED indicates whether the remote control is enabled.



SPECIAL FEATURES

- Small and handy
- Control via high-frequency sounds

THE COMPREHENSIVE REXTON APP

ONE FOR ALL



The Rexton App allows your customers to control their hearing system conveniently. In MotionCore and MyCore hearing aids, telephone calls are forwarded directly from the iPhone and take priority over other audio signals; streamed music can be listened to in stereo, directly and without any additional device. With TruCore hearing aids and the Smart Connect you can stream music and telephone calls through this app.

The free Rexton App turns a smartphone into a discreet remote control for the hearing systems. With this app, the hearing systems can be customized and operated for the following settings: hearing program, volume and tone, without the need for any additional accessories.



Download
the Rexton App now!



SPECIAL FEATURES

- Hearing programs and volumes can be adjusted discreetly
- The volume can be set individually for the right and left hearing system
- The Rexton App creates an environment profile for its user. This renders optimization easier for you when it comes to fine-tuning and after sales care

THE FUNCTIONAL SLIM-RIC TRAVEL CHARGER

WITH QI CHARGING



Featuring an ultra-convenient Qi wireless charging system for the internal power bank and advanced lithium-ion technology, the sleek and compact design provides up to four days hearing on-the-go.

The fast charge feature provides up to five hours wear time in just 30 minutes, while the case can be fully recharged in just three hours.



SPECIAL FEATURES

- 4 days on the go (3 times recharging)
- The size is optimized to accommodate:
- Devices with Mini Receivers 2.0 length 0-3
- Mini Receivers 2.0 types S, M, and P

THE FAST SMART LI-ION POWER

FAST, INDUCTIVE CHARGING



Once the hearing systems are placed in the charging station, the charging process starts automatically. Charging takes place inductively, that is, without direct contact. A complete charge takes approximately 4 hours. Even after just 30 minutes, the hearing systems are ready for another 7 hours of use. The three LEDs display the charging status of the hearing systems.



SPECIAL FEATURES

- Drying function during charging
- Hearing systems automatically switch off when they are placed in the charging station and turn on when they are removed
- Automatic switch-off guarantees overcharge protection
- Standard power supply with micro-USB connector
- Compatible with standard power banks

THE COMFORTABLE CHARGING STATION



The new Charging Station: Not only easy to use for your customers, but also great for the environment. A full charge easily carries through the day, even with streaming music. And - if needed - a quick charge option ensures additional hours of functionality – with only thirty minutes charging time.



SPECIAL FEATURES

- New design with case cover
- Drying function during charging
- Automatic switch-off guarantees overcharge protection
- Inductive charging
- Standard power supply with micro-USB connector

THE NEW CHARGING STATION B-P

ROBUST AND FUNCTIONAL



The all new Charging Station for the P Li-Ion G6. A powerful hearing aid needs a powerful and long lasting charger. Its' galvanic charging system ensures a lossless charging process.

The new housing has a robust and scratch resistant design and is very easy to use. Just lay the hearing aids into the intended slots to charge it over night or make use of the 30 minute fast charging functionality.



SPECIAL FEATURES

- New robust and scratch resistant design
- 30-minute fast charge functionality
- Fixed USB charging cable
- Included power adapter

THE NEW CHARGING+ STATION B-P

ADVANCED PRACTICABILITY



The all new Charging+ Station B-P is the premium version compared to the standard charger with additional features for the new P Li-Ion G6.

It is equipped with an interated drying functionality, an integrated UV cleaning light and offers also a fast 30 minute fast charging functionality.

It's robust and scratch resitant finish is ideal for the daily use and for a long lasting operation.



SPECIAL FEATURES

- New robust and scratch resistant design
- 30-minute fast charge functionality
- Fixed USB charging cable
- Included power adapter
- Integrated drying functionality
- Integrated UV cleaning functionality

THE NEW CHARGING STATION B-HP

ROBUST AND FUNCTIONAL



The all new Charging Station for the HP Li-Ion G6. A highly powerful hearing aid needs a powerful and long lasting charger. Its' galvanic charging system ensures a lossless charging process.

The new housing has a robust and scratch resistant design and is very easy to use. Just lay the hearing aids into the intended slots to charge it over night or make use of the 30 minute fast charging functionality for a quick boost of extra wearing time if you're in a hurry.



SPECIAL FEATURES

- New robust and scratch resistant design
- 30-minute fast charge functionality
- Fixed USB charging cable
- Included power adapter

THE NEW CHARGING+ STATION B-HP

ADVANCED PRACTICABILITY



The all new Charging+ Station B-HP is the premium version compared to the standard charger with additional features for the new HP Li-Ion G6.

It is equipped with an integrated drying functionality, an integrated UV cleaning light and offers also a fast 30 minute fast charging functionality.

It's robust and scratch resistant finish is ideal for the daily use and for a long lasting operation.



SPECIAL FEATURES

- New robust and scratch resistant design
- 30-minute fast charge functionality
- Fixed USB charging cable
- Included power adapter
- Integrated drying functionality
- Integrated UV cleaning functionality



RELY ON REXTON

It's a tough world out there, and even tougher when your hearing starts to go. At Rexton, we won't let hearing loss hold you back. Good hearing is critical: From getting the job done, to getting home safely and being there for the people who count on you, we know what's at stake. That's why we work hard to deliver proven hearing technology in the most reliable way. We understand what reliability means in real life, and make practical, easy-to-use products you can count on. Our hearing aids have been getting the job done since 1955, so whatever life throws at you, you can rely on Rexton.

rexton.com

The Bluetooth® word mark and logos are owned by the Bluetooth SIG, Inc. and any use of such marks by Sivantos GmbH is under license. Other trademarks and trade names are those of their respective owners. Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Google Play is a trademark of Google Inc. "Made for iPhone" and "Made for iPad" mean that an electronic accessory has been designed to connect specifically to iPhone or iPad, respectively, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with iPhone or iPad may affect wireless performance. iPad, iPhone are trademarks of Apple Inc., registered in the U.S. and other countries.

Order No. 04538-99C1-7600 © 2019 Sivantos GmbH