


4.12.4 Fluoroscopy L.I.H. (Last Image Hold) Function

The L.I.H. function remains the fluoroscopy image on the acquisition monitor after releasing the fluoroscopy foot switch, and is set in the [Fluoroscopy] tab in the [Edit Procedure] window.

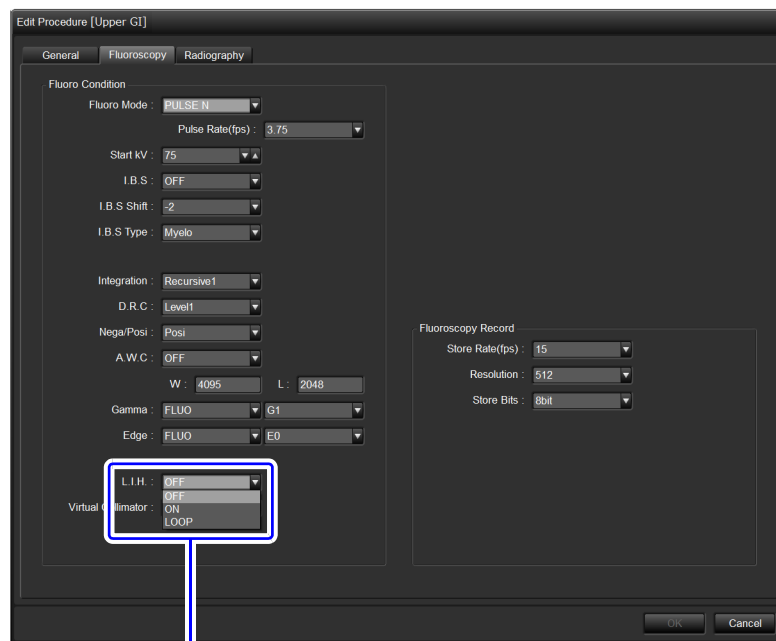

L.I.H. setting

The L.I.H. has three options below.

Option	Function
OFF	The image display area turns completely black on the acquisition monitor after releasing the fluoroscopy foot switch.
ON	The fluoroscopy image acquired last appears on the acquisition monitor after releasing the fluoroscopy foot switch.
LOOP	The fluoroscopy images acquired during fluoroscopy is replayed on the acquisition monitor after releasing the fluoroscopy foot switch.