



**LOGROHISPANIA**  
Material Médico-Sanitario y Ortopédico

Cedeira Road - 49ra - 49  
N- 552 KM/2  
36812 - CEDEIRA  
Pontevedra

BUDGET 986 45 41 200  
CENTRAL. 698 129 2811  
ORDERS 604 061 5855  
[logro@logrohispania.com](mailto:logro@logrohispania.com)  
[info@logrohispania.com](mailto:info@logrohispania.com)  
[www.logrovige.es](http://www.logrovige.es)



98002033

Adjustable 3-position cervical collar for pediatric use

Laerdal Stifneck PEDI-Select Children's Neck Brace

The Pedi-Select adjustable collar for children features:

- Pink Velcro strap for easy identification
- Improved dimensions to suit a wider range of pediatric patients.
- Stronger hooks for oxygen masks/nasal cannulas to secure the devices oxygen
- Integrated Velcro closure
- Improved design to suit the specific anatomical characteristics of children.
- Extra large tracheal opening

Product	Laerdal Stifneck Pedi-Select Children's Neck Brace
SKU	98002033
Manufacturer	Laerdal
Suitable for patient type	Child
Weight	1,000,000

The Stifneck Select pediatric orthopedic collar is very easy to use because it is adjustable and one-piece, allowing for quick and effective protection of a patient with a suspected cervical spine injury. It offers three adjustable sizes specifically designed for children. It is also marked with a pink stripe for easy identification.

in a rescue bag,

All you need to do to use the neck brace correctly is:

1. Measure the length of the patient's neck (instructions are printed on the back of the necklace).
2. Set the size
3. block
4. establish

Product features:

- The simple and practical adjustment of one of the 3 sizes allows the collar to be used effectively by all children.
- The specially designed die-cut latches securely hold the selected size,
- The size adjustment guides ensure neck symmetry,
- The instructions for selecting the victim's height using the "grandfather" technique (engraved on the back of the head) facilitate a good fit before restraining, limiting unnecessary head movement.
- The large openings both at the back and front of the collar allow better access to the patient (for example, to check the pulse or perform a cricothyroidotomy),
- the possibility of X-rays and MRIs,
- hypoallergenic foam,
- Contains no adhesives or glue - improved design to better withstand harsh outdoor conditions.