

medDream

PACS

**STORE, ARCHIVE,
MANAGE, VIEW
MEDICAL IMAGES**



VENDOR NEUTRAL
ARCHIVE



MODALITY
WORKLIST




CUSTOMISABLE
AUTOROUTING



HIS / RIS / EMR
INTEGRATION

 **Softneta**

Workflow with MedDream PACS

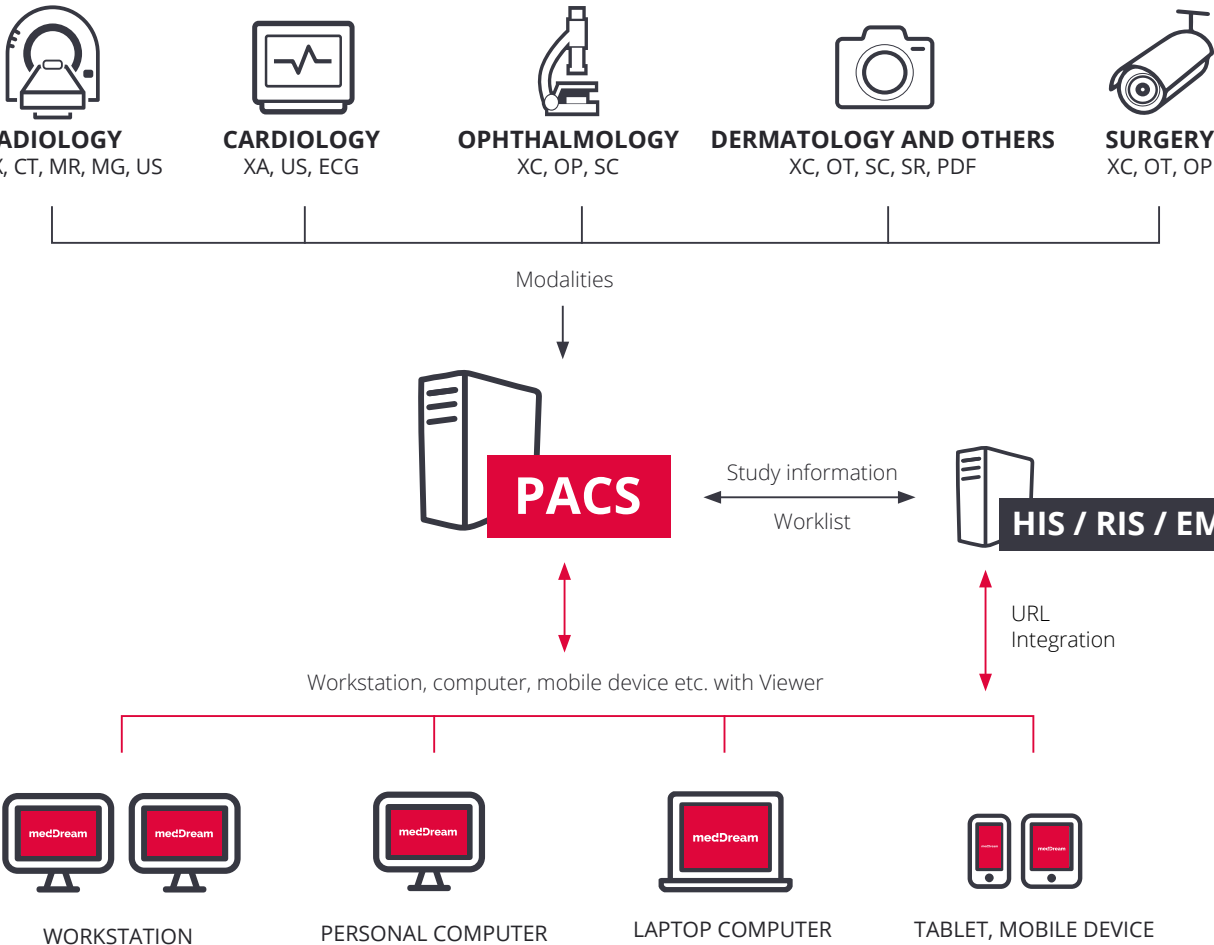

To	Id	Sd	Tw	Fy	Any	All	CX	CT	OK	MH	-	+Q	↓	↑	⌂
Patient ID	Name	Accessionnum	Modality	Description	Date Time	Received On	Source All								
A-1st000A	Anonymized	20160317145141	CT		2016-03-17 14:51:41	2016-03-17 14:23:06	MEDOREAM								
Irbby	Anonymized	20160318153812	CT		2016-03-18 15:38:12	2016-03-17 08:36:01	Upload default								
00-120	Anonymized	20160204182924	US		2016-02-04 18:29:24	2016-02-04 18:29:27	Upload root								
00-127	Anonymized	20160204142349	US		2016-02-04 14:23:49	2016-02-04 18:27:47	Upload root								
qewt	Anonymized	20160204101334	US		2016-02-04 10:13:34	2016-03-01 15:50:00	SENDTOPACS								
16-4353	Anonymized	20160201093054	OT		2016-02-01 09:30:54	2016-02-04 11:43:28	Upload root								
123456	Anonymized	20160108155452	OT		2016-01-08 15:54:52	2016-03-01 16:16:22	SENDTOPACS								
123456	Anonymized	20160108085029	OT		2016-01-08 08:50:29	2016-03-01 16:18:13	SENDTOPACS								
A-MassiveTest	Anonymized	50MO	US	SendTopACS DEMO	2015-12-09 13:42:48	2016-02-10 11:57:12	MEDOREAM								
A-CRIS-180613	Anonymized	247_250	CT	KUNAS	2015-12-08 11:19:48	2016-01-26 11:51:51	MEDOREAM								
A-HLIS-6648	Anonymized	182-185	CT	KUNAS	2015-12-07 07:31:53	2016-01-29 11:38:48	MEDOREAM								
15-11528	Anonymized	20160201124348	OT		2015-09-23 11:24:53	2015-06-17 18:12:49	SENDTOPACS								
B	Anonymized	20161127125648A	DI		2016-11-27 12:56:44	2016-02-04 11:44:02	Upload-root								
[14-198[SENDTOPACS-0411181225]	Anonymized	198	MR	GALVA+C	2014-06-19 14:32:38	2015-06-17 18:12:34	SENDTOPACS								
A-[14-16	Anonymized		MR	GALVAXAPLAS	2014-08-28 10:55:23	2015-06-17 18:11:54	MEDOREAM								
45B1118044	Anonymized		US		2014-02-12 11:38:37	2016-03-01 14:55:23	Upload-default								
AKI15430521	Anonymized		MR	ABDOMEN+PELVIS	2014-01-29 10:09:31	2015-06-17 18:12:04	SENDTOPACS								
AAH16020312	Anonymized		MR	HEAD+ROUTINE	2014-04-28 08:52:18	2015-06-17 18:13:20	SENDTOPACS								
A-14-2P5-BY	Anonymized	977_978	CT	STUBURAS 215	2014-01-27 07:52:11	2015-06-17 18:12:42	MEDOREAM								
A-45403079645 - 4392	Anonymized		CT	Spine/Spine Routine (Sub)	2013-10-21 19:11:20	2015-06-17 18:13:27	MEDOREAM								
1 to 20 of 89								Previous	1	2	3	4	5	Next	

Copyright 2016 © Sukuma MedStream 4.07.3 Build(2016)

MedDream PACS + WEB DICOM Viewer features

Store, archive, manage medical images:

- C-STORE, C-FIND, C-GET and C-MOVE support;
- Implicit Little Endian, Explicit Little Endian;
- JPEG, JPEG-LS, JPEG 2000, RLE, MPEG-2;
- MPEG-4 Transfer Syntaxes supported;
- DICOM ECG file support;
- DICOM MPEG-2 and MPEG-4 Video playback support;
- Study forwarding.

View medical images:

- Regular tools: zoom, pan, windowing, magnifier, etc.;
- Cine playback of multi-frame sequences;
- Wide variety of measurements: distance, angle, density (HU), etc.;
- Reference lines (Scout Lines);
- Comfortable bar of series preview with thumbnails.

Integration:

- Integration to HIS / RIS / EMR. Flexible and open integration interface;
- Multi-language support.



Optional modules of MedDream WEB DICOM Viewer



Report Module

Writing, storing and viewing reports, templates





ECG Module

Viewing and measuring



Video Module

MPEG-2, MPEG-4 viewing

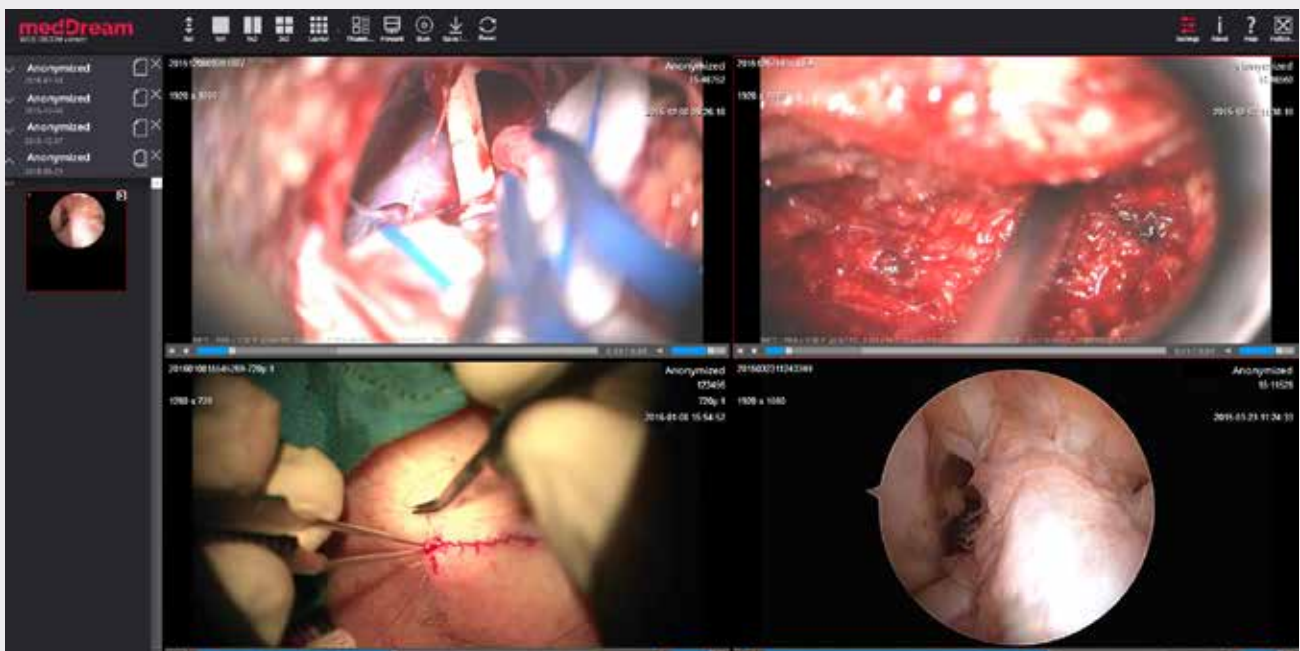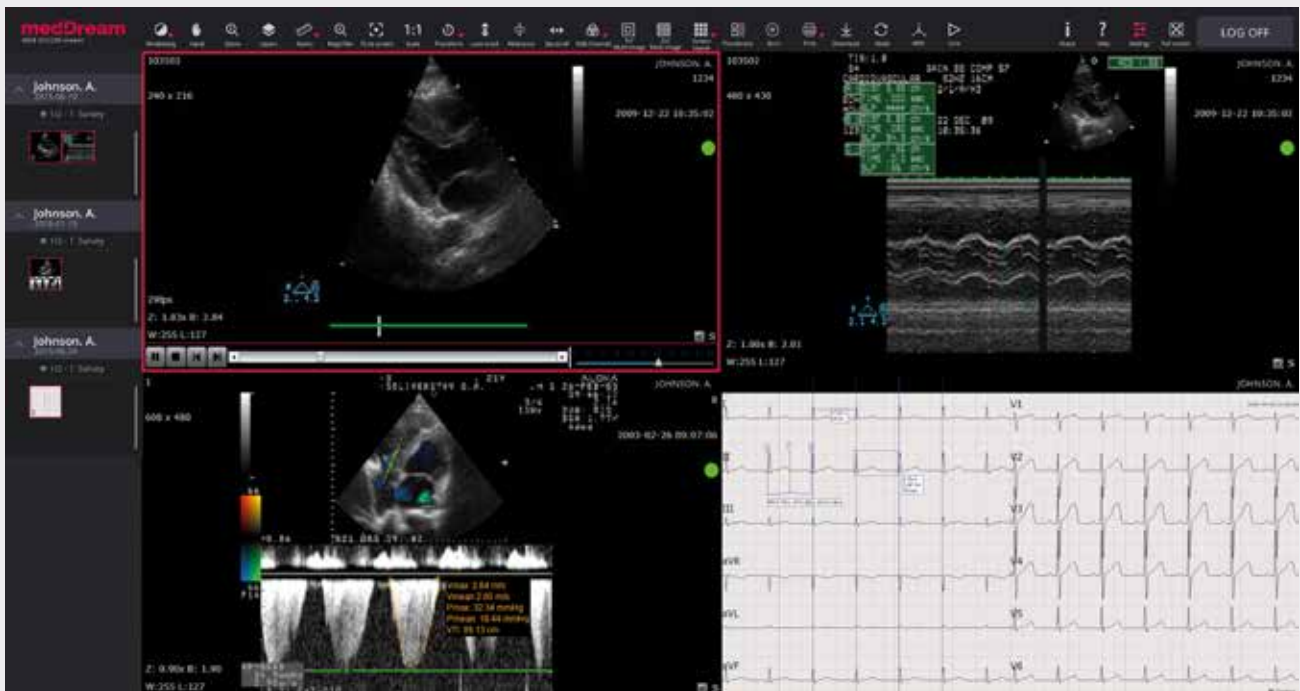

Softneta
Medical imaging

E-mail: info@softneta.com
Phone: +370 672 42852
Web: www.softneta.com

Address: K. Barsausko str. 59,
LT-51423, Kaunas,
Lithuania

