Rheumatoid Factor Calibrator

Package size

5 levels x0.5 mL

Intended use

Rheumatoid Factor Calibrator is used for calibration of quantitative determination of rheumatoid factor (RF) analyte on Mindray BS measurement system.

Summary

The analyte of RF Calibrator is rheumatoid factor. By calibrating the selected analytes, Mindray BS-series chemistry analyzers can generate valid calibration factors, and then calculate the concentrations of each sample automatically after analysis. Mindray BS measurement system is composed of Mindray BS-series chemistry analyzers, Mindray reagent kits, calibrators and control materials.

Components

RF Calibrator is a liquid calibrator based on rheumatoid factor solution. The concentrations of the calibrator components are lot-specific.

Warnings and Precautions

1. For in vitro diagnostic use only.
2. Professional or trained person is required. Please take the necessary precautions for handling all laboratory reagents. Do not swallow and avoid contact with skin and mucous membranes.
3. When the calibrator accidentally enters the eyes and mouth, or contact with the skin, immediately wash with plenty of water. If necessary, visit the doctor for further medical treatment.
4. Avoid direct exposure to sunlight and freezing. The results can't be assured when stored at inappropriate condition.
5. The calibrator values are lot-specific with the matched models listed in the value sheet.
6. Calibration is recommended after reagent lot change or execution of specific maintenance or troubleshooting procedure.
7. Please perform scheduled maintenance and standard operation including calibration and analysis to assure the performance of measurement system.
8. The calibrator was tested with Elisa methods and shown to be negative for antibodies to HIV and HCV and nonreactive for HBsAg. However, as no testing method can rule out the potential risk of infection with absolute certainty, so this material should be handled as a patient specimen¹.

9. Material safety data sheet is available on request.

10. Disposal of all waste material should be in accordance with local guidelines.

Preparation

1. Take the vial out of the refrigerator and allow reaching room temperature.
2. According to the calibration procedure described in the instrument manual, use the specific value listed in the value sheet to set calibration parameter and execute calibration.

Storage and Stability

The calibrator is stable up to expiration date indicated on the label when stored in unopened vial at 2~8°C and protected from light. Once opened, it is stable for 30 days when capped tightly and avoiding microbial contamination at 2~8°C.

Materials required but not provided

1. Mindray BS Chemistry analyzers and reagent kits.
2. General laboratory equipment.

Traceability of Calibrator

The calibrator values assigned by Mindray standard transfer procedure and routine method are listed in the value sheet. The traceability process is based on ISO 17511², the analyte (measurand) in this calibrator is traceable to Manufacturer's selected measurement procedure.

Quality control

Use Mindray control as sample to monitor the status of calibration. The control result should be within the defined range as shown in the control value sheet. If the control result falls outside the range, the measurement system should be checked. For instance, position of reagent or sample placed on the analyzer; expiration date or storage condition of the calibrator, reagent and control; the parameter setting in software; or performance of the analyzer.

References

1. Biosafety in Microbiological and Biomedical Laboratories; U.S. Department of Health and Human Services; US Government Printing Office; Washington: 2007.
2. ISO 17511:2003. In vitro diagnostic medical devices—Measurement of quantities in biological samples—Metrological traceability of values assigned to calibrators and control materials.

Graphical symbols

In Vitro Diagnostic
medical device

Batch Code

European
Conformity

Authorized representative in
the European Community

Use By

Consult
Instructions for use

Biological
Risks

Temperature
Limit

Manufacturer

Catalogue
number

© 2024 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. All rights reserved

Manufacturer: Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

Address: Mindray building, Keji 12th Road South, High-tech Industrial Park, Nanshan, Shenzhen, 518057 P.R.China

E-mail Address: service@mindray.com

Website: www.mindray.com

Tel: +86-755-81888998

Fax: +86-755-26582680

EC-Representative: Shanghai International Holding Corp.
GmbH(Europe)

Address: Eiffestraße 80, Hamburg 20537, Germany

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