

Table 10-2 Customer Care & Maintenance Task Schedule

Service at Indicated Time	Daily	Weekly	Monthly	Annually	Per Facility's QA Program	Notes
Clean Probes	●*					* or before each use
Clean Probe Holders	●					
Clean Air Filters		●				More frequently if necessary, depending on the environment
Inspect AC Mains Cable			●			Mobile Ultrasound system: Check Weekly
Inspect Cables and Connectors			●			
Clean Console			●			
Clean Monitor and Touch Panel			●			
Inspect Wheels, Casters, Brakes and Swivel Locks			●			Mobile Unit: Check Daily
Check Operator Panel Movement			●			Mobile Unit: Check Daily
Console Current Leakage Checks					●	Also after corrective maintenance.
Peripheral Current Leakage Checks					●	Also after corrective maintenance.
Surface Probe Current Leakage Checks					●	Also after corrective maintenance.
Endocavity Probe Current Leakage Checks					●	Also after corrective maintenance.
Transesophageal (TEE) Probe Current Leakage Checks					●	Also after corrective maintenance.
Surgical Probe Current Leakage Checks					●	Also after corrective maintenance.
Functional Checks					●	Also after corrective maintenance.
System Battery				*		It is recommended to replace the System Battery once every two years.