PANTONE 654 C

ENG desam® effekt +

Disinfection and cleaning of medical device surfaces

Liquid concentrate for disinfection of areas and surfaces in healthcare and communal hygiene

Instructions for use: The working solution is prepared by diluting the desam® effekt + concentrate in water at a temperature of 20°C – 25°C. Thoroughly rinse disinfected surfaces and objects which come into contact with food. Select the spectrum of efficacy according to the purpose of use. Please read the instructions on the label and SDS before usage. Do not use on zinc, colour-unstable materials and silicon. It is not intended for the disinfection of invasive medical devices. Observe the exposure time.

Composition: active substances in 100g concentrate of desam® effekt +: Alkyl (C12-16) dimethylbenzyl ammonium chloride 19 g (CAS: 68424-85-1), 2-Phenoxyethanol 10 g (CAS: 122-99-6), N-(3-aminopropyl)-N-dodecylpropan-1,3-diamine 7,2 g (CAS: 2372-82-9), Didecyltrimethylammonium-chloride 3 g (CAS: 7173-51-5). Other substances: non-ionic surfactants, perfume

Efficiency spectrum: bactericidal, virucidal (BVDV/Vaccinia/Rota/Noro/Adeno), mycobactericidal, tuberculocidal, fungicidal.

Attention: For professional use only. Hazard notes and safety advice: **Danger H302** Harmful if swallowed. **H314** Causes severe skin burns and eye damage. **H410** Very toxic to aquatic life with long lasting effects. **P273** Avoid release to the environment. **P280** Wear protective rubber gloves/protective clothing/eye protection/ face protection. **P301+P330+P331** IF SWALLOWED: IF SWALLOWED: rinse mouth. Do NOT induce vomiting. **P303+P361+P353** IF ON SKIN (or hair): Immediately take off all contaminated clothing. Rinse skin with water [or shower]. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P310** Immediately call a TOXICOLOGICAL INFORMATION CENTER / doctor

First aid: In case of inhalation, remove the source of exposure, provide the victim with fresh air, prevent physical exertion (including walking), or seek medical attention. Report any serious adverse events that occur in connection with the product in question to the manufacturer and the appropriate local authority. Choose the efficiency spectrum according to the purpose of use. Read label before use.

Storage and safe disposal of the containers: Store in the original closed containers in dry and clean, ventilated rooms. Do not store near the source of heat and in direct sun light. Storage temperature is in the range from -10 °C to +25 °C. **Disposal:** Residual amount of the product to be applied in accordance with the directions for use, bigger amount to be transmitted to authorised companies for liquidation. Contaminated packaging hand over to specialised company as a dangerous waste. **Durability:** Use before date on the packaging.

Storage and safe disposal of the preparation and packaging: Store in the original closed packaging in dry and clean areas. Do not store near heat sources and in direct sunlight. **Storage temperature:** -10 °C to +25 °C. **Disposal:** Apply residual quantities of the product in accordance with the instructions for use, hand over larger quantities for disposal to authorized companies. Contaminated packaging must be disposed of as hazardous waste. **Use before:** the date stated on the package.

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