

- Press the [ID] key on the alphanumeric keyboard or the custom switch.
 - The ID screen closes and the examination starts using the information displayed in the input area.
- NOTE: If you start an examination using a patient ID of an already examined patient, the following message appears: "This order is already performed. Do you want to perform an additional examination?" Select Yes to perform a new examination using the same study ID.
- NOTE: Selecting [ID] during an examination displays the ID screen, and the information can be edited. However, the patient ID and study ID cannot be edited.

4.3 Switching probes and applications

Use the steps below to switch probes and applications to be used for an examination.

For details about how to connect probes, see "Connecting a probe" in this manual.

For details about how to register applications to probes, see the separate manual "Basic Operations".

NOTE: Operation procedures vary depending on the [Probe << Preset Link] setting on the touch panel. Check the setting first.

NOTE: To switch to an application that is not displayed on the touch panel, select [All Preset]. All registered applications are displayed.

Procedure

- By selecting an application, you can switch both an application and probe at the same time.
 - a. Press the [Probe/Preset] key.
 - b. Set [Probe << Preset Link] to On.
 - c. Select an application from the application preset switches (c) on the touch panel.
 - The probe will also switch at the same time.



- By selecting a probe, you can switch an application.
 - a. Press the [Probe/Preset] key.
 - b. Set [Probe << Preset Link] to Off.
 - c. Select a probe from the probe switches (c) on the touch panel.
 - d. If necessary, select an application from the application preset switches (d).