


An innovative Rapid Test Designed by WANTAI

WHO Prequalified

# Rapid Test for Antibody to (HIV 1+2) Human Immunodeficiency Virus

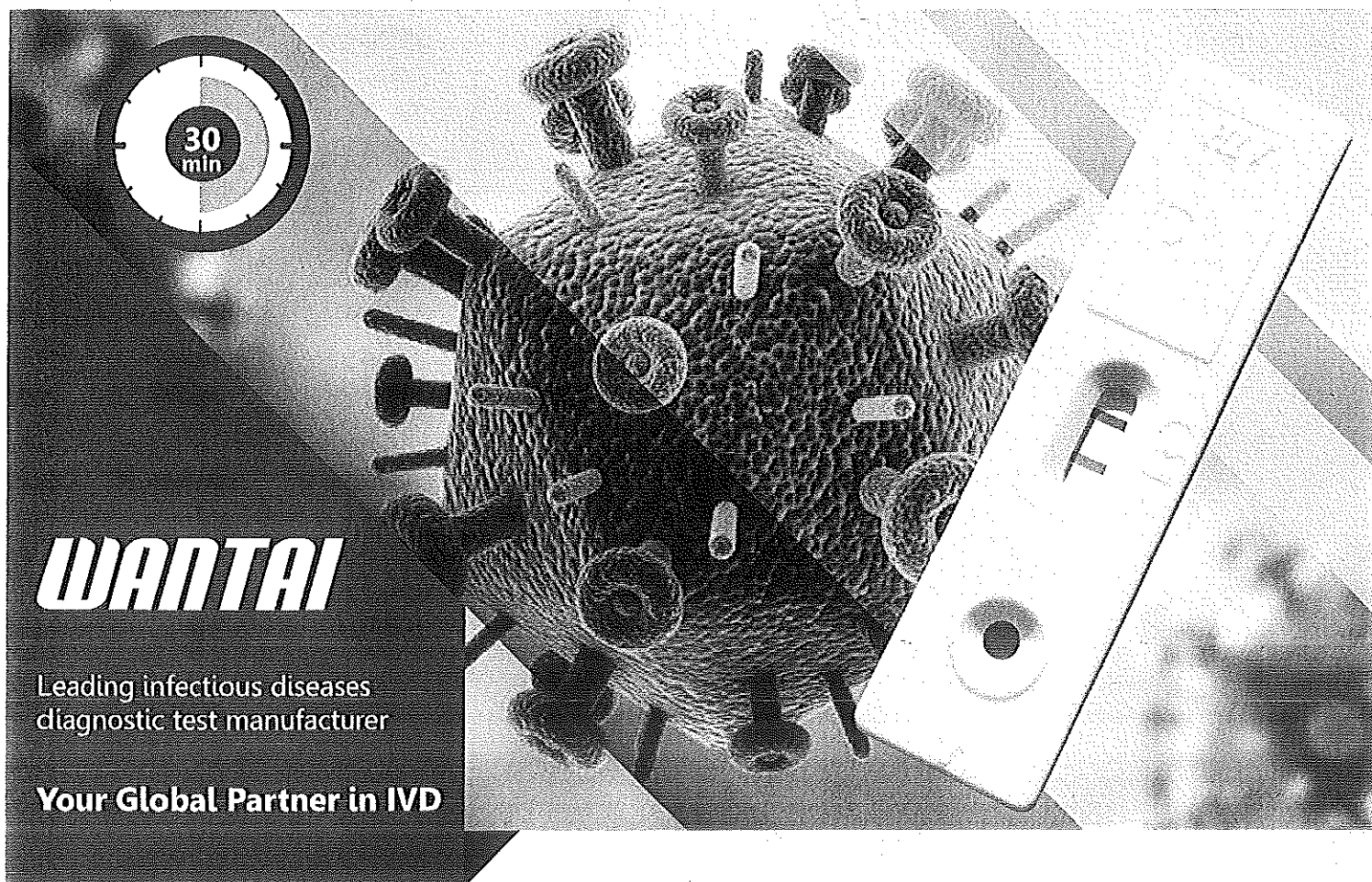

**WANTAI**

Leading infectious diseases  
diagnostic test manufacturer

Your Global Partner in IVD

*Developing Scientifically  
Focusing on the Health*



**Innovative**



**High Performance**



**Easy-to-use**




**Rapid**

## Test Process

### 1. Prepare Test


Allow the test cassette to reach room temperature, use it within 20 minutes after opening.

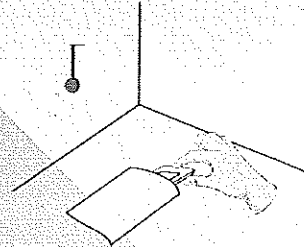


### 2. Add Specimen

1) For whole blood specimens  
Add one drop of blood, then  
Add one drop of Diluent Buffer


2) For the serum/ plasma specimens  
Add two drops of serum/ plasma  
specimens

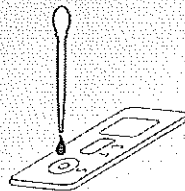

50µl of specimens

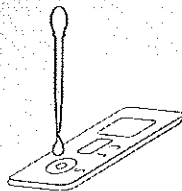


50µl of diluent buffer

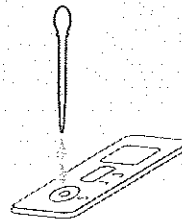

80µl specimens of S/P

### 3. Read Result


Read the results within 10-30 minutes.

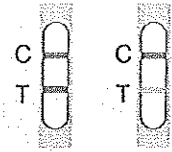

**Positive**



**Negative**

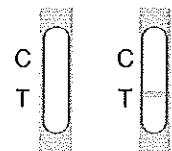

**Invalid**

\* Please refer to product insert for detailed instruction.

## Advanced Diagnostic Technology

- 01 **Lateral Flow Technology:** makes it possible to perform rapid one-site screening
- 02 **Superior Performance:** 100% sensitivity and >99% specificity. Wantai reacts positive to HIV-1 subtypes A, B, C, D, F, G, H, J, K, O, and HIV-2. The test can detect all HIV subtypes in 1st Int.panel for HIV [NIBSC].
- 03 **Easy-to-use:** No special equipment. Works with whole blood, serum and plasma samples
- 04 **Fast:** clear answer within 10-30 minutes
- 05 **All in one:** all accessories in the kit allows to do near-patient testing
- 06 **Room Temperature Storage:** 18 months shelf-life

## Order Information

	WANTAI 10	WANTAI 50	WANTAI 100	WANTAI 500
Test Cassette	x 10	x 50	x 10	x 50
Diluent Buffer	x 1 vial	V3 vials	x 1 vial	x 3 vials
Safety Lancet			x 10	x 50
Disposable Pipette			x 10	x 50
Alcohol Pad			x 10	x 50