

Box: 10 vials / 1 vial / 500 ml

Intended use

The selective supplement for the isolation of Clostridium difficile from clinical samples.

Vial for 500 ml of medium base

D-Cycloserine 125,0 mg Cefoxitin..... 4,0 mg

Directions

Aseptically reconstitute 1 vial with 5 ml of sterile distilled water. Mix gently until complete dissolution and add aseptically to 460 ml of Clostridium difficile LAB-AGAR™ Base (ref. PS 176), autoclaved, cooled to 50°C and with 5-7% defibrinated sterile blood added. Mix well and pour into Petri dishes.

Precautions:

- ★ For Laboratory use only
- ★ The supplement should be used only by adequately trained personnel with knowledge of microbiological techniques in the laboratory.
- ★ Consult the material safety data sheet before the use.
- ★ Do not use beyond stated expiry date

Storage / Shelf life

- ★ Store at 2-8°C - When stored as directed the supplement remains stable until the expiry date shown on the label
- ★ The expiration date is indicated on the label

