



Product Specification of:

Medical Oxygen Sensor

Type: M-15M

Part Number: 410823

Nuova ID: E-15/M

RoHS compliant and SVHC free

Meets the applicable requirements of ISO 80601-2-55

Comes with CE marking. further regulatory registrations available upon request

Produced under EN ISO 13485 Quality Management System

DOCUMENT PURPOSE

The purpose of this document is to present the performance specification and key features of the sensor.

This document should be used in conjunction with the Operating Manual of the instrument and the Product Safety Data Sheet of the sensor.

KEY FEATURES

Long-life galvanic gas sensor with high signal stability at low cross sensitivity to anesthesia gases combined with superior linearity over the entire measurement range.









High Signal Stability

Superior Linearity

Wide Measurement Range

Long Lifetime

This data sheet is subject to change without prior notice

 $Specification_M\text{-}15M_Rev_2$

Page 1 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany

Phone: +49 (0)3841 220 050 .: Fax: +49 (0)3841 220052 2 .: E-Mail: info@itg-wismar.de www.itg-wismar.de



Product Specification of: **Medical Oxygen Senso**r

Type: M-15M

Part Number: 410823

Nuova-ID: E-15/M



at 100 Vol.%O₂ applied for 5 min

TECHNICAL SPECIFICATIONS

| | IJRF | |
|--|------|--|
| | | |

| Operating Principle: | Partial pressure electrochemical cell |
|----------------------|---------------------------------------|
| | |

Measurement Range: 0 ... 100 Vol.%O₂

Initial Output Signal: 8 ... 12 mV at dry ambient air

Output Signal Range: 5 ... 15 mV

Response Time t90: < 12 s

Signal Drift (long-term): $\leq \pm 1 \text{ Vol.}\%O_2/\text{month}$ at ambient air

Linearity Error: $\leq \pm 3\%$

Zero Signal Offset: $\leq 0.3 \text{ Vol.}\%O_2$ at 100 Vol. $\%N_2$ applied for 5 min

Repeatability Error: $\leq \pm 1 \text{ Vol.}\%O_2$ at 100 Vol. $\%O_2$ applied for 5 min

Influence of Gas Humidity: - 0.03 % of signal per %rH

Signale Temperature Compensation NTC on sensor PCB

Cross-Sensitivity: according DIN EN ISO 80601-2-55

ELECTRICAL

Electrical Connector 3.50 mm Mono Jack

Recommended Load Resistor: ≥ 10 kOhm

MECHANICAL

Weight: ≤ 28 g

Material in Contact with Media: PA, PPS, PTFE, Stainless Steel NBR

Gas Connector: fits for M16x1 DIN 13 or 5/8-24 UNEF

ENVIRONMENTAL

Operating Temperature Range: 10 ... 40 °C

Ambient Pressure Range: 700 ... 1250 hPa

Ambient Humidity Range: up to 100 %rH non-condensing

LIFETIME

Expected Operating Life: 3 years at ambient air, depending on application

Nominal Sensor Life: 750 000 Vol.%h O₂ at ambient air, depending on application

Important Note: All characteristics are based on conditions at 25 °C, 50 %rH, 1013 hPa and a gas flow of > 2.5 L_s/min. For sensor performance data under other conditions, contact ITG.

This data sheet is subject to change without prior notice

Specification_M-15M_Rev_2

Page 2 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany

Phone: +49 (0)3841 220 050 .: Fax: +49 (0)3841 220052 2 .: E-Mail: info@itg-wismar.de www.itg-wismar.de



Product Specification of: **Medical Oxygen Senso**r

Type: M-15M

Part Number: 410823

Nuova-ID: E-15/M



STORAGE CONDITIONS IN UNOPENED ORIGINAL PACKAGE

Ambient Temperature Range: 15 ... 25 °C recommended

 $-20 \dots 50 \,^{\circ}$ C maximum (≤ 10 h)

Ambient Pressure Range: 700 ... 1250 hPa

Ambient Humidity Range: 50 ... 100 %rH recommended non-condensing

0 ... 30 %rH maximum one week

RELATED PRODUCTS

| Product | Part Number | Other Specifics | |
|-----------------------------|-------------|---------------------|--|
| O ₂ Sensor M-15 | 41 00 23 | _ | |
| O ₂ Sensor M-15T | 47 00 28 | translucent housing | |
| O ₂ Sensor M-15M | 41 08 23 | _ | |

Cleaning and Disinfection

The sensor outer housing can be cleaned with a dry wipe. Do not wipe the sensor's gas entrance side. Do not use any chemical disinfectant or sanitizer on the sensor.

Poisoning

ITG sensors are designed to operate in a wide range of environments. For optimal sensor lifetime and performance it is important that exposure to high concentrations of solvent vapors is avoided during storage, installation into instruments and operation of the sensor. Do not use adhesives directly on or near the sensor as the solvents may cause stress corrosion on the plastic parts.

Intended Use

The electrochemical oxygen sensors for use in medical technology are used as accessories with a limited shelf life exclusively in conjunction with other medical products (ventilators, incubators, anesthesia machines, portable oxygen monitors, oxygen therapy devices and gas mixing units) so that they can fulfill the manufacturer's intended purpose.

The oxygen sensors measure the oxygen partial pressure in gas mixtures.

Commissioning and use is carried out exclusively by expert clinic or service personnel. Direct contact with these items occurs during installation or maintenance by trained personnel.

Stabilisation Time

When installing a new sensor refer to the instrument manual for stabilization time before calibration.

If not specified otherwise wait at least 15 minutes to ensure the sensor has stabilized in the instrument.

Calibration Interval

ITG sensors are designed to have minimal signal drift over their functional lifetime. For optimal performance and maximum measurement accuracy however they should be calibrated before each use.

This data sheet is subject to change without prior notice

Specification_M-15_Rev_2

Page 3 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany

Phone: +49 (0)3841 220 050 .: Fax: +49 (0)3841 220052 2 .: E-Mail: info@itg-wismar.de



Product Specification of: Medical Oxygen Sensor

Type: M-15M

Part Number: 410823

Nuova-ID: E-15/M



If the Sensor is dropped

If a sensor is dropped, please check for visible mechanical damage of the sensor or if the sensor is leaking electrolyte. If this is the case, do not try to install the sensor into the device. Take safety precautions and immediately dispose the cell.

If the sensor shows no visible damage it should be placed in quarantine for at least 24 hours. Afterwards a follow-up check made by a two point calibration should be done.

Installation in Device

A gas tight sealing is ensured when the sensor is screwed in hand-tight. Don't use any mechanical tools to install the sensor.

Using excessive force may damage the sensor.

Optimal mounting position of the sensor is when the gas sensing area faces downwards. A horizontal position is acceptable. It is not recommended to use the sensor with the gas sensing area facing upward.

Connection should be made via recommended electrical and mechanical connectors only. The specified load resistance must be taken into account. Soldering the sensor will damage it and void the warranty. Please contact ITG for further information.

Since temperature has an influence on the output signal it is not advisable to place any heat or vibration sources (i.e. electric- pumps or valves, coolers, etc.) in close proximity to the cell.

Avoid proximity of the cell to any EMC radiating units as those emit frequencies which might interfere with the sensor's electrical connection and connecting cable.

RFI/EMI Susceptibility

ITG sensors contain metal parts and might be susceptible to RFI or EMI. Before use in MRI environments please contact ITG for further information.

Disposal

At the end of the sensors lifetime the sensor should not be disposed of in normal public waste as it may contain hazardous materials and caustic electrolyte (for more information refer to the PSDS). Please contact your local authorities for environmental legislation to relevant local waste disposal.

This data sheet is subject to change without prior notice

Specification_M-15M_Rev_2

Page 4 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany

Phone: +49 (0)3841 220 050 .: Fax: +49 (0)3841 220052 2 .: E-Mail: info@itg-wismar.de



Product Specification of: **Medical Oxygen Sensor**

Type: M-15M


Part Number: 410823


Nuova-ID: E-15/M



TECHNICAL DRWAING

| Sensor | ITG Part number | Color | | | Electrical pin assignment | |
|--------|--------------------|-------|---------|---------------|---------------------------|-----------|
| | | Сар | Housing | Threaded ring | Pin 1 | Pin 2 |
| M-15M | 410823 | white | white | white | plus (+) | minus (-) |

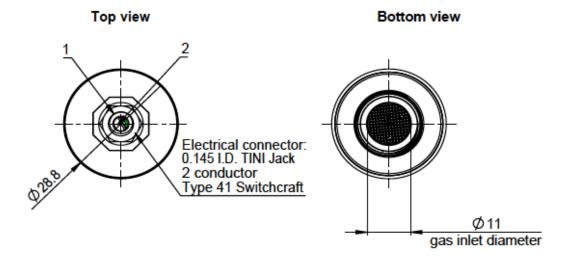

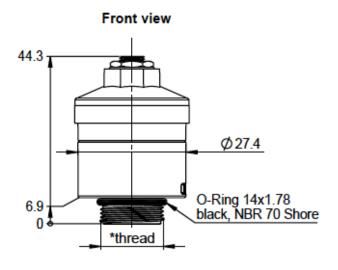

^{*} thread is compatible with M16x1 and 5/8-24 UNEF

Dimension unit: mm Dimension tolerances: linear $\pm 0.5~\text{mm}$, diameter $\pm 0.3~\text{mm}$

This data sheet is subject to change without prior notice

Specification_M-15M_Rev_2

Page 5 of 5

IT Dr. Gambert GmbH .: Hinter dem Chor 21 .: 23966 Wismar .: Germany

Phone: +49 (0)3841 220 050 .: Fax: +49 (0)3841 220052 2 .: E-Mail: info@itg-wismar.de www.itg-wismar.de