EtOH AQ 0,5 g/L – In vitro diagnosticum

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Aqueous Ethanol Standards

Application

This aqueous ethanol standard is intended for the calibration of analytical techniques used to determine the concentration of ethanol.

User guide

This ACQ Science EtOH AQ standard requires no additional preparation and is ready to use. This material should be used as defined by an individual laboratory's operating procedures and used for instrument calibration or as a control material

Assigned Value

The assigned ethanol concentration was determined by 3 independent laboratories, each accredited to DIN EN 17025. Measurements were carried out daily, in duplicate, on 5 separate days using two independent analytical methods (Gas Chromatography and Enzymatic determination (ADH)). The material is traceable to primary ethanol standards (ERM¹ ethanol standard in aqueous solution).

Storage and stability

Storage: 2 ° to 8 ° C

Shelf Life:

- Unopened vial, stored in the dark: see expiration date on the package.
- Stored tightly capped in the dark: see expiration date on package.

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Confidence Range

The confidence interval indicates the range in which the target value is located with a significance level of 95%.

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ACQ Science GmbH