

# The DrillCut-X® II-35 Handpiece with 35k Sinus Burrs

Power meets precision



# UNIDRIVE® SELECT with DrillCut-X® II-35

## Power meets precision

The DrillCut-X® II-35 handpiece, in conjunction with the 35k sinus burrs, is an innovative expansion of the KARL STORZ product portfolio. Furthermore, it is specially optimized to accommodate high speeds.

Together with the UNIDRIVE® SELECT motor control unit, the system provides a wealth of functions: The DrillCut-X® II-35 handpiece reaches a speed of up to 35,000 rotations per minute. In addition to the drill function, the DrillCut-X® II-35 also features a shaver function which can be used to remove polyps for example. Various shaver or sinus burr attachments enable adaptation to individual patient anatomies and pathologies. The versatility of the motor unit makes it possible to perform operations on the ear or larynx as well as paranasal and skull base surgery.



## The DrillCut-X® II-35 Shaver Handpiece



### Safety/hygiene



- Quick coupling mechanism
- Machine washable

### Convenience/Ergonomy

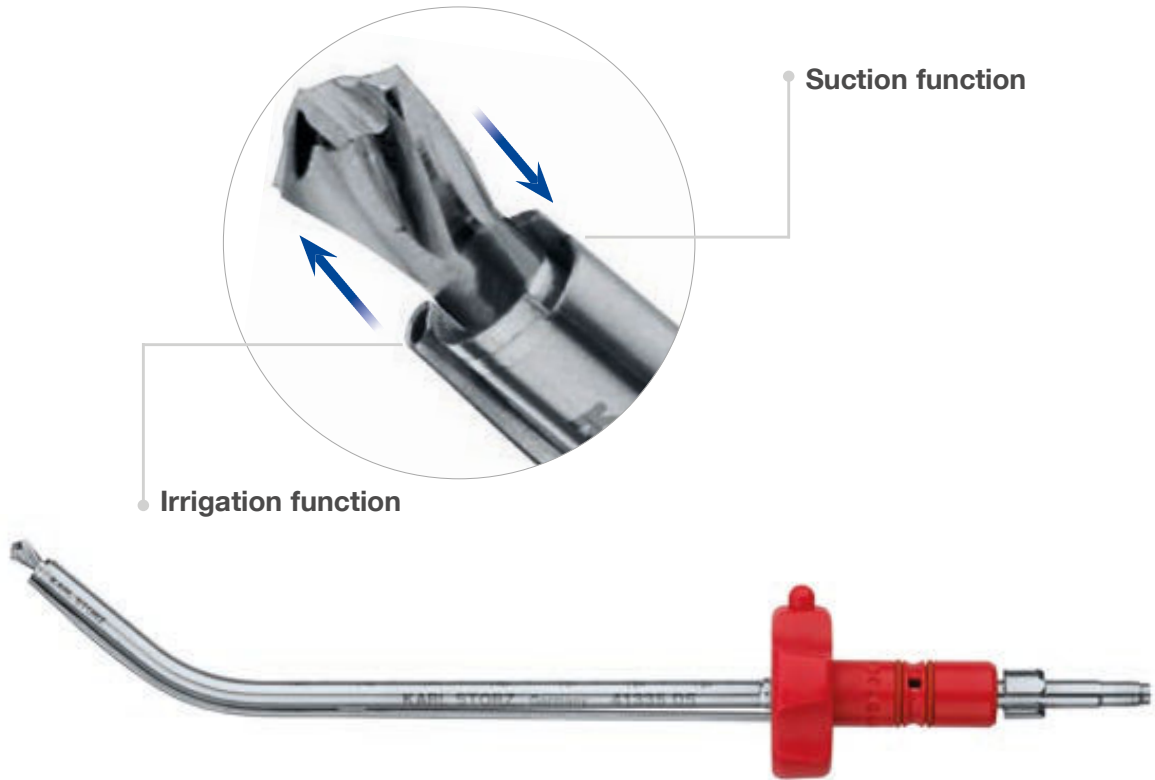








- Ergonomic thanks to variable handle

### Functionality



- Straight suction channel and integrated irrigation



Detail	Sinus Burr 35k, with integrated irrigation, length 12 cm, sterile, for single use, package of 5, color code: red <span style="float: right;">STERILE </span>	
	41335W	curved 40°, cylindrical, burr diameter 3 mm, shaft diameter 4 mm
	41335RN	curved 15°, bud drill, burr diameter 4 mm, shaft diameter 4 mm
	41335DW	curved 40°, diamond head, burr diameter 5 mm, shaft diameter 4 mm
	41335DT	curved 70°, diamond head, burr diameter 3.6 mm, shaft diameter 4 mm
	41335DS	curved 40°, diamond-shaped cutting, burr diameter 4 mm, shaft diameter 4 mm

## The DrillCut-X® II-35 Shaver Handpiece



40712035

**DrillCut-X® II-35 Shaver Handpiece**,  
for use with UNIDRIVE® S III ENT/NEURO and  
UNIDRIVE® SELECT



40712090

**Handle**, adjustable, for use with all DrillCut-X® II  
shaver handpieces



41200RA

**Cleaning Adaptor**, LUER-Lock,  
for cleaning the inner and outer blades of  
DrillCut-X® accessories  
consisting of:  
41200RA1 **Cleaning Adaptor**, for inner blades  
41200RA2 **Cleaning Adaptor**, for outer blades



41250RA

**Cleaning Adaptor**, LUER-Lock, for cleaning  
DrillCut-X® shaver handpieces



39550A

**Wire Tray**, provides safe storage of accessories  
for KARL STORZ paranasal sinus shaver systems  
during cleaning and sterilization  
For storage of:

- Up to 7 shaver attachment
- Paranasal sinus shaver handpiece
- Connecting cable



280053

**Universal Spray**, 6x 500 ml bottles,  
- HAZARDOUS GOODS - UN 1950  
consisting of:  
280053B **Universal Spray**, 500 ml  
280053C **Spray Nozzle**

## UNIDRIVE® SELECT



UM600

**UNIDRIVE® SELECT**, motor system with color display, touch screen, two motor outputs, power supply 100-240 VAC, 50/60 Hz

UM601

**ENT Software**, license, allows the use of shaver handpieces, micro ear burrs, high-speed and INTRA motors, for use with UNIDRIVE® SELECT UM600



UP220

**ENDOMAT® SELECT**, suction or irrigation pump, incl. mains cord, power supply 100-240 VAC, 50/60 Hz, with integrated unit communication via KS HIVE®

UP611

**ENT/NEURO/SPINE Software**, license, allows irrigation for the UNIDRIVE® SELECT UM600 motor system, for use with ENDOMAT® SELECT UP220



UF201

**Two-Pedal Footswitch**, one-stage

UF201W

**Two-Pedal Footswitch**, one-stage, wireless, including:  
**Footswitch**  
**Receiver**  
**Power Supply**

ET92-1414421

**Spare Part Kit**, carrying handle (2-pedal)



031531-10

**Tubing Set**, irrigation, UNIDRIVE®, sterile, for single use, package of 10, for use with KARL STORZ ENDOMAT® SELECT UP220

## Shaver Blades

Benefits of shaver blades

- Available either as a sterile single-use product or as a reusable version
- Straight working attachments, freely rotatable 360°

Various versions available:



## DrillCut-X® Handpieces

Cleaning adaptors for all DrillCut-X® handpieces and reusable shaver blades



41250RA **Cleaning Adaptor**, LUER-Lock, for cleaning DrillCut-X® II/DrillCut-X® II-35 handpieces












41200RA **Cleaning Adaptor**, LUER-Lock, for cleaning the inner and outer blades of DrillCut-X® II/DrillCut-X® II-35 accessories

## Shaver Blades, straight for paranasal sinus and skull base surgery

For use with DrillCut-X® II-35 handpiece



### Shaver Blades, straight, sterilizable

Detail	for use with	Shaver Blade, with integrated irrigation for DrillCut-X® handpiece, length 12 cm
	40712035 DrillCut-X® II-35 Handpiece	
	41201KN	serrated cutting edge, rectangular cutting window, diameter 4 mm, color code: blue-red
	41201KK	double serrated cutting edge, rectangular cutting window, diameter 4 mm, color code: blue-yellow
	41201GN	concave cutting edge, oval cutting window, diameter 4 mm, color code: blue-green
	41201LN	concave cutting edge, oblique cutting window, diameter 4 mm, color code: blue-black
	41201SN	straight cutting edge, diameter 4 mm, rectangular cutting window, color code: blue-blue
	41201KSA	serrated cutting edge, rectangular cutting window, diameter 3 mm, color code: blue-red
	41201KKSA	double serrated cutting edge, rectangular cutting window, diameter 3 mm, color code: blue-yellow
	41201KKSB	double serrated cutting edge, rectangular cutting window, diameter 2 mm, color code: blue-yellow
	41201LSA	concave cutting edge, oblique cutting window, diameter 3 mm, color code: blue-black



## Shaver Blades, curved for paranasal sinus and skull base surgery

For use with DrillCut-X® II-35 handpiece



### Shaver Blades, curved 35°/40°, sterilizable

Detail	for use with	Shaver Blade, with integrated irrigation for DrillCut-X® handpiece, length 12 cm
	40712035 DrillCut-X® II-35 Handpiece	
	41202KN	curved 35°, cutting edge serrated backwards, rectangular cutting window, diameter 4 mm, color code: blue-red
	41204KKF	curved 40°, cutting edge serrated forwards, double serrated, rectangular cutting window, diameter 4 mm, color code: blue-yellow
	41204KKB	curved 40°, cutting edge serrated forwards, double serrated, rectangular cutting window, diameter 4 mm, color code: blue-yellow
	41204KKFA	curved 40°, cutting edge serrated forwards, double serrated, rectangular cutting window, diameter 3 mm, color code: blue-yellow
	41204KKBA	curved 40°, cutting edge serrated backwards, double serrated, rectangular cutting window, diameter 3 mm, color code: blue-yellow

## Shaver Blades, curved for paranasal sinus and skull base surgery

For use with DrillCut-X® II-35 handpiece



### Shaver Blades, curved 65°, sterilizable

Detail	for use with	Shaver Blade, with integrated irrigation for DrillCut-X® handpiece, length 12 cm
	40712035 DrillCut-X® II-35 Handpiece	
	41203KNF	curved 65°, cutting edge serrated forwards, rectangular cutting window, diameter 4 mm, color code: blue-red
	41203KNB	curved 65°, cutting edge serrated backwards, rectangular cutting window, diameter 4 mm, color code: blue-red
	41203KKF	curved 65°, cutting edge serrated forwards, double serrated, rectangular cutting window, diameter 4 mm, color code: blue-yellow
	41203KKB	curved 65°, cutting edge serrated backwards, double serrated, rectangular cutting window, diameter 4 mm, color code: blue-yellow
	41203KKFA	curved 65°, cutting edge serrated forwards, double serrated, rectangular cutting window, diameter 3 mm, color code: blue-yellow
	41203KKBA	curved 65°, cutting edge serrated backwards, double serrated, rectangular cutting window, diameter 3 mm, color code: blue-yellow
	41203GNF	curved 65°, cutting edge concave forwards, oval cutting window, diameter 4 mm, color code: blue-green
	41203GNB	curved 65°, cutting edge concave backwards, diameter 4 mm, color code: blue-green

## Shaver Blades, straight for paranasal sinus and skull base surgery









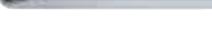
For use with DrillCut-X® II-35 handpiece



41301KK

### Shaver Blades, straight, sterile, for single use, package of 5

STERILE 

Detail	for use with	Shaver Blade, with integrated irrigation for DrillCut-X® handpiece, length 12 cm
	40712035 DrillCut-X® II-35 Handpiece	
	41301KN	serrated cutting edge, rectangular cutting window, diameter 4 mm, color code: blue-red
	41301KK	double serrated cutting edge, rectangular cutting window, diameter 4 mm, color code: blue-yellow
	41301GN	concave cutting edge, oval cutting window, diameter 4 mm, color code: blue-green
	41301LN	concave cutting edge, oblique cutting window, diameter 4 mm, color code: blue-black
	41301SN	straight cutting edge, rectangular cutting window, diameter 4 mm, color code: blue-blue
	41301KSA	serrated cutting edge, rectangular cutting window, diameter 3 mm, color code: blue-red
	41301KKSA	double serrated cutting edge, rectangular cutting window, diameter 3 mm, color code: blue-yellow
	41301KKSB	double serrated cutting edge, rectangular cutting window, diameter 2 mm, color code: blue-yellow
	41301LSA	concave cutting edge, oblique cutting window, diameter 3 mm, color code: blue-black

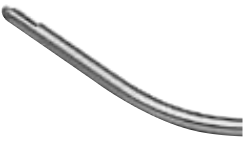


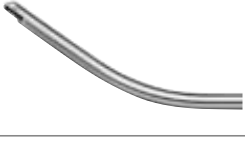

## Shaver Blades, curved for paranasal sinus and skull base surgery

For use with DrillCut-X® II-35 handpiece



Shaver Blades, curved 35°/40°, sterile, for single use,  
package of 5

STERILE 

Detail	for use with	Shaver Blade, with integrated irrigation for DrillCut-X® handpiece, length 12 cm
	40712035 DrillCut-X® II-35 Handpiece	
	41302KN	curved 35°, cutting edge serrated backwards, rectangular cutting window, diameter 4 mm, color code: blue-red
	41304KKF	curved 40°, cutting edge serrated forwards, double serrated, rectangular cutting window, diameter 4 mm, color code: blue-yellow
	41304KKB	curved 40°, cutting edge serrated backwards, double serrated, rectangular cutting window, diameter 4 mm, color code: blue-yellow
	41304KKFA	curved 40°, cutting edge serrated forwards, double serrated, rectangular cutting window, diameter 3 mm, color code: blue-yellow
	41304KKBA	curved 40°, cutting edge serrated backwards, double serrated, rectangular cutting window, diameter 3 mm, color code: blue-yellow

## Shaver Blades, curved for paranasal sinus and skull base surgery

For use with DrillCut-X® II-35 handpiece



Shaver Blades, curved 65°, sterile, for single use,  
package of 5

STERILE

Detail	for use with	Shaver Blade, with integrated irrigation for DrillCut-X® handpiece, length 12 cm
	40712035 DrillCut-X® II-35 Handpiece	
	41303KNF	curved 65°, cutting edge serrated forwards, rectangular cutting window, diameter 4 mm, color code: blue-red
	41303KNB	curved 65°, cutting edge serrated backwards, rectangular cutting window, diameter 4 mm, color code: blue-red
	41303KKF	curved 65°, cutting edge serrated forwards, double serrated, rectangular cutting window, diameter 4 mm, color code: blue-yellow
	41303KKB	curved 65°, double serrated cutting edge, rectangular cutting window, diameter 4 mm, color code: blue-yellow
	41303KKFA	curved 65°, cutting edge serrated forwards, double serrated, rectangular cutting window, diameter 3 mm, color code: blue-yellow
	41303KKBA	curved 65°, cutting edge serrated backwards, double serrated, rectangular cutting window, diameter 3 mm, color code: blue-yellow
	41303GNF	curved 65°, cutting edge concave forwards, oval cutting window, diameter 4 mm, color code: blue-green
	41303GNB	curved 65°, cutting edge concave backwards, oval cutting window, diameter 4 mm, color code: blue-green

## Larynx Shaver Blade, distally and proximally curved for laryngeal surgery

For use with DrillCut-X® II-35 handpiece



Larynx Shaver Blades, distally curved 18°, proximally curved 30°,  
sterile, for single use, package of 5



Detail	for use with	Larynx Shaver Blade, with integrated irrigation, length 27 cm
	40712035 DrillCut-X® II-35 Handpiece	
	41301LX	distally curved 18°, proximally curved 30°, oval cutting edge, oval cutting window, diameter 4 mm, color code: blue-green
	41301LXA	distally curved 18°, proximally curved 30°, oval cutting edge, oval cutting window, diameter 3 mm, color code: blue-green



KARL STORZ SE & Co. KG  
Dr.-Karl-Storz-Straße 34, 78532 Tuttlingen/Germany  
Postbox 230, 78503 Tuttlingen/Germany  
Phone: +49 7461 708-0  
Fax: +49 7461 708-105  
E-Mail: [info@karlstorz.com](mailto:info@karlstorz.com)  
[www.karlstorz.com](http://www.karlstorz.com)