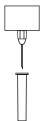
100  
buc./pics.

STERILE EO 0344

**EN** Single use.  
Sterile, non toxic, latex free.  
Use immediately after opening.  
**DONT REUSE.**

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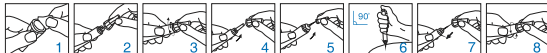
EN

## How to use pen needles:

- patients should remember to rotate injections sites, according to applicable insulin delivery recommendations;

- pen needles are sterile and for single use only. Sterility is guaranteed if the needle seal is unbroken.

## REMEMBER!

PEN NEEDLES SHOULD BE USED ONLY ONCE AND DISPOSED OF IN A SHARPS CONTAINER.

## Re-use can:

- cause lipohypertrophy;
- dull or bend up the tip, causing bleeding, bruising or scarring;
- increase the risk of needle breaking off in the skin;
- increase the risk of injection because the needle is no longer sterile.

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EN

Single use.  
Use immediately after opening.  
Sterile, non toxic, latex free.  
**DONT REUSE.**

STERILE EO 0344

100  
buc./pics.

## Characteristics:

- needles for use with the pen for subcutaneous insulin injection;
- fine three-facet grinding and siliconizing of needle minimizes tissue trauma;
- extra thin wall allows for a larger inner diameter and an optimized insulin flow and reduce injection time;
- integrated plunger backstop;
- latex, PVC and DEHP free;
- EO sterilization.

## Insulin needles are compatible with the following pen models for insulin administration:

**Novo Nordisk:** NovoPen® 3, NovoPen® 4, NovoPen® 5, NovoPen® Junior, NovoRapid® FlexPen®;

**Lilly:** HumaPen® luxura, HumaPen® luxura Hd;

**Owen Mumford:** Autopen® 24, Autopen® classic 2;

**Sanofi:** Aventis Lantus, Optipen® Pro, TactiPen®;

**BD:** Puregon pen®

STERILE EO 0344

4 mm  
32G 0,23mm100  
buc./pics.