

Keewell Medical Technology Co., LTD

Assembly Instruction

Model: KWT10

PLEASE SAVE THESE INSTRUCTIONS FOR FUTURE USE



Keewell Medical Technology Co., Ltd


Xiananyi Industrial Zone, Pingzhou, Nanhai,
Foshan, Guangdong 528251, P.R.China

T +86 757 86260100

marketing@keewell.cn / en.keewell.cn


WARNING: Important! Read and understand these instructions before assembling or using the IV Pole. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product.

Accessories of infusion stand

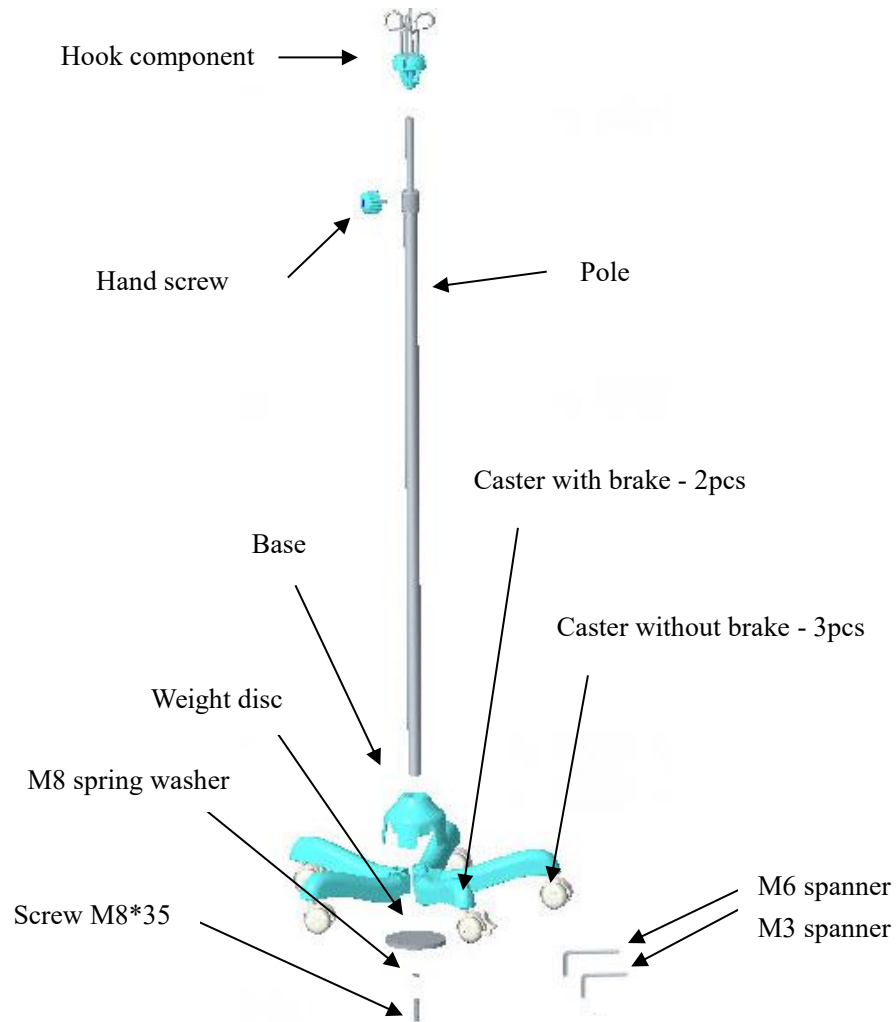


Installation procedure:


Step 1: Put on the white gloves provided in package, install the casters on the corresponding hole of the base one by one. The 2 casters with brake need to be intersected.

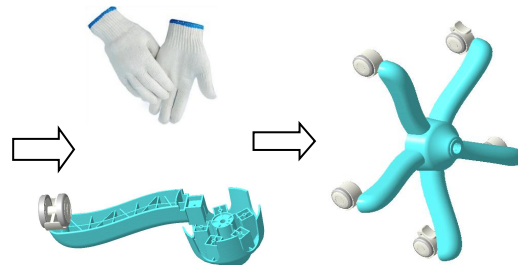

Step 2: Screw the hand screw to lock the inner tube.

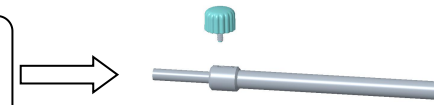


Step 3: Insert the pole horizontally into the middle hole of the base. Attach the weight disc to the bottom of the base. Firstly insert the M8 spring washer into the M8*35 screw, go through the center hole of the weight disc, and use the M6 spanner to fasten the base and pole with screw.

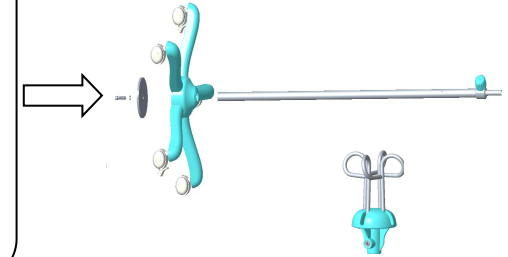

Step 4: Stand up the infusion pole. Loosen the hand screw to pull out the inner tube, adjust it to a suitable length. Lock the hand screw by turning it clockwise (counterclockwise is to loosen).

Step 5: Insert the hook component into the top of the inner tube, use an M3 spanner to tighten the set screw in the hook component.

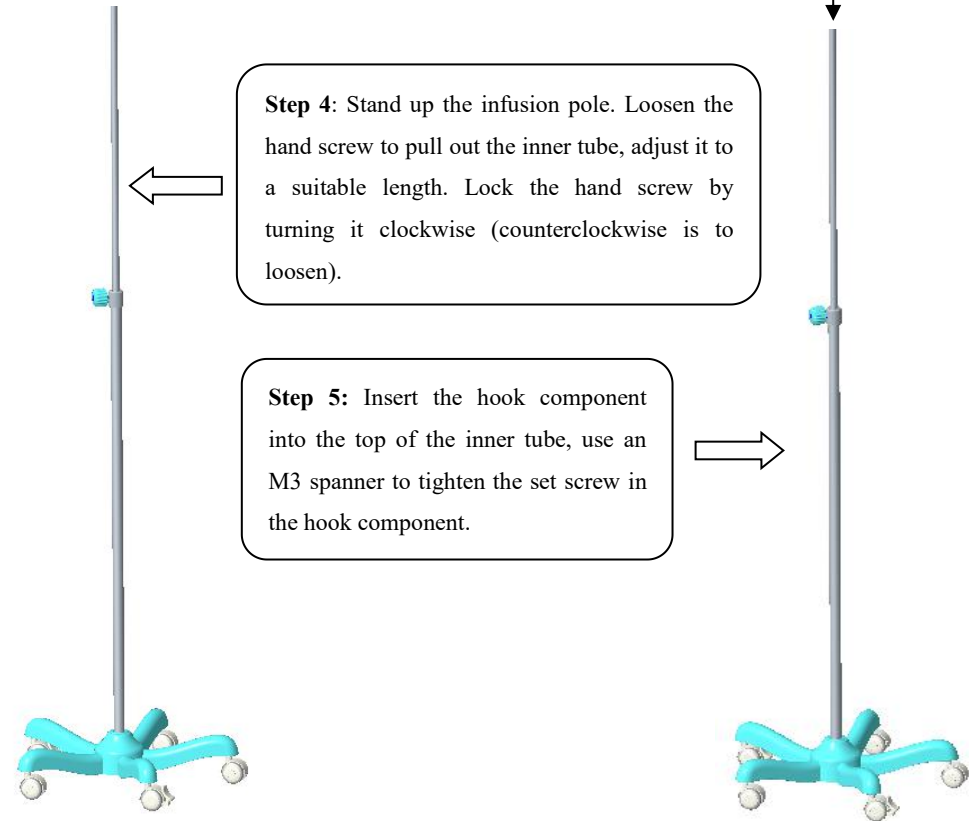

Note: Picture for reference only. The appearance and color subject to the actual product