

MEDIVATORS™ ISA™

Endoscope Reprocessor
Washer-Sterilizer



EVOLUTIONARY PROCESS, REVOLUTIONARY SYSTEM

The **MEDIVATORS ISA Endoscope Reprocessor** provides a distinctive alternative for rapid, efficient reprocessing and has been designed for endoscopy units requiring an advanced technology and high quality standard.

The MEDIVATORS ISA Endoscope Reprocessor provides cold chemical sterilization with Sterility Assurance Level 10^{-6} . As a result of this process, the washer-sterilizer MEDIVATORS ISA Endoscope Reprocessor helps ensure:

- High quality standards for the operator, endoscopes and patients
- Reduction of costs related to hospital infections



FAST & SIMPLE OPERATION FOR IMPROVED EFFICIENCY

- Sterilisation cycle in only 19 minutes with channel monitoring
- Full sterilisation cycle complete with cleaning and channel monitoring in 25 minutes
- Simple, user-friendly handling and operation
- Insertion of endoscopes and operator information through RFID system – helps to avoid errors and save time



MEDIVATORS ISA Endoscope Reprocessor

Assurance of the sterilisation process with a rapid and efficient system

COMPACT & ERGONOMIC DESIGN FOR MAXIMUM USER COMFORT

- Compact size contributes to easy installation, operation and maintenance
- Large and easy to load basin accommodates all types of flexible endoscopes
- Intuitive interface assures fast and easy access to all the cycle information



SAFETY FOR OPERATORS, PATIENTS & ENDOSCOPES

- Hands-free operation and RFID-based technology for improved infection control
- Continuous monitoring of all channels and full leak test traceability through printed and electronic cycle documentation
- Compliant to EN ISO 15883 parts 1, 4 and 5, and to UNI EN ISO 14937. Tested with multiple test soils and in accordance with German cleaning standards.



Foot pedal allows hands-free door lid opening and closing

THE RIGHT FORMULA FOR THE BEST PERFORMANCE & RESULT


- Dedicated room temperature chemicals compatible with endoscope materials
- Improved water and chemical consumption
- Designed specifically for use with MEDIVATORS ISA Endoscope Reprocessor (sterilant compliant to the paragraph 5.3.1 of the UNI EN ISO 14937)



PART OF OUR COMPLETE CIRCLE OF GI REPROCESSING PRODUCTS

Efficient and effective sterilization.

Incorporating STERIS products into every phase of your endoscope reprocessing will help improve department efficiency, reduce the risk of cross-contamination and provide reliable outcomes.

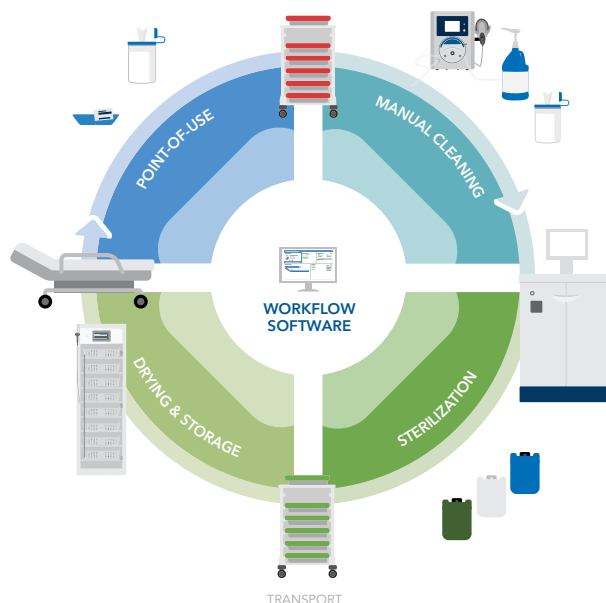

PRODUCT SPECIFICATIONS

Dimensions

700 mm (l) x 1400 mm (h) x 650 mm (d) - monitor included

Weight

75 kg

Electrical Requirements

100 V-240 V, 50-60Hz

Operating Room Temperature

5° - 40° C

Humidity

20 - 80%

Reprocessing Time

25 minutes (complete cleaning and sterilization cycle)

19 minutes (sterilization only cycle)

Process Capacity

1 endoscope per cycle

High Level Disinfectant/Sterilant

ISASPOR™ SINGLE SHOT High-Level Disinfectant and Sterilant, two parts, Peracetic acid and ISAZONE™ Molecule

Detergent

ISACLEAN™ AER Detergent and Decontaminant, Multienzymatic PROTEAZONE™ PLUS Detergent and Cleaner, Enzymatic

| ORDERING INFORMATION | | |
|----------------------|---|-------------------------|
| PART NUMBER | DESCRIPTION | QUANTITY |
| MEDISAWS3 | MEDIVATORS ISA Endoscope Reprocessor Washer-Sterilizer with ISASPOR SINGLE SHOT High-Level Disinfectant and Sterilant 5L and ISACLEAN AER Detergent 1L | 1 x AER |
| MEDISAWS4 | MEDIVATORS ISA Endoscope Reprocessor Washer-Sterilizer with ISASPOR SINGLE SHOT High-Level Disinfectant and Sterilant 10L and ISACLEAN AER Detergent 1L | 1 x AER |
| MEDISAWS6 | MEDIVATORS ISA Endoscope Reprocessor Washer-Sterilizer using ISASPOR SINGLE SHOT High-Level Disinfectant and Sterilant 5L and ISACLEAN AER Detergent 5L | 1 x AER |
| MEDISAWS7 | MEDIVATORS ISA Endoscope Reprocessor Washer-Sterilizer using ISASPOR SINGLE SHOT High-Level Disinfectant and Sterilant 10L and ISACLEAN AER Detergent 5L | 1 x AER |
| MEDISAWS8 | MEDIVATORS ISA Endoscope Reprocessor Washer-Sterilizer using ISASPOR SINGLE SHOT High-Level Disinfectant and Sterilant 10L and PROTEAZONE PLUS Detergent 5L | 1 x AER |
| MEDISAWS9 | MEDIVATORS ISA Endoscope Reprocessor Washer-Sterilizer using ISASPOR SINGLE SHOT High-Level Disinfectant and Sterilant 5L and PROTEAZONE PLUS Detergent 5L | 1 x AER |
| 500187 | ISASPOR SINGLE SHOT High-Level Disinfectant and Sterilant | 10L Part A + 10L Part B |
| 500188 | ISASPOR SINGLE SHOT High-Level Disinfectant and Sterilant | 5L Part A + 5L Part B |
| 4005635 | ISACLEAN AER Detergent and Decontaminant (EMEA 1) | 12 x 1L |
| 4005636 | ISACLEAN AER Detergent and Decontaminant (EMEA 2) | 12 x 1L |
| 4005637 | ISACLEAN AER Detergent and Decontaminant (EMEA 1) | 2 x 5L |
| 4006082 | ISACLEAN AER Detergent and Decontaminant (EMEA 2) | 2 x 5L |
| 500199 | PROTEAZONE PLUS Detergent and Cleaner | 2 x 5L |
| 11061031 | MEDIVATORS ISA Water Filters Kit (0,1µm + 0,45µm) | 1 |
| ISA4400 | ISA Air Filter | 1 |
| 4008833 | MEDIVATORS ISA Water Filter (0,1µm) | 1 |
| 4008832 | MEDIVATORS ISA Water Filter (0,45µm) | 1 |
| ISAROTS2 | ISA Printer Roll | 1 |

Contact your STERIS representative for more information or for the detergent and disinfectant combination available in your country.