AUML

AUML-0427	R 6 x 50 mL + Std 1 x 5 mL
AUML-0497	R 1 x 100 mL + Std 1 x 5 mL
AUML-0507	R 6 x 100 mL + Std 1 x 5 mL
AUML-0707	R 4 x 250 mL + Std 1 x 5 mL
AUML-0420	R 6 x 50 mL
AUML-0500	R 6 x 100 mL
AUML-0250	R 12 x 20 mL
☞ AUML-5405	R 50 mL
☞ AUML-5505	R 100 mL
☞ AUML-5710	R 250 mL
☞ AUML-5220	R 20 mL

FTRO-AUML-v25 (06/2023)_ (PIT-AUML-4-v25)

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☞ BIBLIOGRAFIE

1. Panteghini, M., Bais, R., *Enzymes, Tietz Fundamentals of Clinical Chemistry*, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E., (W.B. Saunders eds.), (2008), 317.
2. Wu, A.H.B., *Tietz Clinical guide to laboratory test*, 4th Ed., (W.B. Saunders eds.), (2006), 64.
3. Dufour, R., *The liver: Function and Chemical Pathology, Clinical Chemistry: Theory Analysis, Correlation*, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 586 and appendix.
4. Schumann, G., et al., *Clin Chem Lab Med.*, (2002), **40**, 718.
5. Guder, W.G., et al., *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
6. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, *Acta Clin Belg.*, (2004), **59**, 263.
7. Young, D.S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed., AACC Press, (1997).
8. Young, D.S., *Effects of drugs on clinical laboratory tests*, 4th Ed., AACC Press, (1995).

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☞ BIBLIOGRAFIE

1. Wu, A.H.B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 66.
2. Dufour D. R., The Liver: Function and Chemical Pathology. Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds), (2010), 586 and appendix.
3. Johnson, A. M., Amino Acids and Proteins. Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 286.

4. Doumas, B. T., et al., Albumin standards and the measurement of serum albumin with bromocresol green, Clin Chim Acta (1971), **31**, 87.

5. Berth, M. & Delanghe, J. Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature, Acta Clin Belg., (2004), **59**, 263.

6. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd edition, AACC Press (1997).

7. Young D.S., Effects of drugs on clinical laboratory tests, 4th edition, AACC Press (1995).

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BIBLIOGRAFIE

1. Higgins, T., et al., *Hemoglobin, Iron, and Bilirubin*, Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R. Bruns, D.E. (W.B. Saunders eds.), (2008), 509.
2. Dufour, D.R. *The liver: Function and Chemical Pathology*, Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds.), (2010), 586 and appendix.
3. Guder, W.G., et al., *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2
4. Wu, A.H.B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 172.
5. Howard, C.R., *Breastmilk & Breastfeeding Jaundice*, Pediatric Clinical Advisor: Instant Diagnosis and Treatment, 2nd Ed., Garfunkel, L.C., Kaczorowski, J.M. & Christy, C., (Mosby Inc. eds.), (2007), 82.
6. Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
7. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd edition, AACC Press (1997).
8. Young D.S., Effects of drugs on clinical laboratory tests, 4th edition, AACC Press (1995).

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ELITech Clinical Systems SAS
Zone Industrielle-61500 SEES Franța
www.elitechgroup.com

Chloride Single Reagent

REF: 233 001 (2 x 25ml) 50 test
REF: ZL-233 001 50 test

Intended Use

Spectrum Chloride reagent is intended for the in-vitro quantitative diagnostic estimation of Chloride in human serum and urine.

Background

Chloride is the most abundant extracellular anion. Together with sodium, chloride is responsible for the maintenance of osmotic pressure, the anion-cation balance and therefore of the water distribution in the extracellular fluid compartment.

Decreased plasma Cl^- concentrations (hypochloremia) can result from salt-losing nephritis, persistent gastric secretion, prolonged vomiting and metabolic acidosis that are caused by increased production or reduced secretion of organic acids.

Increased plasma Cl^- concentrations (hyperchloremia) occur with dehydration, renal tubular acidosis, acute renal failure, in adrenocortical hyperfunction, salicylate intoxication and metabolic acidosis associated with prolonged diarrhoea and loss of sodium bicarbonate. Chloride is often analyzed in combination with Sodium and Potassium to determine the anion gap in serum and/or urine. The urinary anion gap is useful in the initial evaluation of hyperchloremic metabolic acidosis.

Due to the different reactivity equivalents of chloride and bromide the thiocyanate method is less disturbed by the presence of bromide than measurement with an ion-selective electrode.

Method

Colorimetric method.

Assay Principle

The chloride ion displaces thiocyanate from non-ionized mercuric thiocyanate to form Mercuric chloride and thiocyanate ions. The released thiocyanate ions react with ferric ions to form a color complex that absorbs light at 480 nm. The intensity of the color produced is directly proportional to the chloride concentration.

Reagents

Reagent (R)
Hg II - thiocyanate 2 mmol/l
Fe III - nitrate 30 mmol/l
 HNO_3 40 mmol/l

Standard (S)
Chloride 100 mmol/l (354.6 mg/dl)

Precautions and Warnings

The reagent contains mercuric thiocyanate which is toxic and harmful if inhaled or absorbed through skin. Do not ingest or inhale. In case of contact with eyes or skin; rinse immediately with plenty of soap and water. In case of severe injuries; seek medical advice immediately.

Reagent Storage and Stability

Reagents and standard are ready-to-use. When stored at $2 - 8^\circ\text{C}$; they are stable up to the expiry date stated on the label. Once opened, the reagent and standard are stable for 2 months at the specified temperature.

SYMBOLS IN PRODUCT LABELLING

EC REP	Authorised Representative	Use by/Expiration Date
IVD	For in-vitro diagnostic use	CAUTION. Consult instructions for use
LOT	Batch Code/Lot number	
REF	Catalogue Number	Manufactured by
	Consult instructions for use	(Xi) - Irritant
	Temperature Limitation	

Sample

Serum

Freshly drawn non hemolysed serum is the specimen of choice. Chloride in serum is stable for 7 days at $2-8^\circ\text{C}$.

Urine

Urine has to be diluted 1+2 with distilled water. Multiply result by 3.

System Parameters

Wavelength	492 nm (460 - 500 nm)
Optical path	1 cm
Assay type	colorimetric end-point
Direction	Increase
Sample: Reagent Ratio	1:100
e.g.: Reagent volume	1 ml
Sample volume	10 μl
Temperature	25°C , 30°C , 37°C
Zero adjustment	Against reagent blank
Linearity	130 mmol/l (462 mg/dl)
Incubation	5 min.

Procedure

Pipette into clean test tubes:

	Blank	Standard	Sample
Reagent (R)	1 ml	1 ml	1 ml
Standard		10 μl	
Sample			10 μl

Mix well, let stand for 5 minutes, then read absorbances. A standard and A sample against Reagent Blank at 492 nm.

Calculation

$$\text{Serum Chloride Conc. (mmol/l)} = \frac{\Delta A_{\text{Sample}}}{\Delta A_{\text{Standard}}} \times 100$$

Expected Values

Serum	97 – 108 mmol/l.
Urine	24 h urine 95 – 240 mmol/24h
morning urine	54 – 158 mmol/l

Conversion between conventional and SI units: $1 \text{ mEq/l} = 1 \text{ mmol/l}$

Conversion between mmol/l and mg/dl: $\text{mmol/l} = 0.282 \times \text{mg/dl}$

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Methods Comparison

A comparison between Spectrum Chloride reagent and a commercial reagent of the same methodology was performed on 200 human sera. A correlation of 0.980 was obtained.

Sensitivity

When run as recommended, the minimum detection limit of the assay 12 mmol/L

Linearity

The assay is linear up to 130 mmol/l (462 mg/dl)

Interfering substances

Bromide and Fluoride

They can cause falsely elevated chloride values.

Lipemia

Lipemic specimens do not interfere with the test.

Icterus

Icteric serums do not interfere with the reaction.

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References

1. Bablok W. et al. A General Regression Procedure for Method Transformation. J Clin Chem Clin Biochem 1988;26:783-790.
2. Battle DC. et al. The use of the urinary anion gap in the diagnosis of hyperchloremic metabolic acidosis. N Engl J Med 1988, 318:594-599.
3. Krieg M. et al. Comparative quantitative clinico-chemical analysis of the characteristics of 24-hour urine and morning urine (in German). J Clin Chem Clin Biochem 1986, 24:863.
4. Passing H., Bablok W. A New Biometrical Procedure for Testing the Equality of Measurements from Two Different Analytical Methods. J Clin Chem Clin Biochem 1983;21:709-720.
5. Schönfeld, RG. Lewellen, CJ. A colorimetric method for determination of serum chloride. Clin Chem., 10, 533 (1964)
6. Tietz N.W. Clinical Guide to Laboratory Tests, 3rd Philadelphia: W.B. Saunders Company, 1995:516-519.

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BIBLIOGRAFIE

1. Rifai, N., Warnick, G.R., Remaley, A.T., *Lipids, lipoproteins, apolipoproteins and other cardiovascular risk factors*. Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 402.
2. Burnett, J.R., *Coronary Artery Disease: Lipid metabolism*. Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 691 and appendix.
3. Langlois, M.R., *et al.* for the European Atherosclerosis Society (EAS) and the European Federation of Clinical Chemistry and Laboratory Medicine (EFLM) Joint Consensus Initiative, Clin. Chem. Lab. Med., (2020), **58**, 496.
4. Allain, C.C., *et al.*, Clin. Chem., (1974), **20**, 470.
5. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
6. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
7. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press, (1997).
8. Young D.S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press, (1995).

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Attention User: Before using this product, please read the following important information:

The Creatinine Jaffe kit (CRCO-0600) you have received has being modified.

- For the ELITechGroup ECS recommended procedure, the order of addition of reagents has changed. The Sodium Hydroxide (clear /colourless reagent) is now added first (R1). The Alkaline Picrate (yellow coloured reagent) is now the second reagent (R2) added.
- Labelling has been adapted accordingly, and the red hazard pictogram is now located on the R1 bottle (Sodium hydroxide).
- Place “R1” in first position and “R2” in second position on the Selectra Pro analyser.
- The change in the order of reagent addition is not compatible with older lots of aqueous standard CREN-4050, and vice versa. It is necessary to use the standard provided in this kit.
- This modification requires the uploading of a modified instrument application, which is available in the Documentation section of the ELITechGroup website.
- Adjusted values for Elical and Elitol I & II are also provided hereafter and in the Documentation section of the ELITechGroup website.
- All elements of the CREATININE JAFFE system (reagent, standard, application & control recoveries) need to be updated at the same time.

The modification does NOT impact those users that are combining Reagent 1 and Reagent 2 prior to use (mono reagent procedure).

ELICAL 2 - values for "CRCO improved"				
Elical	vial lot	kit lot	expiration	value
ELICAL 2	R200385	03-3130	2022-03	4.03
ELICAL 2	R200946	03-3140	2022-05	
ELICAL 2	R210511	03-3150	2023-01	4.03
ELICAL 2		03-3160	2023-01	
ELICAL 2	R211194	03-3170	2023-06	4.01

ELITROL 1 - values for "CRCO improved"				
Elitrol	vial lot	kit lot	expiration	target(range)
Elitrol 1	R200718 R201112	01-1110	2022-05	0.84(0.69-0.99)
Elitrol 1	R210334 R210590	01-1120 01-1130	2022-10	0.87(0.71-1.03)
Elitrol 1	R210858	01-1140	2022-12	0.87(0.71-1.03)
Elitrol 1	R211203	01-1150	2023-08	0.92(0.75-1.09)

ELITROL 2 - values for "CRCO improved"				
Elitrol	vial lot	kit lot	expiration	target(range)
Elitrol 2	R200719 R201113	02-2110	2022-05	3.27(2.68-3.86)
Elitrol 2	R210335 R210591	02-2120 02-2130	2022-10	3.38(2.77-3.99)
Elitrol 2	R210859	02-2140	2022-12	3.38(2.77-3.99)
Elitrol 2	R211205	02-2150	2023-08	3.37(2.76-3.98)

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BIBLIOGRAFIE

- Kaplan, J.M., First, M.R., *Renal Function. Clinical Chemistry: Theory, Analysis, Correlation*, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds), (2010), 567 and appendix.
- Delaney, M.P., Price, C.P., Lamb, E.J., *Kidney Function and Disease. Tietz Fundamentals of Clinical Chemistry*, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 631.
- Wu, A. H. B., *Clinical guide to laboratory tests*, 4th Ed., (W.B. Saunders eds.), (2006), 316.
- Vasiliades, J., *Clin. Chem.*, (1976), **22**, 1664.
- Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
- Cerioti, F. *et al.*, *Reference Intervals for Serum Creatinine Concentrations: Assessment of Available Data for Global Application, Clin. Chem.*, (2008), **54**, 559.
- Young D.S., *Effects of drugs on clinical laboratory tests*, 4th Ed., AACC Press (1995).
- Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature, Acta Clin Belg.*, (2004), 59, 263.
- Young, D.S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed., AACC Press (1997).

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English - EN

INTENDED USE

This *in vitro* diagnostic device is intended for the calibration of ELITech Clinical Systems quantitative tests listed in the value sheet.
This *in vitro* diagnostic device is for professional use only.

COMPOSITION

- Lyophilized product prepared from human serum spiked with chemical and biological additives.
- Sodium azide < 0.1 % (w/w)
- Concentrations for each analyte to test are lot-specific.

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TRACEABILITY

The traceability is indicated in the value sheet enclosed in the kit.

PRECAUTIONS FOR USE AND WARNINGS

- Each unit of human blood used in the manufacture of these products was tested and found to be negative/non-reactive for the presence of HbsAg, HCV and HIV1/2. The methods used were FDA approved or cleared in compliance with European Directive 98/79/EC, Annex II, List A. Nevertheless, since the risk of infection cannot be fully excluded these products must be handled as potentially infectious. In case of exposure, follow the guidelines of the competent health authorities.
- Take precautions when handling broken glass vials as sharp edges can injure the user.
- These products contain sodium azide which may react with lead or copper plumbing to form potentially explosive metal azides. When disposing of these reagents always flush with copious amounts of water to prevent azide buildup.
- Take normal precautions and adhere to good laboratory practice.
- Use clean or single use laboratory equipment only to avoid contaminations.
- Consult Safety Data Sheet (SDS) for a proper handling.

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PREPARATION

- Carefully open the vial, avoiding the loss of lyophilizate.
- Add in exactly 3 mL of distilled or deionized water.
- Carefully close the vial and dissolve the contents completely within 30 minutes by occasional gentle stirring avoiding the formation of foam.

PRODUCT DETERIORATION

- The product may present a slightly hazy appearance after reconstitution. This has no effect on the performances of the product. All presence of particulates would indicate deterioration.
- Do not use the product if there is visible evidence of contamination or damage (e.g. particle matter after reconstitution).
- Damage to the container may impact on product performance. Do not use the product if there is physical evidence of deterioration (e.g. leakages).

STABILITY

Prior to reconstitution :

- Stored at 2-8 °C and protected from light.
- Do not use after expiration date indicated on the vial label.

After reconstitution :

- These products should be immediately and tightly capped to prevent contamination and evaporation.

- Stability of the components:
Between 15 and 25 °C : 8 hours
Between 2 and 8 °C : 2 days
Between -25 and -15 °C : 4 weeks (when frozen once)

Exceptions:

- Stability of total bilirubin (when stored protected from light):
Between 15 and 25 °C : 6 hours
Between 2 and 8 °C : 1 day
Between -25 and -15 °C : 2 weeks (When frozen once)
- Stability of direct bilirubin (when stored protected from light):
Between 15 and 25 °C : 3 hours
Between 2 and 8 °C : 8 hours
Between -25 and -15 °C : 2 weeks (when frozen once)

Note: Store vials tightly capped and protected from light.

PROCEDURE

To use ELICAL 2, follow the procedure described in the instructions for use of the ELITech Clinical Systems reagent used.

LIMITATIONS

Using ELICAL 2 has been validated with the ELITech systems (Analyzers and reagents listed in the value sheet).

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VALUES

The concentrations are indicated in the value sheet enclosed in the kit.
For users of Selectra instruments allowing data import for tests, calibrators and controls, use corresponding barcode, available on the value sheet.

DECLARATION OF SERIOUS INCIDENT

Please notify the manufacturer (through your distributor) and competent authority of the Member State of the European union in which the user and/or the patient is established, of any serious incident that has occurred in relation to the device. For other jurisdictions, the declaration of serious incident should be in accordance with local, state and federal regulatory requirements. By reporting a serious incident, you provide information that can contribute to the safety of *in vitro* medical devices.

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BIBLIOGRAFIE

1. Sacks, D.B., *Carbohydrates. Tietz Fundamentals of Clinical Chemistry*, 6th Ed., Burtis, C.A, Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 373.
2. Dods, R.F., *Diabetes Mellitus. Clinical Chemistry : Theory, Analysis, Correlation*, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds.), (2010), 729 and appendix
3. Wu, A. H. B., *Clinical guide to laboratory tests*, 4th Ed., (W.B. Saunders eds.), (2006), 444.
4. World Health Organization (WHO), *Definition and Diagnosis of Diabetes Mellitus and Intermediate Hyperglycemia*, (2006).
5. Trinder, P., *Ann. Clin. Biochem.*, (1969), 6, 24.
6. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
7. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, *Acta Clin Belg.*, (2004), 59, 263.
8. Young, D.S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed., AACC Press, (1997).
9. Young, D.S., *Effects of drugs on clinical laboratory tests*, 4th Ed., AACC Press, (1995).

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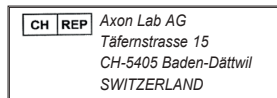
English - EN

❖INTENDED USE

This *in vitro* diagnostic device is intended for the calibration of VitalScientific reagents, used for HDL or LDL cholesterol quantitative determination. This *in vitro* diagnostic device is for professional use only.

COMPOSITION

- Lyophilized product prepared from human serum.
- The HDL and LDL concentrations of this calibrator are lot-specific.

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- VitalScientific reagent, used for HDL or LDL cholesterol quantitative determination (Ref. CHDL-XXXX or CLDL-XXXX)
- General Laboratory equipment.

TRACEABILITY

HDL LDL CALIBRATOR is traceable to the reference method approved by the CDC (Centers for Disease Control and Prevention).

PRECAUTIONS FOR USE AND WARNINGS

- Consult Safety Data Sheet (SDS) for a proper handling.
- Each unit of human blood used in the manufacture of these products was tested and found to be negative/non-reactive for the presence of HbsAg, HCV and HIV1/2. The methods used were FDA approved or CE compliant. Nevertheless, since the risk of infection cannot be fully excluded these products must be handled as potentially infectious. In case of exposure, follow the guidelines of the competent health authorities.
- Take precautions when handling broken glass vials as sharp edges can injure the user.
- Take normal precautions and adhere to good laboratory practice.
- Use clean or single use laboratory equipment only to avoid contaminations.

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PREPARATION

- Carefully open the vial, avoiding the loss of lyophilized.
- Add exactly 1 mL of distilled or deionised water.
- Carefully close the vial and dissolve the contents completely by occasional gentle stirring avoiding the formation of foam.
- Keep at room temperature for 30 minutes before use.

PRODUCT DETERIORATION

- The product may present a slightly hazy appearance after reconstitution. This has no effect on the performances of the product. All presence of particles would indicate deterioration.
- Do not use the product if there is visible evidence of contamination or damage (e.g. particle matter after reconstitution).
- Damage to the container may impact on product performance. Do not use the product if there is physical evidence of deterioration (e.g. leakages).

STABILITY

Prior to reconstitution :

- Store at 2-8 °C and protect from light. Do not freeze.
- Do not use after the expiry date stated on the vial label.

After reconstitution :

- These products are stable 2 weeks at 2-8 °C or 10 weeks at -20°C.
- These products should be immediately and tightly capped to prevent contamination and evaporation.

❖PROCEDURE

To use HDL LDL CALIBRATOR, follow the procedure described in the instructions for use of the VitalScientific reagent used.

❖LIMITATIONS

Using HDL LDL CALIBRATOR has been validated with the VitalScientific systems (Analyzers and reagents used for HDL or LDL cholesterol quantitative determination (Ref. CHDL-XXXX or CLDL-XXXX)). Using any other system should be validated by the laboratory.

VALUES

The concentrations are indicated in the value sheet enclosed in the kit. For users of Selectra instruments allowing data import for tests, calibrators and controls, use the corresponding barcode available on the value sheet.

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- Symbols used on our documentation are defined on ISO-15223-1 standard, except for some presented in the symbols glossary available on the VitalScientific Website. (Symbols glossary).

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IMPORTANT NOTIFICATION

Adjustment of LDL CHOLESTEROL value in HDL LDL CALIBRATOR (ref HLCA-0041)

Document reference: ADD-CLDL-MP-1.1/2021-09

Dear Selectra users,

In order to maintain performances, the LDL CHOLESTEROL value in HDL LDL CALIBRATOR (ref HLCA-0041) has been adjusted for the LDL CHOLESTEROL reagent (ref CLDL-xxxx) **from lot 21-0894 and greater.**

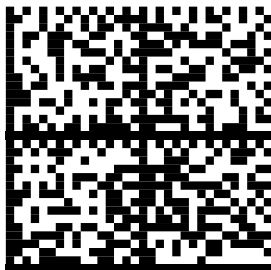
	Reagent lots before lot 21-0894	Reagent lots from lot 21-0894
LDL CHOLESTEROL value in HLCA lot 20-4005	60 mg/dL 1.54 mmol/L	66 mg/dL 1.71 mmol/L
LDL CHOLESTEROL value in HLCA lot 21-4019	62 mg/dL 1.61 mmol/L	67 mg/dL 1.73 mmol/L

TouchPro users on Selectra PRO can scan the barcode on the back.

Results obtained previously with former lots of LDL CHOLESTROL and former value of LDL CHOLESTEROL in HLCA are not questioned.

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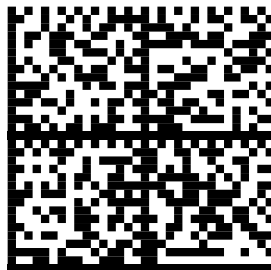
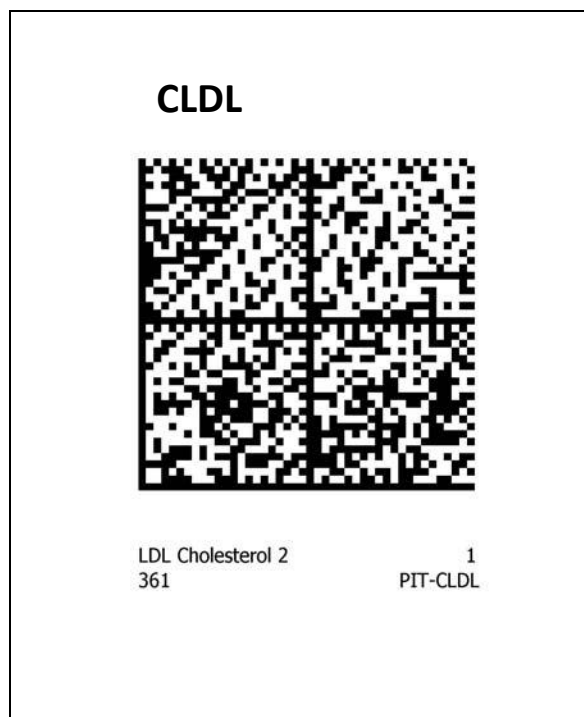
BIBLIOGRAFIE

1. Rifai, N., Warnick, G.R., Remaley, A.T., *Lipids, lipoproteins, apolipoproteins and other cardiovascular risk factors*. Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 402.
2. Burnett, J.R., *Coronary Artery Disease: Lipid metabolism*. Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds), (2010), 691 and appendix.
3. Expert Panel on Detection, Evaluation, and Treatment of High Blood Cholesterol in Adults (Adult Treatment Panel III), JAMA, (2001), **285**, 2486.
4. Wu, A.H.B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 684.
5. Ahmadraji, T. & Killard, A., Analytical Methods, (2013), **5**, 3612.
6. Langlois, M.R., *et al.* for the European Atherosclerosis Society (EAS) and the European Federation of Clinical Chemistry and Laboratory Medicine (EFLM) Joint Consensus Initiative, Clin. Chem. Lab. Med., (2020), **58**, 496.
7. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
8. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press, (1997).
9. Young, D.S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press, (1995).

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Potassium (Single Reagent)

REF: 298 001 (2 x 25ml) 50 test
 REF: 298 002 (4 x 25ml) 100 test
 REF: 298 003 (2 x 100ml) 200 test
 REF: 298 004 (4 x 100ml) 400 test
 REF: ZL-298 001 50 test

Intended Use

Spectrum Potassium reagent is intended for the in-vitro quantitative diagnostic estimation of potassium in human serum or plasma on manual systems.

Background

Sodium and Potassium are the major cations of extracellular and intracellular fluids respectively. Sodium maintains the normal distribution of water and the osmotic pressure in the various fluid compartments. Potassium influences the acid base balance and osmotic pressure including water retention. Increased sodium levels are found in severe dehydration and excessive treatment with sodium salts. Decreased levels are found in severe polyurea, metabolic acidosis, diarrhea and renal insufficiency. Increased potassium levels are found in renal failure, dehydration, shock and adrenal insufficiency. Decreased levels are found in malnutrition, gastrointestinal fluid loss and hyperactivity of the adrenal cortex.

Method

Turbidimetric Tetraphenylborate (TPB)

Assay Principle

At an alkaline pH Potassium ions and TPB form a turbid emulsion, the increase of which can be measured quantitatively in a photometer at 578 nm. The increase of the absorbance (A) is directly proportional to the concentration of Potassium in the sample.

Reagent

Reagent R	NaOH TPB-Na	0.50 mol/L 240 mmol/L
Standard	Potassium	5.0 mmol/L

Precautions and Warnings

Do not ingest or inhale. In case of contact with eyes or skin; rinse immediately with plenty of soap and water. In case of severe injuries; seek medical advice immediately.

Irritant (Xi): R36/38: Irritating to eyes and skin. S26: In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. S37/39: Wear suitable gloves and eye/face protection.

Reagent Storage and Stability

Reagent and standard are ready-to-use and stable till the expiration date stated on the vial label at 2 – 8 °C. Once opened the reagent and the standard are stable for 3 months at the specified temperature.

Caution: Reagent is a microemulsion. Therefore, a slight apparent precipitation could occur, showing a light white deposit at the bottom of the vial. It is a normal behaviour and it is recommended to resuspend solution before analysis with mild shaking.

Specimen Collection and preservation

Human nonhemolyzed serum is the preferred specimen. Heparin is the only acceptable anticoagulant. No other anticoagulant can be used. Do not use lipemic or turbid samples.

SYMBOLS IN PRODUCT LABELLING			
	Authorised Representative		Use by/Expiration Date
	For in-vitro diagnostic use		CAUTION. Consult instructions for use
	Batch Code/Lot number		Manufactured by
	Catalogue Number		(Xi) - Irritant
	Consult instructions for use		
	Temperature Limitation		

System Parameters

Wavelength	578 nm
Optical path	1 cm
Assay type	Colorimetric end-point
Direction	Increase
Sample: Reagent Ratio	1:50
Temperature	25-37 °C
Equilibration Time	30 seconds
Zero adjustment	Against distilled water
Reagent blank	Low 0.0 AU High 1.5 AU
Sensitivity	1.5 mmol/L
Linearity	10 mmol/L

Procedure

	Standard	Sample
Reagent R	1 mL	1 mL
Standard	20 µL	
Sample		20 µL

Mix and incubate for 3 minutes at 37 °C or 5 minutes at 25 °C. Mix again thoroughly and read absorbance of sample (A_{sample}) and standard (A_{standard}) against distilled water.

Calculation

$$\text{Serum Potassium Conc. (mmol/L)} = \frac{A_{\text{Sample}}}{A_{\text{Standard}}} \times 5$$

Quality Control

Normal and abnormal commercial control serum of known concentrations should be analyzed with each run.

Performance Characteristics

Precision

Within run (Repeatability)

	Level 1	Level 2
n	20	20
Mean (mmol/L)	4.1	7.4
SD	0.21	0.3
CV%	5.12	4.05

Run to run (Reproducibility)

	Level 1	Level 2
n	20	20
Mean (mmol/L)	4.1	7.4
SD	0.4	0.5
CV%	9.76	6.76

Methods Comparison

A comparison between Spectrum Potassium reagent and a commercial reagent of the same methodology was performed on 200 human sera. A correlation of 0.978 was obtained.

Sensitivity

When run as recommended, the minimum detection limit of the assay is 1.5 mmol/L.

Linearity

The assay is linear up to 10 mmol/L

Interfering substances

Haemolysis

Hemolyzed sera produce elevated results.

Icterus

No significant interference up to bilirubin level of 40 mg/dL.

Lipemia

Turbid or lipemic samples produce falsely elevated results.

Nitrogen

Urea Nitrogen above 80 mg/dL will produce elevated results. Sera containing high levels of ammonia should be avoided.

Expected Values

Serum 3.6 - 5.5 mmol/L
Plasma 4.0 – 4.8 mmol/L

Note:

It is recommended for each laboratory to establish and maintain its own reference values. The given data are only an indication.

Spectrum Diagnostics does not interpret the results of a clinical laboratory procedure; interpretation of the results is considered the responsibility of qualified medical personnel. All indications of clinical significance are supported by literature references.

Waste Disposal

This product is made to be used in professional laboratories. Please consult local regulations for a correct waste disposal.

S56: dispose of this material and its container at hazardous or special waste collection point.

S57: use appropriate container to avoid environmental contamination.

S61: avoid release in environment. refer to special instructions/safety data sheets.

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BIBLIOGRAFIE

1. Dufour D. R., *The Liver: Function and Chemical Pathology. Clinical Chemistry: Theory, Analysis, Correlation*, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds), (2010), 586 and appendix.
2. Wu, A.H.B., *Clinical guide to laboratory tests*, 4th Ed., (W.B. Saunders eds.), (2006), 916.
3. Dumas, B.T., *et al.*, *Clin. Chem.*, (1981), **27**, 1642.
4. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
5. Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, *Acta Clin Belg.*, (2004), **59**, 263.
6. Young, D.S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed, AACC Press (1997).
7. Young D.S., *Effects of drugs on clinical laboratory tests*, 4th Ed, AACC Press (1995).

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English - EN**INTENDED USE**

These *in vitro* diagnostic devices are intended for the quality control of the performances of ELITech Clinical Systems quantitative tests listed in the value sheet.

These *in vitro* diagnostic devices are for professional use only.

COMPOSITION

- Lyophilized product prepared from human serum spiked with chemical and biological additives.
- Concentrations for each analyte to test are lot-specific.

MATERIALS REQUIRED BUT NOT PROVIDED

- ELITech Clinical Systems reagents listed in the value sheet.
- General Laboratory equipment.

TRACEABILITY

The traceability is indicated on the instructions for use of the ELITech Clinical Systems reagent used in combination with the recommended calibrator.

PRECAUTIONS FOR USE AND WARNINGS

- Each unit of human blood used in the manufacture of these products was tested and found to be negative/non-reactive for the presence of HBsAg, HCV and HIV1/2. The methods used were FDA approved. Nevertheless, since the risk of infection cannot be fully excluded these products must be handled as potentially infectious. In case of exposure, follow the guidelines of the competent health authorities.
- Take precautions when handling broken glass vials as sharp edges can injure the user.
- Take normal precautions and adhere to good laboratory practice.
- Use clean or single use laboratory equipment only to avoid contaminations.
- Consult Safety Data Sheet (SDS) for a proper handling.

WASTE MANAGEMENT

Disposal of all waste material should be in accordance with local, state and federal regulatory requirements (please refer to the Safety Data Sheet (SDS)).

PREPARATION

- Gently tap the vial so that the control material can be at the bottom of the vial.
- Carefully open the vial, avoiding the loss of lyophilized.
- Add exactly 5 mL of distilled or deionized water.
- Carefully close the vial and dissolve the contents completely within 30 minutes by occasional gentle stirring avoiding the formation of foam.

Important: To reactivate the alkaline phosphatase, allow the reconstituted control serum to stand for one hour at 15-25 °C.

PRODUCT DETERIORATION

- The product may present a slightly hazy appearance after reconstitution. This has no effect on the performances of the product. All presence of particules would indicate deterioration.
- Do not use the product if there is visible evidence of contamination or damage (e.g. particle matter after reconstitution).
- Damage to the container may impact on product

performance. Do not use the product if there is physical evidence of deterioration (e.g. leakages).

STABILITY

Prior to reconstitution :

- **Store at 2-8 °C and protect from light. Do not freeze.**
- Do not use after the expiry date stated on the vial label.

After reconstitution :

- These products should be immediately and tightly capped to prevent contamination and evaporation.
- Stability of the components:
Between 2 and 8 °C : 5 days
Between -25 and -15 °C : 4 weeks (when frozen once)

Exceptions:

- Stability of total bilirubin (when stored protected from light) :
Between 2 and 8 °C : 24 hours
Between -25 and -15 °C : 5 days (when frozen once)
- Stability of direct bilirubin (when stored protected from light) :
Between 2 and 8 °C : 8 hours
Between -25 and -15 °C : 5 days (when frozen once)

- Stability of alkaline phosphatase:

- Between 2 and 8 °C : 24 hours**
Between -25 and -15 °C : 4 weeks (when frozen once)

Note:

- *Store the vials tightly capped and protected from light after reconstitution.*
- *Mix content thoroughly prior to use.*
- *Thawing frozen specimen at 37°C, and vigorous mixing (vortex), should be avoided as this may denature ferritin.*

PROCEDURE

To use ELITROL I & II, follow the procedure described in the instructions for use of the ELITech Clinical Systems reagent used.

LIMITATIONS

Using ELITROL I & II has been validated with the ELITech systems (Analyzers and reagents listed in the value sheet).

Using any other system should be validated by the laboratory.

VALUES

The concentrations are indicated in the value sheet enclosed in the kit.

Individual laboratories may obtain values different from those announced. Technique, equipment and experimental error may produce slightly different values.

Each laboratory should determine their own mean values.

For users of Selectra instruments allowing data import for tests, calibrators and controls, use corresponding barcode, available on the value sheet.

DECLARATION OF SERIOUS INCIDENT

Please notify the manufacturer (through your distributor) and competent authority of the Member State of the European union in which the user and/or the patient is established, of any serious incident that has occurred in relation to the device. For other jurisdictions, the declaration of serious incident should be in accordance with local, state and federal regulatory requirements.

By reporting a serious incident, you provide information that can contribute to the safety of *in vitro* medical devices.

ASSISTANCE TECHNIQUE

Contact your local distributor or ELITech Clinical Systems SAS (CCsupport@elitechgroup.com).

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Intended Use

Spectrum Sodium reagent is intended for the in-vitro quantitative diagnostic estimation of sodium in human serum on manual systems.

Background

Sodium and Potassium are the major cations of extracellular and intracellular fluids respectively. Sodium maintains the normal distribution of water and the osmotic pressure in the various fluid compartments. Potassium influences the acid base balance and osmotic pressure including water retention. Increased sodium levels are found in severe dehydration and excessive treatment with sodium salts. Decreased levels are found in severe polyurea, metabolic acidosis, diarrhoea and renal insufficiency. Increased potassium levels are found in renal failure, dehydration, shock and adrenal insufficiency. Decreased levels are found in malnutrition, gastrointestinal fluid loss and hyperactivity of the adrenal cortex.

Method

Colorimetric method.

Assay Principle

The Present method is based on reaction of sodium with a selective chromogen producing a chromophore whose absorbance varies directly as the concentration of sodium in the test specimen.

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Reagent Storage and Stability

Reagents and standard are ready-to-use. When stored at 15 - 25°C; they are stable up to the expiry date stated on the label. Once opened, the reagent and standard are stable for 3 months at the specified temperature.

Deterioration

Failure to recover control values within the assigned range may be an indication of reagent deterioration.

Sample Preparation and Preservation

Serum and plasma

Freshly drawn non-hemolysed serum is the specimen of choice. Heparinised plasma can also be used.

Stability: Serum Sodium is stable for at least 24 hours at room temperature and two weeks at 2-8°C.

Urine

Urine diluted 1+1 with distilled water can be used for Sodium estimation.

SYMBOLS IN PRODUCT LABELLING

	Authorised Representative		Use by/Expiration Date
	For in-vitro diagnostic use		CAUTION. Consult instructions for use
	Batch Code/Lot number		
	Catalogue Number		Manufactured by
	Consult instructions for use		(Xi) - Irritant
	Temperature Limitation		

System Parameters

Wavelength	630 nm
Optical path	1 cm
Assay type	colorimetric end-point
Direction	Increase
Sample: Reagent Ratio	1:100
Temperature	Room temperature
Zero adjustment	Against reagent blank
Sensitivity	55 mEq/l
Linearity	180 mEq/l
Incubation	5 min.
Blank absorbance limit	1.2

Procedure

Pipette into clean test tubes:

	Blank	Standard	Sample
Reagent (R)	1 ml	1 ml	1 ml
Standard		10 µl	
Sample			10 µl

Mix well and let stand for 5 minutes at Room Temperature. Read absorbances, A standard and A sample against Reagent Blank at 630 nm.

Calculation

$$\text{Serum Sodium Conc. (mEq/l)} = \frac{A_{\text{Sample}}}{A_{\text{Standard}}} \times 150$$

Quality Control

Normal and abnormal control serum of known concentrations should be analyzed with each run.

Performance Characteristics

Precision

Within run (Repeatability)

	Level 1	Level 2
n	20	20
Mean (mEq/l)	140	170
SD	0.72	1.44
CV%	0.51	0.84

Run to run (Reproducibility)

	Level 1	Level 2
n	20	20
Mean (mEq/l)	140	170
SD	0.76	1.58
CV%	0.54	0.93

Methods Comparison

A comparison between Spectrum Sodium reagent and a commercial reagent of the same methodology was performed on 200 human sera. A correlation of 0.979 was obtained.

Sensitivity

When run as recommended, the minimum detection limit of the assay 55 mEq/l.

Linearity

The assay is linear up to 180 mEq/l.

Interfering Substances

Hemoglobin and Lithium

Demonstrates positive interference

Lipemia

No significant interference

Other Ions

No adverse influence is exerted on the procedure by blood calcium, chloride and potassium levels of up to 3 times normal values. Hypermagnesemia may interfere with sodium assay.

Anticoagulants

Complexing Anticoagulants such as citrate and oxalate must be avoided.

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BIBLIOGRAFIE

1. Rifai, N., Warnick, G.R., Remaley, A.T., *Lipids, lipoproteins, apolipoproteins and other cardiovascular risk factors*, Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 402.
2. Burnett, J.R., *Coronary Artery Disease: Lipid Metabolism*. Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 691 and appendix.
3. Expert Panel on Detection, Evaluation, and Treatment of High Blood Cholesterol in Adults (Adult Treatment Panel III), *Executive Summary of the Third Report of the National Cholesterol Education Program (NCEP)*, JAMA, (2001), **285**, 2486.
4. Fossati, P. & Prencipe, L., Clin. Chem., (1982), **28**, 2077.
5. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
6. Langlois, M.R., *et al.* for the European Atherosclerosis Society (EAS) and the European Federation of Clinical Chemistry and Laboratory Medicine (EFLM) Joint Consensus Initiative, Clin. Chem. Lab. Med., (2020), **58**, 496.
7. Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
8. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press (1997).
9. Young D.S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press (1995).

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Contactați distribuitorul dumneavoastră local sau ELITech Clinical Systems SAS (ccsupport@elitechgroup.com).