www.thermofisher.com

Resazurin Anaerobic Indicator

EN

REF BR0055B

Intended Use

Resazurin Anaerobic Indicator (BR0055B) is a cotton strip impregnated with resazurin used as a redox indicator. The strip is suitable for detecting redox conditions suitable for the growth of anaerobic organisms. This is an accessory to the anaerobic atmosphere generation systems.

The accessory is for professional use only, is not automated and is not a companion diagnostic.

Summary and Explanation

Anaerobic bacteria are important pathogens that can cause a variety of infections in humans. The site of anaerobic infection is commonly the site of normal colonization. Anaerobic means "life without air." Anaerobic bacteria grow in places which completely, or almost completely, lack oxygen. The spectrum of infections ranges from local abscesses to life-threatening infections. Anaerobic microorganisms, including pathogens such as *Clostridium perfringens*, *Bacteroides fragilis*, *Fusobacterium necrophorum* and *Fusobacterium nucleatum*, can be highly infectious and can cause life-threatening diseases. For example, *Fusobacterium necrophorum* is one of the most commonly isolated species of *Fusobacterium*, and is an uncommon cause of bacteremia, which has a high mortality rate of 10%¹. *Fusobacterium nucleatum* is another member of the *Fusobacterium* species and has been linked to infections such as necrobacillosis and Lemierre's disease².

Principle of Method

Detection of redox conditions suitable for the growth of anaerobic organisms is achieved through the inclusion of resazurin as a redox sensitive, oxidation-reduction indicator. Myacide, Bronopol, fructose, citric acid and ascorbic acid are used as preservatives to prevent spoilage and contamination of the impregnated strip. Disodium hydrogen orthophosphate is present as a biological buffer to maintain the pH of the resazurin solution.

Materials Provided

BR0055B: 100 x indicator strips

Each strip should only be used once.

Materials Required but Not Supplied

Accessories / other in vitro diagnostic medical devices and other products	Description
Anaerogen 2.5L Sachet AN0025A	Anaerogen 2.5L Sachets are anaerobic gas generating sachets for use with Thermo Scientific™ Oxoid 2.5L jar.
Anaerogen 3.5L Sachet AN0035A	Anaerogen 3.5L Sachet is for use with Thermo Scientific™ Oxoid Anaerobic 3.5L jars
Anaerogen Compact AN0010C	Anaerogen Compact is a simple anaerobic gas generator system for the anaerobic incubation of up to

	four petri dishes or an identification panel
Anaerogen W-Zip Compact Gas Generator System AN0010W	Anaerogen W-Zip Compact is a simple anaerobic gas generator system for the anaerobic incubation of up to four petri dishes or an identification panel.
Anaerogen Compact Sachet AN0020D	Anaerogen™ Compact Sachet is a simple anaerobic gas generator system for the anaerobic incubation of up to four petri dishes or an identification panel.

Storage

- Store product in its original packaging at 2–8°C until used.
- The product may be used until the expiry date stated on the label.

Warnings and Precautions

- For in vitro diagnostic use only.
- For professional use only.
- Inspect the product packaging before first use.
- Do not use the product if there is any visible damage to the packaging or plates.
- Do not use the product beyond the stated expiry date.
- Do not use the device if signs of contamination are present.
- Do not use the device if the colour has changed or there are other signs of deterioration.
- It is the responsibility of each laboratory to manage waste produced according to their nature and degree of hazard and to have them treated or disposed of in accordance with any federal, state and local applicable regulations. Directions should be read and followed carefully. This includes the disposal of used or unused reagents as well as any other contaminated disposable material following procedures for infectious or potentially infectious products.

Refer to the Safety Data Sheet (SDS) for safe handling and disposal of the product (www.thermofisher.com).

Serious Incidents

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the relevant regulatory authority in which the user and/or the patient is established.

Specimen Collection, Handling and Storage

Specimens should be collected and handled following local recommended guidelines, such as the UK Standards for Microbiology Investigations (UK SMI) Q 5.

Procedure

After opening, fill gas jar with plates. Ensure that the indicator strip does not touch the side of the jar/ pouch. Place sachets in the gas jar/ pouch as soon as possible. Use Anaerogen™ to attain anaerobic conditions within jar/ pouch and incubate overnight at 37°C.

Quality Control

It is the responsibility of the user to perform Quality Control testing taking into account the intended use of the product, and in accordance with any local applicable regulations (frequency, number of strains, incubation temperature etc.).

It is recommended that an Anaerobic Indicator (BR0055B) is used in the jar/pouch/ W Zip seal pouch as a visual

check that anaerobic conditions have been achieved and maintained.

The user should periodically check their anaerobic system for its ability to provide adequate conditions for the growth of appropriate bacteria.

Limitations

Do not use strips which are dry upon opening the disposable sachet. After incubation of the entire system, if the strip does not change colour to white, the integrity of the system must be checked, and the culture process repeated with a new strip.

Performance Characteristics

Accuracy has been demonstrated through review of the QC data. Correct detection of redox conditions is confirmed by the inclusion of a well-characterised isolate in the QC processes performed as part of the manufacture of each batch of the device. The precision of Resazurin Anaerobic Indicator (BR0055B) was demonstrated by an overall pass rate of 100% obtained for the product over four years of testing (20.12.2017 – 20.01.2022; 10 batches). This shows that the performance is reproducible.

Resazurin Anaerobic Indicator (BR0055B) demonstrates high sensitivity, which was assessed by challenging the indicator strip using the method described.

Resazurin Anaerobic Indicator (BR0055B) is an easy-to-read redox indicator strip that provides rapid, accurate results within 24 hours.

Bibliography

- Centers for Disease Control and Prevention. 2021a. 'Covid-19-Associated *Fusobacterium nucleatum* Bacteremia, Belgium'. 27(3) https://wwwnc.cdc.gov/eid/article/27/3/20-2284_article.
- Public Health England. 2015b. 'Identification of Anaerobic Gram-Negative Rods'. UK Standards for Microbiology Investigations. ID 25 (2). <https://www.gov.uk/government/publications/smi-id-25-identification-of-anaerobic-gram-negative-rods>.

Symbol Legend

Symbol	Definition
	Catalogue number
	In Vitro Diagnostic Medical Device
	Batch code
	Temperature limit
	Use-by date
	Keep away from sunlight
	Do not re-use
	Consult instructions for use or consult electronic instructions for use

	Contains sufficient for <n> tests
	Do not use if packaging damaged and consult instructions for use
	Manufacturer
	Authorized representative in the European Community/ European Union
	European Conformity Assessment
	UK Conformity Assessment
	Unique device identifier
	Importer - To indicate the entity importing the medical device into the locale. Applicable to the European Union
Made in Germany	Made in Germany

ATCC Licensed Derivative[®]

©2022 Thermo Fisher Scientific Inc. All rights reserved. All other trademarks are the property of Thermo Fisher Scientific Inc. and its subsidiaries.

Oxoid Limited, Wade Road, Basingstoke, Hampshire, RG24 8PW, United Kingdom

For technical assistance please contact your local distributor.

Revision Information

Version	Date of modifications introduced
3.0	2024-01-03

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Oxoid Limited, Wade Road, Basingstoke, Hampshire, RG24 8PW, Storbritannien