




Screening

In screening display mode, images in multiple series are observed in multiple frames. Each series is shown in a separate frame.
This enables comparative observations with images of the same slice position in different series (T2WI and T1WI, for example) displayed next to each other.
Cine display and stitching are performed in this display mode.

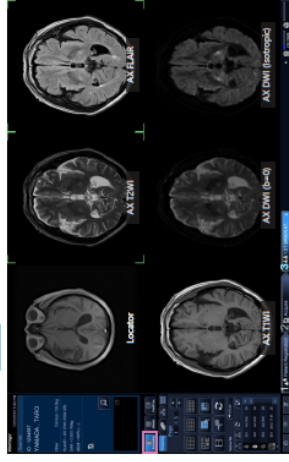

Slices are initially arranged in the order scanned.
You can rearrange the series layout to display series in other frames. You can also select and display individual series by themselves.


Viewing images

All images in a series can be observed within each frame.
Turn the mouse wheel to scroll through and browse images.



Quick menu

Image processing such as rotation and zooming is possible with mouse operations over a certain area of the image frame.
This area at the bottom of each frame is divided into three sections for mouse operations for each function, as shown in the following figure. The menu selected with mouse operations in this area is called the quick menu.

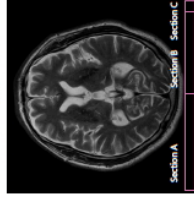

- **Section A**
Click in this section to display the names of post-processing applications that can be executed for the current image.

