Bioline™
TB Ag MPT64

Test for the identification of *Mycobacterium tuberculosis* Complex by rapid test
Test pour l'identification du complexe *Mycobacterium tuberculosis* par test rapide
Identificación del complejo *Mycobacterium tuberculosis* por una prueba rápida
Identificação do complexo *Mycobacterium tuberculosis* através de um teste rápido
Nachweis des *Mycobacterium tuberculosis* -Komplexes mittels Schnelltest
Identificazione del complesso *Mycobacterium tuberculosis* con test rapido

PREPARATION / PRÉPARATION / PREPARACIÓN / PREPARAÇÃO /
VORBEREITUNG / PREPARAZIONE

- 1

EN

Now, open the package and look for the following:

1. Test devices individually foil pouched with a desiccant

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EN

First, read carefully the instruction on how to use the Bioline™ TB Ag MPT 64 test kit.

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EN

Next, look at the expiry date at the back of the foil pouch. Use another kit, if expiry date has passed.

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- 4

EN

Open the foil pouch and look for the following:

1. Test device

2. Desiccant

Then, label the device with the patient identifier.

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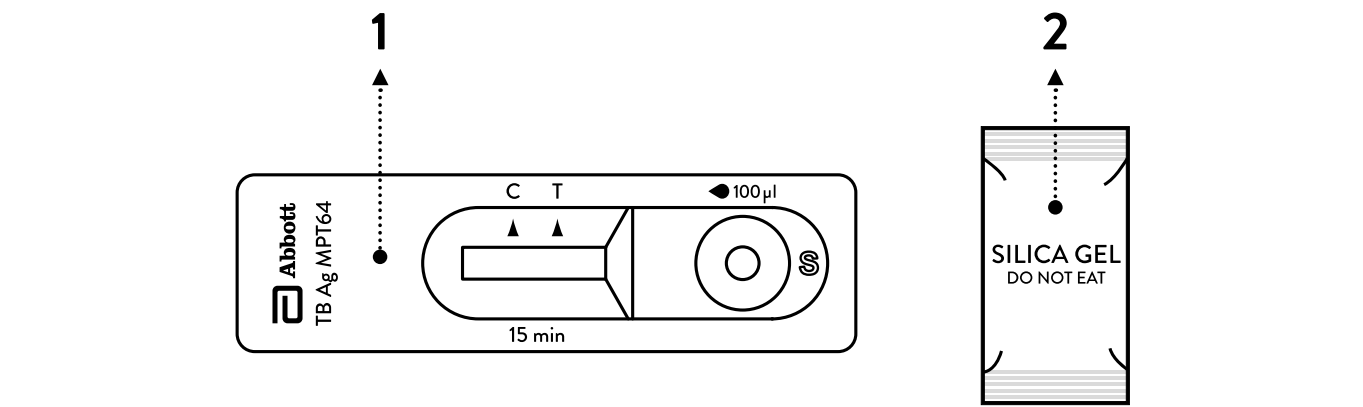
- 1

EN

Take 100 µl of specimen from liquid cultures.

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2

EN

Add 100 µl of liquid cultures into the specimen well.

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T

EN

⚠ **Caution** : The presence of any test line, no matter how faint, the result is considered positive.

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INVALID / NON VALIDE / INVÁLIDO / INVÁLIDO / UNGÜLTIG
/ NON VALIDO

EN

It is recommended that the specimen should be retest

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Doc. No. : TCF-TBM-10
Rev. No. : 6
Rev. Date : Jul. 02, 2020
Page : 1/1

Declaration of Conformity

Manufacturer Abbott Diagnostics Korea Inc.
65, Borahagal-ro, Giheung-gu, Yongin-si, Gyeonggi-do,
17099, Republic of Korea

European Representative MT Promedt Consulting GmbH
Altenhofstrasse 80 66386 St. Ingbert Germany

Product Designation Bioline™ TB Ag MPT64

EDMS Code 15 01 07 01 00 [Mycobacterial Antigen Detection]

Catalogue No. 08FK50

Classification Others ; Self-Declaration IVD
- No self-test
- No annex II List A or List B

Conformity Assessment Route Annex III Applied (IVDD 98/79/EC)

We herewith declare that above mentioned products meet the provisions of the Council Directive 98/79/EC for in vitro diagnostic medical devices. All supporting documentation is retained under the premises of the manufacturer. This declaration of conformity is issued under the sole responsibility of the manufacturer.

Standard Applied List of (Harmonized) standards for which documented evidence for compliance can be provided.

* EN ISO 13485:2016
Medical devices – Quality management systems –
Requirements for regulatory purposes
(ISO 13485:2016) DIN EN ISO 13485:2016
Certificate No. : Q5 043136 0055 Rev. 00

Start of CE marking Sep. 20, 2007

Date of Issue Jul. 02, 2020

On the behalf of
Abbott Diagnostics Korea Inc.

Signature
Jung, Jae-Ho
Site Director

Certificate of Registration

QUALITY MANAGEMENT SYSTEM - ISO 9001:2015

This is to certify that:

Abbott Informatics Corporation
4000 Hollywood Blvd
Suite 333 - S
South Hollywood
Florida
33021
USA

Holds Certificate No:

FM 636368

and operates a Quality Management System which complies with the requirements of ISO 9001:2015 for the following scope:

Design, manufacture, distribution, installation and servicing of Laboratory Information Management Systems software for various industries.

For and on behalf of BSI:

Carlos Pitanga, Chief Operating Officer Assurance – Americas

Original Registration Date: 2016-05-20

Latest Revision Date: 2021-04-22

Effective Date: 2021-06-26

Expiry Date: 2024-06-25

Page: 1 of 1

...making excellence a habit.™

Certificate of Registration

QUALITY MANAGEMENT SYSTEM - ISO 13485:2016

This is to certify that:

Abbott Informatics Corporation
4000 Hollywood Blvd
Suite 333 - S
South Hollywood
Florida
33021
USA

Holds Certificate No:

FM 636367

and operates a Quality Management System which complies with the requirements of ISO 13485:2016 for the following scope:

Design, manufacture, distribution, installation and servicing of Laboratory Information Management Systems software for the medical device industry.

For and on behalf of BSI:

Gary E Slack, Senior Vice President - Medical Devices

Original Registration Date: 2016-05-20

Latest Revision Date: 2021-04-22

Effective Date: 2021-06-26

Expiry Date: 2024-06-25

Page: 1 of 1

...making excellence a habit.™