Scheda tecnica – Technical sheet
Codice Prodotto ANG00001 – Product Code ANG00001
Rev. 13 – 01.02.2023

ANGIODROID THE CO₂ INJECTOR

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ANGIODROID THE CO₂ INJECTOR

Angiodroid The CO₂ Injector is a medical device that allows CO₂ injections at given pressures and volumes into vascular cavities.

CO₂ acts as contrast media during an angiography exam, offering high benefits as well as low complications when compared to the iodinated contrast. The user can set Volume and Pressure values of the injection, to obtain radiological images with a good diagnostic value.

Angiodroid The CO₂ Injector is an automatic digital injector that guarantees a stable level of CO₂ injection pressure and an extremely accurate dose volume setting, as well as a sophisticated control system to avoid contamination with air.

Angiodroid The CO₂ Injector is a multi-functional angiographic device, extremely easy to use, which can be connected directly to the catheter to start the injection.

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PRINCIPAL FEATURES

- Digital volume doses setting
- Digital injection pressure setting
- Fast automatic reload for repeatable injections
- High accuracy of the set injection pressure
- High accuracy of the set volume doses
- Dual microcontroller to ensure high performance and high safety
- Remote controller to start the injection
- Possibility to save injection settings for different body districts
- Touch screen console

Angiodroid The CO₂ Injector allows to perform high quality angiography using a correct setting of the parameters, also in patients with critical diseases such as renal failure,

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diabetes and hypersensitivity to iodinated contrast material.

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APPLICATION FIELDS

Angiodroid The CO₂ Injector has three main application fields:

- interventional radiology
- vascular surgery
- interventional cardiology

DISTRETTI ESAMINABILI

- aorta addominale
- arti inferiori
- fistola emodialitica
- arterie renali
- arterie pelviche
- arterie epatiche
- sistema venoso periferico

INVESTIGABLE DISTRICTS

- abdominal aorta
- lower limbs
- haemodialysis fistula
- renal arteries
- pelvic arteries
- hepatic arteries
- peripheral venous system

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SYSTEM UNIQUENESS

Angiodroid The CO₂ Injector is the only device with a computerized system able to manage each aspect of the injection, starting from the patient safety to the injection parameter settings. The system is characterized by a CO₂ cylinder connected to an internal closed pneumatic circuit in which CO₂ can flush through different pneumatic sectors and a terminal hole for easy connection with the catheter. Each single aspect of the system is completely managed and controlled digitally by means of a central CPU, and different microcontrollers able to drive the pneumatic actuators.

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Fast and Simple

The key features of the system are its velocity and simplicity in executing an injection. The operator is only asked to insert the desired volume dose and injection pressure, using the simple touch screen monitor of the device. Angiodroid The CO₂ Injector needs just these two parameters, and the system will automatically load a dose of CO₂ ready to be injected in the vascular tree. External LEDs, visible even from a certain distance, give a clear information regarding the state of injector in order to correctly identify the CO₂ loading phase or the possibility to start the injection.

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Easy and Immediate Connection to the Catheter

The system is furnished with a terminal hole in which, for each patient, the Single Use Adaptor (disposable line of 250 cm length, with antibacterial filter) can be easily connected. The Single Use Adaptor ends with a luer-lock connector, equipped with a non-return valve (avoiding any possibility of air contamination), suitable for the connection with any kind of catheter.

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Easy to Move

Thanks to 4 wheels with dedicated brakes the device can be easily moved from one room to another without problems and without other devices or kits. It is all in one and stand-alone.

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Repeatable Procedures

The digital system allows the operator to repeat the injection procedure using the same parameters. The reload is very

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fast, just 30 seconds needed to reload the system for a new injection

Patient Safety and Air Contamination Prevention

The system ensures high reliability to avoid air contamination using several sensors placed in each sector of the pneumatic circuit.

The system has been developed to manage the possibility to perform:

- *Low volume injections*
- *Low pressure injections*

These features ensure comfortable injection for the patient, and a better out flow from the catheter, resulting in an improvement of radiological image quality.

Injection System

The system allows different modalities to start the injection, in order to better adapt to each cath lab's requests.

In addition to the possibility to start the injection from the touch screen, the system is equipped with a wireless remote controller that allows to start injections from as far as 15 meters from the injector. This can help reduce the exposure of the operators to the X-ray beams.

Furthermore, it is possible perform a synchronized injection with the X-ray emission thanks to the synchronization cable provided by Angiodroid SpA.*

**synchronization cables are currently available for the following acrs:*

*Ziehm Vision, Ziehm Vision Vario, Ziehm Vision Vario 3D, Ziehm Vision R, Ziehm Vision FD, Ziehm Vision FD Vario, Ziehm Vision FD Vario 3D, Ziehm Vision RFD, Ziehm Vision RFD 3D, Philips (all models equipped with IDS connection interface v1.07)***

****EXCLUSION CONDITION:**

The device Angiodroid The CO₂ Injector is not magnetically compatible with Niobe TM Magnetic Navigation systems. Therefore, the device Angiodroid The CO₂ Injector cannot be used in combination with Niobe Navigation systems and Allura X for R7.7x (a Niobe Navigation only release).

SOFTWARE DESCRIPTION

The software runs in Windows 7 environment and it is shown to the user by means of a touch screen monitor. The main window of the program is extremely user friendly, characterized by a few buttons and text boxes in order to insert just two parameters:

- *Volume of CO₂ dose to inject (in ml)*
- *Pressure of injection (in mmHg)*

Buttons and Text Boxes are quite big and clearly readable to avoid mistakes. However, parameters values can only be edited within a safety range for the patient: values outside the range are blocked by the system, to limit the possibility of errors and the potential damage to the patient.

DISPOSABLE ACCESSORIES

Angiodroid The CO₂ Injector ensures the correct use only if it is used with its disposable accessories:

- *Single Use Adaptor: for the interconnection between the injector and the catheter.*
- *Angiodroid Procedural Kit: for the interconnection between the injector and the catheter, and for the*

ANGIODROID S.p.A.

Head Office: via Speranza, 35 – 40068 San Lazzaro di Savena, Bologna (IT)

Tel/Fax: +39 051 0261383 – email: info@angiodroid.com

Website: <http://www.angiodroid.com>

ANGIODROID® is a trademark property of Angiodroid S.p.A. – Bologna (IT)

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protection of the injector's surfaces

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DIMENSIONS

Weight: 43 kg
Height: 108 cm
Width: 41 cm
Depth: 57 cm

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PROCEDURE MODALITY

Volume Dose setting: Automatic
Injection Pressure Setting: Automatic
Single Use Adaptor Washing: Automatic
Start Injection: Touchscreen / Remote Controller

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INJECTOR SPECIFICATIONS

Classification according to CEI EN-60601-1:2006	Class I – Applied part type CF	
Classification according to Directive CEE 93/42 (2007/47)	Class IIb	
Voltage	110 – 240	VAC
Frequency	50 - 60	Hz
Volume Dose Range	10 - 100	ml
Injection Pressure Range	50 - 750 6 – 100	mmHg KPa
CO ₂ Cylinder*	1,5	Kg
CO ₂ Dose Loading Time	6 - 30	sec
Volume Set Accuracy	<10%	
Pressure Set Accuracy	< 4%	

*only for cylinders provided by Angiodroid SpA.

PRINCIPALI STANDARD APPLICABILI

- Direttiva Europea 93/42/CE concernente i Dispositivi Medici e successive modifiche
- EN 60601-1:2006 / A11:2011/ A1:2013 - Medical electrical equipment - Part 1: General requirements for basic safety and essential performance
- IEC 60601-1-2: 2014 - Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances - Requirements and tests

MAIN APPLICABLE STANDARDS

- European Directive 93/42/CE and subsequent amendments
- EN 60601-1:2006 / A11:2011/ A1:2013 - Medical electrical equipment - Part 1: General requirements for basic safety and essential performance
- IEC 60601-1-2: 2014 - Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances - Requirements and tests

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MANUFACTURING PROCESS

Production processes are conducted according to approved quality system procedures ISO 13485.

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QUALITY CONTROL

Systematic quality controls are performed at different levels: raw material / components income, manufacturing process and product release according to approved quality system procedures.

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STORAGE AND TRANSPORT CONDITIONS

- Ambient temperature between -5 °C and +50 °C
- Keep dry
- Pressure range: 700 hPa - 1060 hPa

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ENVIRONMENTAL WORKING CONDITIONS

- Use indoor in medical facilities
- Ambient temperature between + 5°C and + 35°C
- Relative humidity between 30% and 75%
- Pressure range: 700 hPa – 1060 hPa

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PACKAGING

Cardboard packaging with shaped polystyrene, fixed on wooden pallets.

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LABELLING**IDENTIFICATION DATA**

Code and name of the product, serial number, data of the Manufacturer, the CE Mark with the Notified Body ID, Bar Code for traceability.

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PACKAGING DATA

The symbols for the correct storage and handling of the product, Code and product name, serial number, Manufacturer data, Bar Code for traceability, are shown on the labels for each level of packaging. The symbols used are those suggested by the harmonized international standard UNI CEI EN ISO 15223-1.

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REGULATORY REQUIREMENTS

The product is a class IIb medical device and complies with 93/42/CEE directive and with Italian D.Lg. 46/97

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