



3.2 Overview of the Mistral-Air® Warming Unit (MA1200-QC)

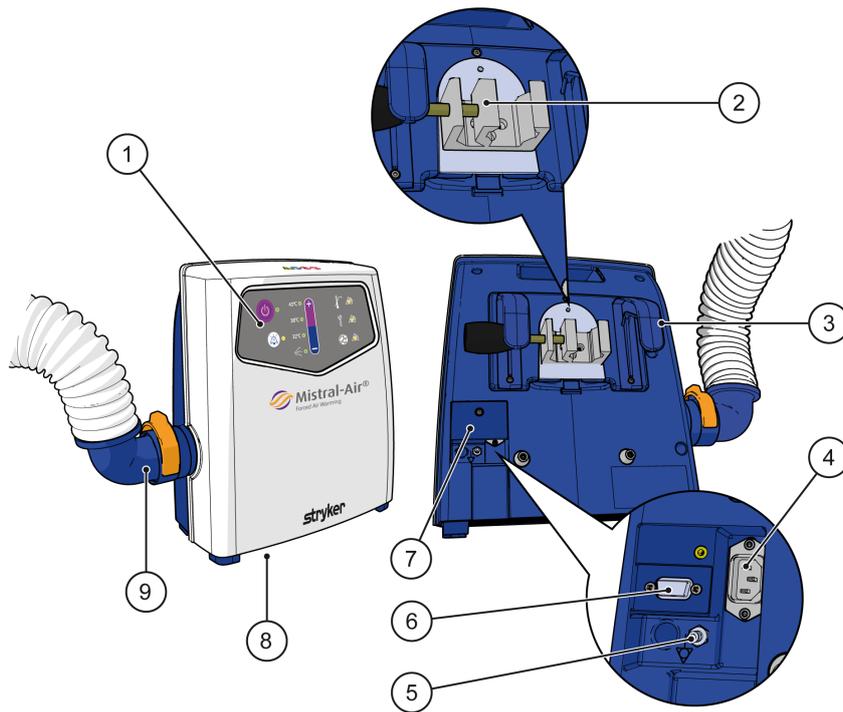

1. Control panel
2. Mounting clamp
3. Mounting options (accessory)
4. Appliance inlet
5. Equipotential pin
6. Data connection
7. Mains cover
8. Filter (air inlet)
9. Removable hose

3.3 Overview of the control panel

S1 The control panel is located at the front top of the device and can be operated by pressure sensitive buttons. The device is easy to use. All settings are visible on the control panel and you can select the preferred temperature by pressing the temperature selection buttons. When an alarm condition is detected, an audible alarm will be activated and an alarm LED will flash yellow.

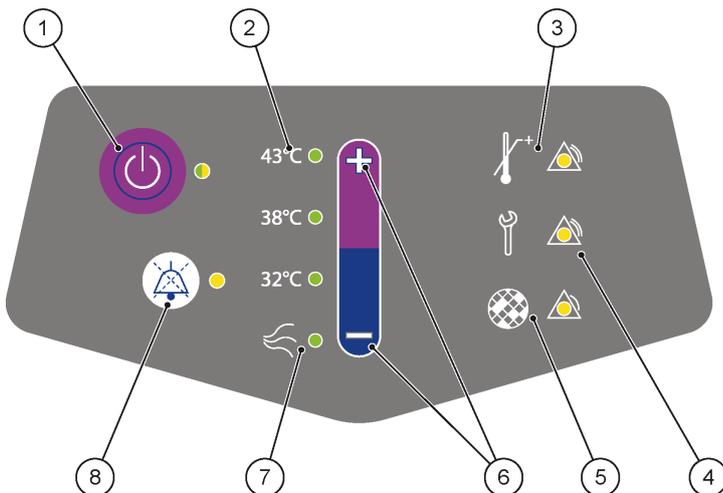

1. Standby button
2. Temperature selection indicators
3. Overtemperature alarm LED
4. Technical alarm LED
5. Filter replacement indicator LED
6. Temperature selection + and – buttons
7. Fan only/ambient air indicator
8. Temporarily audible alarm suppression button