

SELECT LANGUAGE:

Allows operator to select one of two languages to operate Easy Lyte Instrument.

Total Two language options will be available. **ENGLISH/ SPANISH**

Select Language Process.

Press YES to SELECT LANGUAGE? Then Press 'NO' until the desired language is displayed.

Then press YES.

PLEASE CONFIRM! is displayed. To activate the language selected, press YES.

The Easy Lyte will immediately display and print messages in the new language.

To change the Language again, the operator must re-enter

USER OPTIONS? (in the selected language) and proceed to SELECT LANGUAGE?