
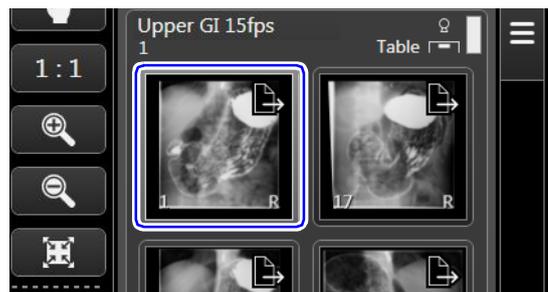
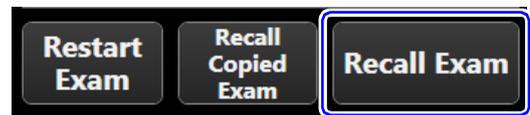
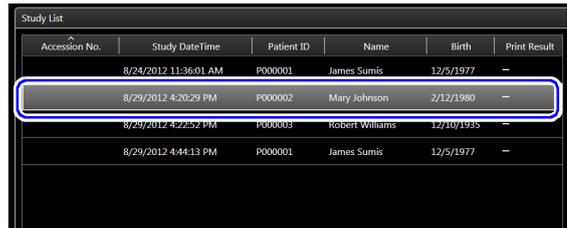


4.8 Post Processing

You can use the buttons on the operation panel to change or process the image displayed on the monitor.

1 Select a patient (examination) image

- 1 Click [PAST].
 - ▶ The Examination List screen will be displayed.
- 2 Click a Patient's name.
 - ▶ The patient (examination) images will be displayed as thumbnail images (reduced images).
- 3 Click [Recall Exam].
 - ▶ The Image Process screen will open and an image will be displayed.
- 4 Select the image to be processed from the thumbnail images.

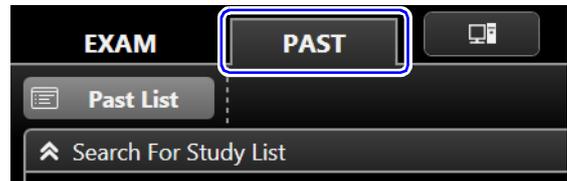

2 Process the image

- 1 Use the following buttons to process images.

Button	Description
 (Zoom In/Out)	Enlarges/reduces the image.
 (Mask)	Blacks out everything but the specified range.
 (Black and White Inversion)	Inverts the black and white colors of the image.
 (Flip Vertical/Horizontal)	Flips the image over in the up/down or rear/front direction.

