


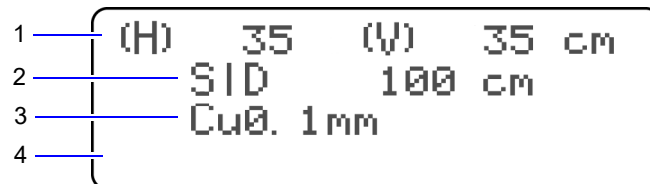
2.3 Operation Panel (RC-300)

When using RC-300, buttons are displayed on the touch panel of the X-ray tube support CH-200. For details, refer to the operation manual of the X-ray tube support CH-200.

 **Reference** CH-200 Operation Manual

2


2.4 Display Panel (R-300)



No.	Name	Function	See
1	Irradiation field	The irradiation field is displayed here. (Units: cm/inch)	P.23
2	SID	The SID, which is the reference for the irradiation field size indication, is displayed here. However, when using a technique that allows fluoroscopy (DR technique), it will not be displayed.	P.14
3	Auto-filter	The type of filter is displayed here.	P.35
4	Message	Error messages and messages in response to operations are displayed here.	P.44

2.5 Display Panel (RC-300)

When using RC-300, indications are displayed on the touch panel of the X-ray tube support CH-200. For details, refer to the operation manual of the X-ray tube support CH-200.

 **Reference** CH-200 Operation Manual