

PAGINA 0

**CREATA PENTRU A
PUTEA SEMNA
ELECTRONIC
DOCUMENTUL**

A clinical review of drug-coated balloons (DCB)

The IN.PACT™ Admiral™ Paclitaxel coated PTA Balloon Catheter provides unparalleled effectiveness and safety, with 75% of patients re-intervention free at five years.¹

▲ Indications, Safety, and Warnings




Overview

Data comes from different individual studies and may differ in a head to head comparison, and therefore may not be predictive of clinical results.

- Highest patency benefit versus PTA
- Consistency in outcomes
- Clinical evidence

Highest patency benefit versus PTA

When comparing long term durability of DCBs to PTA, IN.PACT™ Admiral™ Paclitaxel coated PTA Balloon Catheter has the highest patency benefit and sustained delta through three years.

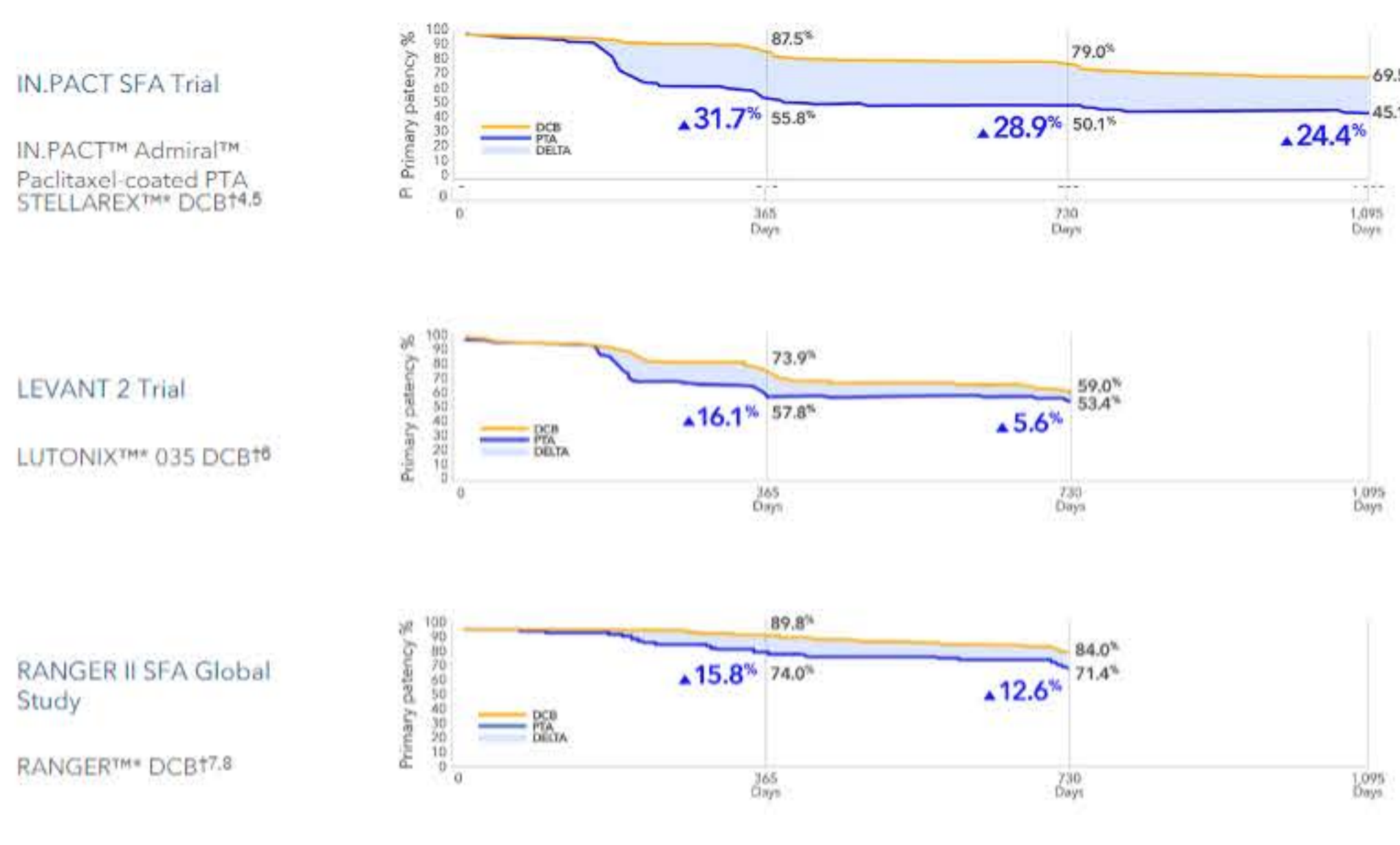


Back to top

Consistency in outcomes

IN.PACT™ Admiral™ Paclitaxel coated PTA Balloon Catheter demonstrates consistent performance across lesion complexity and patient diversity.


IN.PACT Global pre-specified cohorts: freedom from CD-TLR through five years²

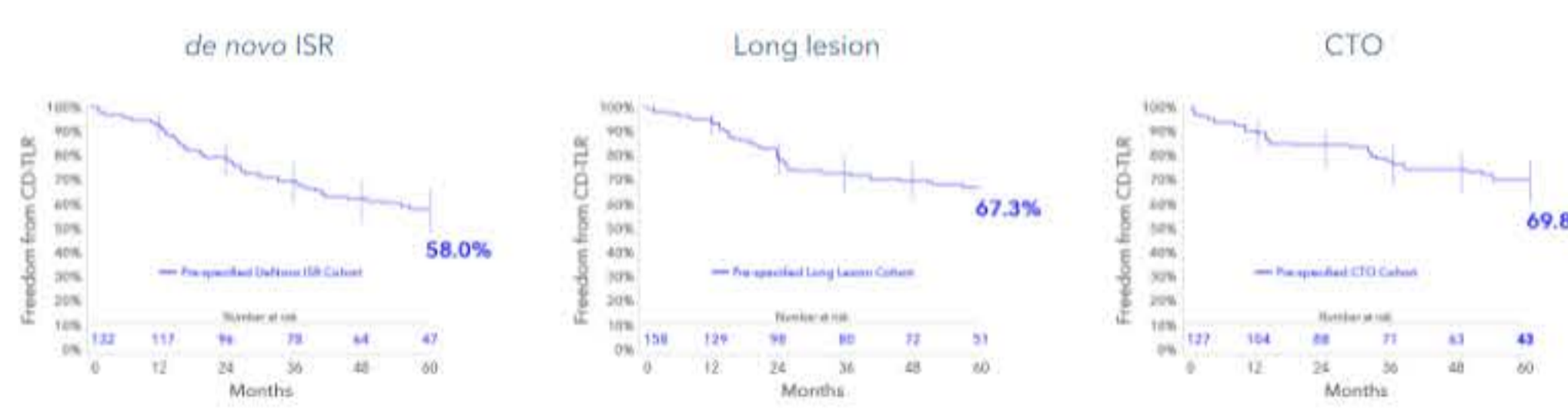

IN.PACT global full cohort five-year freedom from CD-TLR rate: 69.4%²


EXPLORE THE DATA

12-month primary patency in long lesions

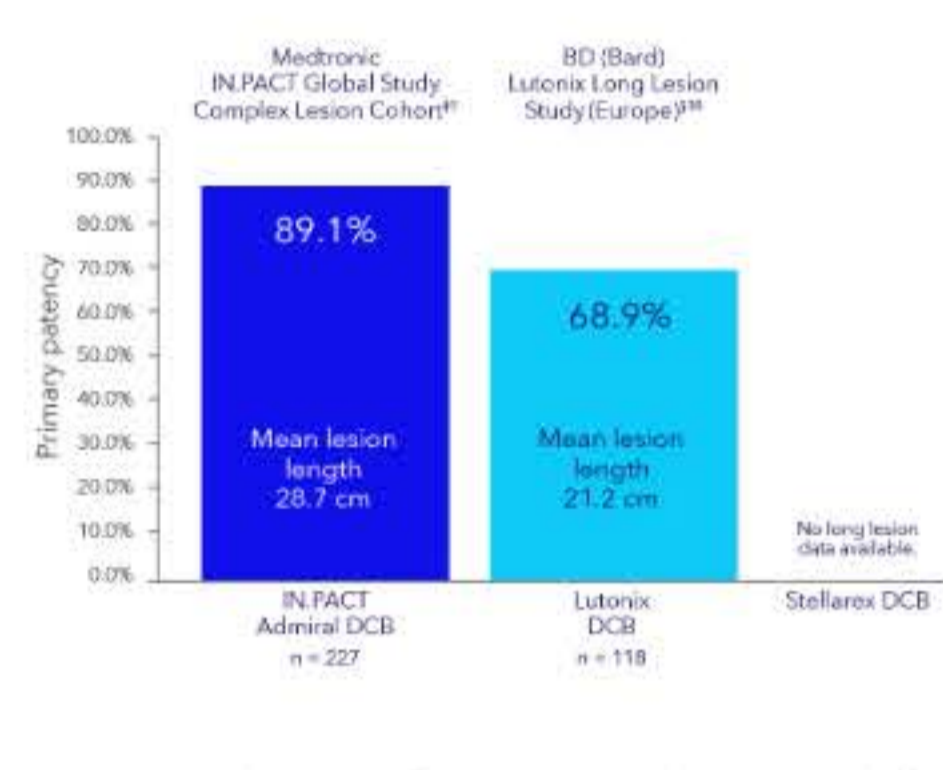


Note: Provisional stent rate for IN.PACT Admiral is 42.5% and is for Lutonix 65.2%.^{9,13}


12-month primary patency in in-stent restenosis (ISR)

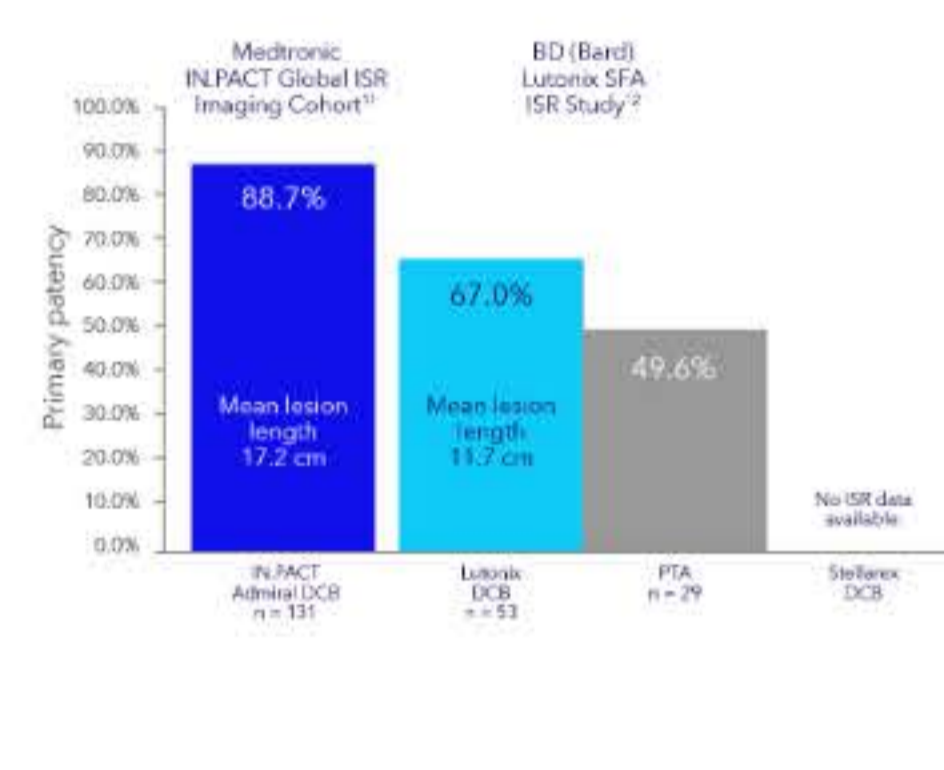

Japan Trial 12-month primary patency

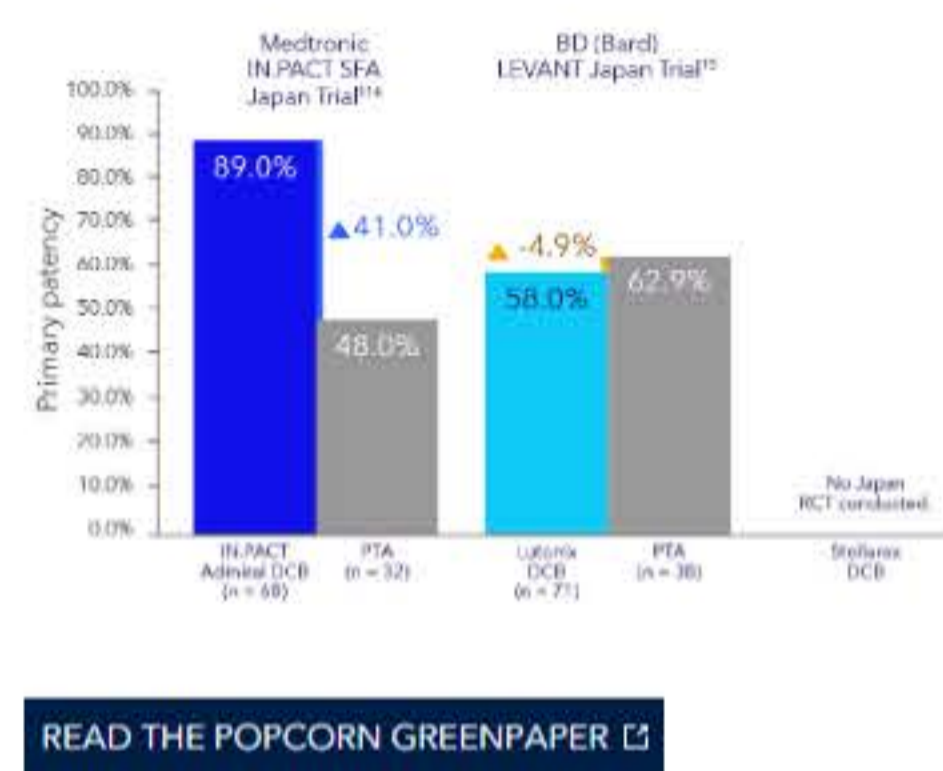

Females 12-month primary patency

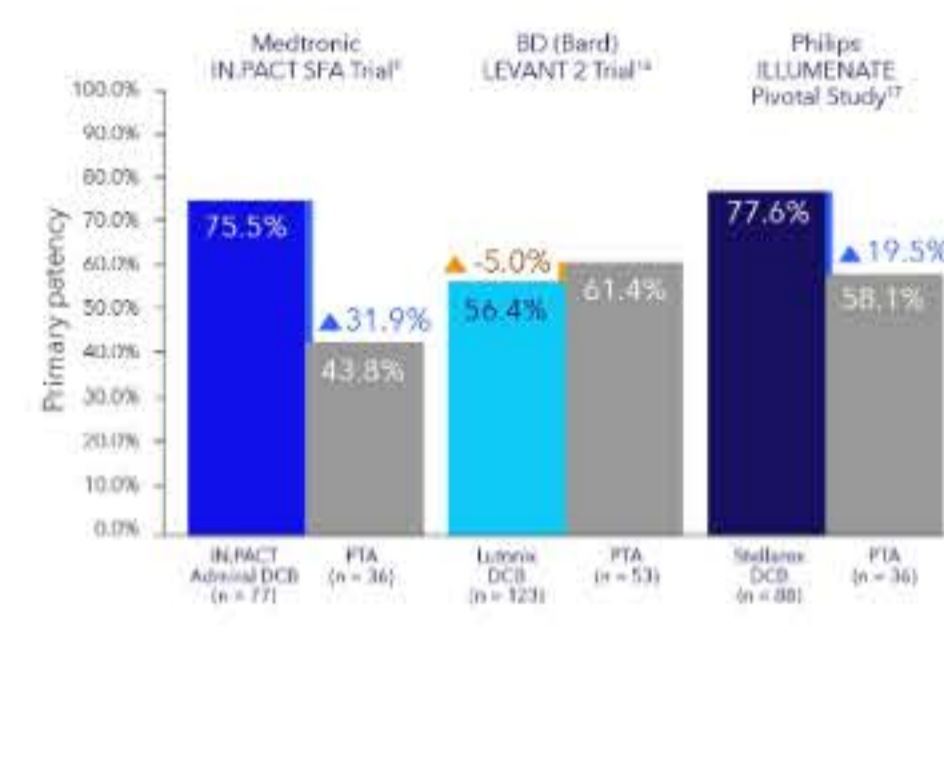


READ THE POPCORN GREENPAPER

Back to top

Clinical evidence

Published data from five IN.PACT trials demonstrates our commitment to clinical evidence.

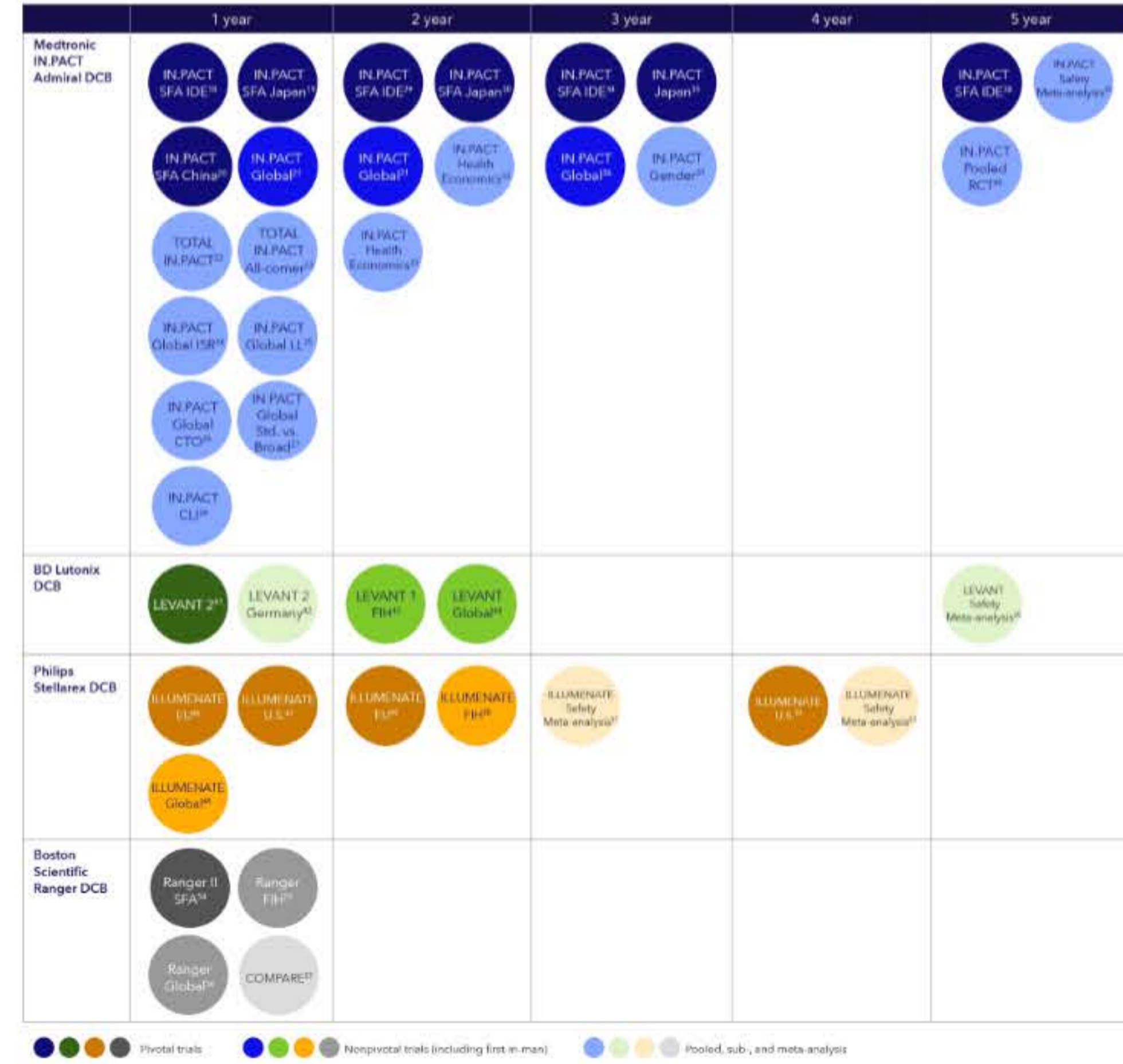

Back to top

Additional resources

Related pages

- IN.PACT™ Admiral™ Paclitaxel coated PTA Balloon Catheter
- Clinical outcomes
- Paclitaxel safety

Related products

- Drug coated balloons
- Cardiovascular products

Connect with us

Reach a Medtronic sales rep
Contact us here

24-hour Technical Support
1-763-514-4000 (worldwide)
rs.cstechsupport@medtronic.com

* Third party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company. The approved product name for the drug-coated balloon is IN.PACT™ Admiral™ Paclitaxel-coated PTA Balloon Catheter.

† Patency rates from clinical trials may be calculated differently. Chart is for illustrative purposes only and results may differ in head-to-head comparison, and therefore may not be predictive of clinical results.

‡ Primary patency is defined as freedom from CEC-adjudicated clinically driven TLR and from core lab-adjudicated binary restenosis. Patency per Kaplan-Meier estimates at 12 months (day 365).

§ Primary patency based on intent-to-treat (ITT) analysis. Primary patency is defined as freedom from clinically driven target lesion revascularization and freedom from restenosis as determined by duplex ultrasound-derived PSVR's 2.4. Indication statement for IN.PACT Admiral (Japan): This device, IN.PACT Admiral Drug Coated Balloon Catheter, is indicated for percutaneous transluminal angioplasty of de novo or restenotic lesion with a reference vessel diameter of 4 mm and 6.5 mm and a length of 15 cm in the native femoropopliteal artery (excluding in-stent lesion) to improve luminal diameter and to reduce restenosis.

References

- 1 Laird JA, Schneider PA, Jull MR, et al. Long-Term Clinical Effectiveness of a Drug-Coated Balloon for the Treatment of Femoropopliteal Lesions: 5-year results from the IN.PACT SFA Trial. *Circ Cardiovasc Interv.* June 2019;12(6):e007702.
- 2 Tepe G. 5-year results from the IN.PACT Global Study Pre-specified Cohorts: ISR, CTO and Long Lesions. Presented at VIVA, 2021.
- 3 Schneider PA, Laird J, Tepe G, et al. Treatment Effect of Drug-Coated Balloons Is Durable to 3 Years in the Femoropopliteal Arteries: Long-Term Results of the IN.PACT SFA Randomized Trial. *Circ Cardiovasc Interv.* January 2018;11(1):e005891.
- 4 Mathews SJ. 1- and 2-Year Outcomes. Presented at NCVI 2018, New Orleans, LA.
- 5 Mathews SJ. 3-Year Outcomes. Presented at NCVI 2019, New Orleans, LA.
- 6 Primary Patency is listed as reported in the IFU. Lutonix BAW13874009 Section 10.3.5 Table 7.
- 7 Brodmann M. 1 Year Outcomes. Presented at LINC 2020, Leipzig, Germany.
- 8 Sachar R. 2 Year Outcomes. Presented at VIVA 2020, Las Vegas, NV.
- 9 IN.PACT Admiral IFU M252A24T001. Rev. 1G.
- 10 Lutonix IFU BAW13874009. Primary patency per KM analysis at day 365.
- 11 Brodmann M, Keese K, Schmeint D, et al. Drug-Coated Balloon Treatment for Femoropopliteal Artery Disease: The IN.PACT Global Study De Novo In-Stent Restenosis Imaging Cohort. *JACC Cardiovasc Interv.* October 23, 2017;10(20):2113-2123.
- 12 Lutonix IFU BAW13874009 - Table 33: Procedural Data (Balloon Spot Stent used Post-DCB Dilatation): 65.2% (45/69).
- 13 Soga Y, Urasawa K, et al. Drug-Coated Balloon vs Standard Percutaneous Transluminal Angioplasty for the Treatment of Atherosclerotic Lesions in the Superficial Femoral and Proximal Popliteal Arteries: One-Year Results of the MDT-2113 SFA Japan Randomized Trial. *J Endovasc Ther.* February 2018;25(1):109-117.
- 14 Bard Data: 1-year outcomes from the LEVANT Japan Trial. Pharmaceuticals and Medical Devices Agency. Available at: http://www.pmda.go.jp/medical_devices/2017/M20170830001/780045000_229002X00252000_A100_1.pdf (in Japanese). Accessed March 23, 2022.
- 15 Stellarex IFU Nu. P011966.
- 16 Rosenfield K, et al. *Circulation*. 2015;131:1495-502.
- 17 Ida O, et al. *J Endovasc Ther*. 2018;25:109-117.
- 18 Chen Z, et al. *J Endovasc Ther*. 2019;26:471-478.
- 19 Zaker T, et al. *Circ Cardiovasc Interv*. 2019;12:e007730.
- 20 Shiahshahmorad MH, et al. *J Vasc Surg*. 2019;70:1177-1191.e9.
- 21 Kobayashi DS, et al. *J Invasive Cardiol*. 2020;32:243-248.
- 22 Laird JR, et al. *J Am Coll Cardiol*. 2015;66:2329-2336.
- 23 Schneider PA, et al. *Catheter Cardiovasc Interv*. 2019;93:668-672.
- 24 Schneider PA, et al. *Circ Cardiovasc Interv*. 2018;11:e005891.
- 25 Soga Y, et al. *J Endovasc Ther*. 2020;27:946-955.
- 26 Laird JA, et al. *Circ Cardiovasc Interv*. 2019;12:e007702.
- 27 Schneider PA, et al. *J Am Coll Cardiol*. 2019;73:2550-2563.
- 28 Brodmann M, et al. *JACC Cardiovasc Interv*. 2017;10:2113-2123.
- 29 Schmeint D, et al. *Circ Cardiovasc Interv*. 2018;11:e005654.
- 30 Tepe G, et al. *JACC Cardiovasc Interv*. 2019;12:484-493.
- 31 Ansel GM, et al. *J Endovasc Ther*. 2018;25:673-682.
- 32 Reijnen MMPJ, et al. *J Endovasc Ther*. 2019;26:305-315.
- 33 Micari A, et al. *JACC Cardiovasc Interv*. 2018;11:945-953.
- 34 Salfubary AC, et al. *JACC Cardiovasc Interv*. 2016;9:2343-2352.
- 35 Patacz JB, et al. *Cardiovasc Intervent Radiol*. 2022;45:298-305.
- 36 Torisello G, et al. *J Endovasc Ther*. 2020;27:693-705.
- 37 Kishi M, et al. *J Vasc Interv Radiol*. 2020;31:1410-1418.e10.
- 38 Schneider PA, et al. *Catheter Cardiovasc Interv*. 2020;98:1087-1099.
- 39 Rosenfield K, et al. *N Engl J Med*. 2015;373:145-153.
- 40 Schmeint D, et al. *J Endovasc Ther*. 2016;23:609-616.
- 41 Schmeint D, et al. *JACC Cardiovasc Interv*. 2014;7:10-19.
- 42 Thorne M, et al. *JACC Cardiovasc Interv*. 2017;10:1682-1690.
- 43 Ornel K, et al. *JACC Cardiovasc Interv*. 2019;12:2315-2324.
- 44 Kishinani P, et al. *Circulation*. 2017;136:1102-1113.
- 45 Schroeder H, et al. *Circulation*. 2017;135:2227-2236.
- 46 Schroy H, et al. *Catheter Cardiovasc Interv*. 2018;91:497-504.
- 47 Brodmann M, et al. *JACC Cardiovasc Interv*. 2016;11:2357-2364.
- 48 Schroeder H, et al. *Catheter Cardiovasc Interv*. 2015;86:278-286.
- 49 Grey WA, et al. *Circulation*. 2019;140:1145-1155.
- 50 Lyden SP, et al. *J Endovasc Ther*. Published online January 8, 2022.
- 51 Lyden SP, et al. *J Vasc Surg*. 2022;75:600-607.
- 52 Sachar R, et al. *JACC Cardiovasc Interv*. 2021;14:1123-1133.
- 53 Steiner S, et al. *JACC Cardiovasc Interv*. 2018;11:934-941.
- 54 Lichsteinberg M, et al. *J Cardiovasc Surg (Torino)*. 2018;59:45-50.
- 55 Steiner S, et al. *Eur Heart J*. 2020;41:2541-2552.