



DISINFECTION SPRAY FOR SURFACES

USER MANUAL

AKASPRAY

- **READY TO USE DISINFECTION WIPE**
- **EFFECTIVE AT 30 SECOND**

Product Properties

- Effective at 30 second.
- Aldehyde and Phenol free.

Usage Areas

AKASPRAY is aldehyde and phenol free and a fast effective alcohol based disinfection spray. It is used at medical equipments (patient beds, doors, medical instruments, cabinets etc.) and surfaces existing at surgery rooms, intensive care rooms, dental units.

Usage form

- AKASPRAY is sprayed from 30 cm distance to area. Wait for effect time. It is possible to be sprayed to a cloth and then apply by wiping. Product makes cleaning and disinfection. Switch off electronic devices before applying their surfaces.

Uyarı: Appy electronic devices after switch off.

Ingredient

%50 Etanol, %10 2-Propanol, %0,05 Didesilmetiloksietilamonyumpropionat

Advantages

- Low effect duration
- Baktericidal (incl. TBC ve MRSA), fungucidal ve virucidal (HIV, HBV, HCV)
- Not Irritant
- Not harmful for surfaces.
- Easy drying and nice odour.

First aid informations:

- In the case of eye contact, wash plenty of water minimum 15 min. Appy to physician.
- In the case of swallowing, not vomit. Drink plenty of water and shake your mouth and throat with water. Immediately appy to physician.

DETRO HEALTHCARE KİMYA SANAYİ A.Ş.

☎ 212 659 77 62 📠 212 659 77 63

✉ info@detrox.com.tr – www.detrox.com.tr