


Lot: 3

Instructions for use

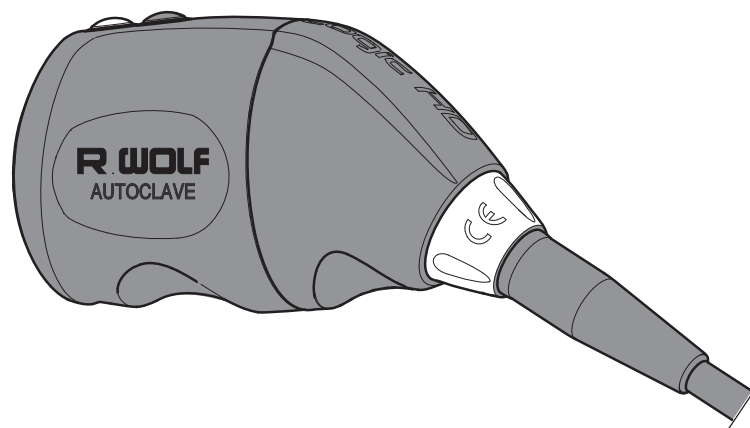

Logic HD camera head

8552592x



BB-A282-03 / en / EU / V2.0 / 2020-12 / PK17-8970

12.1.3 Checks and maintenance

Visual check

1. Check visually for cleanliness. Repeat the processing procedure if necessary until the product is clean.
2. Carry out a visual check.
⇒ Please refer to section 9.1 Visual check

Ensure perfect functionality.

- ◇ Carry out a function check.
⇒ Please refer to section 9.2 Function check

12.1.4 Packaging

- ◇ Place the product in a sterilization basket, e.g. reprocessing basket (model 38002, 38047111).

Packaging Packaging of the medical devices to be sterilized in accordance with ISO 11607-1 taking into account the national requirements.

12.1.5 Sterilization



NOTE

Use suitable sterilization procedure

There are two sterilization procedures for sterilizing the camera head. Depending on the mounted lens, carry out sterilization using the following sterilization procedures:

- **Steam sterilization or low temperature sterilization procedure.**
for camera head with lens (models 85261144, 85261244) or zoom lens (model 85261504).
- **Low temperature sterilization procedure**
for camera head with zoom lens (model 5261504).

Steam sterilization

Sterilize the products using the fractionated prevacuum cycle (ISO 17665) following the corresponding national requirements.

- Temperature exposure time: 3 minutes at 134 °C
- Evacuation phases: 3
- Drying time: 10-20 minutes (The drying time depends on the sterilization process used.)
- Maximum temperature: 138 °C

The prion program (134°C, 18 minutes) is permissible.

14 Replacement parts Lot: 3



NOTE

For the materials required for product reprocessing see section 12 Reprocessing and maintenance.



NOTE

As the camera cable with camera plug is a wear and tear part, any warranty is excluded.

Product no.	Designation
15135463	COVER PLATE
64300090	REPAIR SET CAMERA CABLE 3M
64300091	REPAIR SET CAMERA CABLE 5M
64300094	REPAIR SET FLAT PLUG

15 Disposal



CAUTION

- Reprocess the products before disposal in accordance with the instructions for use to protect the patient, user and third parties.



NOTE

- For the disposal observe the relevant regulations and laws valid in your country.
- For further information please contact the manufacturer.