IT

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EN

DESCRIPTION OF DEVICE:

Sterile bag for tissue storage in liquid nitrogen and/or nitrogen fumes. Class 2 medical device pursuant to Medical Devices Directive on 93/42/EEC and subsequent amendments and additions thereto.

CONTENT:

Each device is packaged in a double blister made of paper and PET/PP.

USE:

Comply strictly with the instructions given in the instruction manual supplied with the product.

To be used exclusively by skilled personnel.

Utilisation temperature: in the range from -196°C to +37°C.

WARNINGS AND PRECAUTIONS:

Use only if the packaging is intact and the device is undamaged. Use immediately after opening the package and removing the protective layers.

Use before the expiration date specified.

Remove needle-free connectors.

Dispose of product according to the applicable national regulations, using appropriate medical waste containers.

STORAGE AND TRACEABILITY:

Store in a clean and dry area, shielded from direct light and free from biological contaminants and chemical vapours.

Period of validity: 5 years from the date of sterilization.

Lot number and expiration date are on the back of the blisters.

Lot number also appears on the primary bag.

PACKAGING:

Each cardboard box contains an envelope in LDPE material with 10 pieces, as described above under "CONTENT", inside.

CODE	SIZE
TC-SMALL	500
TC-LARGE	750

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Paolo Gobbi Frattini S.r.l.